

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005358

Location: ToxL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN MP3003 Lot No. 201610A Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN MP3070 Lot No. 16060 Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. ICS4 Solution 0.10 AC + 0.05% ACETONE ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 17316080A2 Cyl No. 028 Exp. Date 8/5/2018

Test 1 <u>0.078</u>	Test 4 <u>0.079</u>	Test 7 <u>0.079</u>
Test 2 <u>0.078</u>	Test 5 <u>0.079</u>	Test 8 <u>0.078</u>
Test 3 <u>0.079</u>	Test 6 <u>0.079</u>	Test 9 <u>0.079</u>

Average 0.079

- J. Tank pressure, Level 3 Func D Sub G P: 550 psi Reg 550 psi
- K. Remarks/Maintenance record: ANNUAL INSPECTION

5/5/2017
Date

Charles E. Edin
Field Inspector's Signature

8 May 17
Date

Roberta Muga-Nimmo
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005358
Location = TOXL 8164.14.00 09/16
05/05/2017 10:33

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 28
Standard Expiration? 08/05/2018
Oper No? 666666

Flow Cal. Date: 04/17/2014
Slope 622
Intercept -324577

IR Calibration Date: 04/17/2015

	3um	9um
0th Coef(*100):	-16867	-25778
1st Coef(*100):	266890	139601
2nd Coef(*100):	3270	1479
H2O adj(mg/l*10k):	332	443

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks: *PRINT TEST*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005358
Location = TOXL 8164.14.00 09/16
05/05/2017 10:40

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:41
02 Std. Sol.	0.017	10:41
03 Room Air	0.000	10:42
04 Std. Sol.	0.017	10:43
05 Room Air	0.000	10:43
06 Std. Sol.	0.018	10:44
07 Room Air	0.000	10:45

08 Sim Temp = 34.0°C

Simul Ser No = MP3003
Std Sol No = 201610A
County = 08 Oper No. = 666666

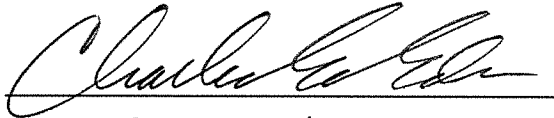

Operator Signature
CHARLES EDER

Remarks:

Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005358
Location = TOXL 8164.14.00 09/16
05/05/2017 10:58

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:59
02 Std. Sol.	0.293	11:00
03 Room Air	0.000	11:00
04 Std. Sol.	0.294	11:01
05 Room Air	0.000	11:02
06 Std. Sol.	0.294	11:02
07 Room Air	0.000	11:03

08 Sim Temp = 34.0°C

Simul Ser No = MP3070
Std Sol No = 16060
County = 08 Oper No. = 666666

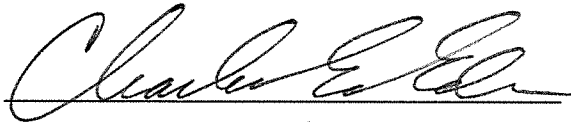

Operator Signature
CHARLES EDER

Remarks: *HIGH AC*
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005358
Location = TOXL 8164.14.00 09/16
05/05/2017 11:05

Test	AC	Time
01 Room Air	0.000	11:06
02 *Subject Test	INT*	11:06
03 Room Air	0.000	11:07

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

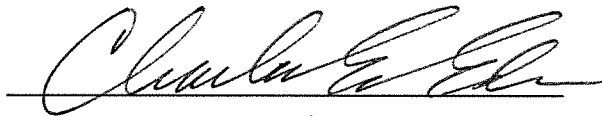

Operator Signature
CHARLES EDER

Remarks:

Interferent
0.10 AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005358
Location = TOXL 8164.14.00 09/16
05/05/2017 11:07

Test	AC	Time
01 Diagnostic	OK	11:08
02 Room Air	0.000	11:09
03 *Subject Test	RFI*	11:09
04 Room Air	0.000	11:09

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005358
Location = TOXL 8164.14.00 09/16
05/05/2017 11:10

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:10
02 Std. Gas	0.078	11:10
03 Room Air	0.000	11:11
04 Std. Gas	0.078	11:11
05 Room Air	0.000	11:12
06 Std. Gas	0.079	11:12
07 Room Air	0.000	11:13

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005358
Location = TOXL 8164.14.00 09/16
05/05/2017 11:13

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:14
02 Std. Gas	0.079	11:14
03 Room Air	0.000	11:15
04 Std. Gas	0.079	11:15
05 Room Air	0.000	11:15
06 Std. Gas	0.079	11:16
07 Room Air	0.000	11:16

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005358
Location = TOXL 8164.14.00 09/16
05/05/2017 11:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:17
02 Std. Gas	0.079	11:18
03 Room Air	0.000	11:18
04 Std. Gas	0.078	11:18
05 Room Air	0.000	11:19
06 Std. Gas	0.079	11:19
07 Room Air	0.000	11:20

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000