

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005949

Location: BSPD  
Bismarck PD

- Check When Done
- A. Is the breath tube heated? X
- B. Is the diagnostic check complete? X
- C. Is the time, date, and year correct (re-set if necessary)? X
- D. Print test. (Attach test record.) X
- E. Low AC. Use  $\leq 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN DR5113 Lot No. 2D1507F Rep AC 0.020 AC X
- F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN DR5578 Lot No. 16060 Rep AC 0.300 AC X
- G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.  
(Attach test record.) X  
Sim SN DR 3808 Lot No. 1CS4 Solution 0.10 AC + 0.05% Acetone
- H. RFI check. Run CMS mode. Key radio during CMS test.  
(Attach test record.) X
- I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.) 0.080 AC

Lot No. 17316080A2 Cyl No. 25 Exp. Date 5 Aug 18

Test 1 0.078 Test 4 0.078 Test 7 0.079

Test 2 0.078 Test 5 0.077 Test 8 0.078

Test 3 0.078 Test 6 0.078 Test 9 0.078

Average 0.078

J. Tank pressure, Level 3 Func D Sub G P: 1003 psi Reg 1000 psi

K. Remarks/Maintenance record: Annual inspection

10 May 17  
Date

16 May 17  
Date

Roberta Huggins-Nimmo  
Field Inspector's Signature

Dob Shanover  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005949  
Location = BSPD                    8164.14.00 09/16  
05/10/2017                            12:07

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=\_|  
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=\_|  
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%&^\*()\_+?

Current Instrument Setup


Data Entry Mode:                    Enabled  
Start Test Sequence:                DABACABA  
Display Prelim Rslt?                Yes  
Display Third Digit?                Yes  
Inhib Printer(Y/N)?                No  
Display Volume?                    No  
Disable On Memfull?                Yes  
# of Print Copies?                  1  
Select Std (D/W/I)?                Dry  
Standard Value?                    0.080  
Standard Lot #?                    17316080A2  
Standard Cyl #?                    25  
Standard Expiration?                08/05/2018  
Oper No?                            888888

Flow Cal. Date:                    06/20/2013  
Slope                                692  
Intercept                            -708987

IR Calibration Date:                04/06/2015  
    3um                                9um

-----  
0th Coef(\*100):                    -24726                                -28868  
1st Coef(\*100):                    263448                                133361  
2nd Coef(\*100):                    3398                                    1497  
H2O adj(mg/l\*10k):                326                                    367

\*\*\*\*\* Printer Test End \*\*\*\*\*

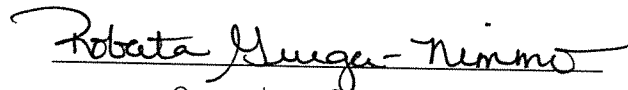

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks:                            Print Test

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-005949  
Location = BSPD                                      8164.14.00 09/16  
05/10/2017    12:23

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:24
02 Std. Sol.	0.018	12:25
03 Room Air	0.000	12:25
04 Std. Sol.	0.018	12:26
05 Room Air	0.000	12:27
06 Std. Sol.	0.018	12:27
07 Room Air	0.000	12:28

08 Sim Temp = 34.0°C

Simul Ser No = DR5113  
Std Sol No = 201507F  
County = 08                                      Oper No. = 888888

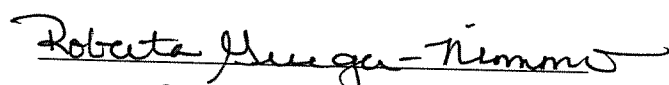  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Low AC  
0.020 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005949  
Location = BSPD                    8164.14.00 09/16  
05/10/2017                            12:29

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:30
02 Std. Sol.	0.302	12:31
03 Room Air	0.000	12:31
04 Std. Sol.	0.303	12:32
05 Room Air	0.000	12:32
06 Std. Sol.	0.305	12:33
07 Room Air	0.000	12:34


08 Sim Temp = 34.0°C

Simul Ser No = DR3378

Std Sol No = 16060

County = 08

Oper No. = 888888

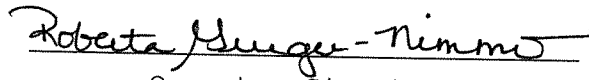

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: High AC  
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005949  
Location = BSPD      8164.14.00 09/16  
05/10/2017      12:39

Test	AC	Time
01 Room Air	0.000	12:40
02 *Subject Test	INT*	12:40
03 Room Air	0.000	12:41

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 25  
Expiration Date = 08/05/2018  
County = 08      Oper No. = 888888

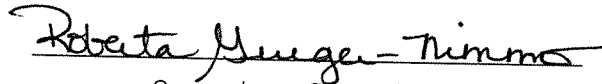

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Interferent Check  
0.10 AC + 0.05% Acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-005949  
Location = BSPD                                      8164.14.00 09/16  
05/10/2017    12:41

Test	AC	Time
01 Diagnostic	OK	12:42
02 Room Air	0.000	12:43
03 Subject Test 1	0.000	12:43
04 Room Air	RFI*	12:44
05 Room Air	0.000	12:45

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                                      Weight = 150  
Test = DUI    Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 25  
Expiration Date = 08/05/2018  
County = 08    Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

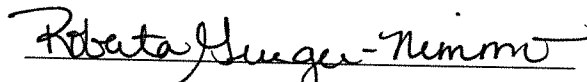  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-005949  
Location = BSPD                              8164.14.00 09/16  
05/10/2017    12:10

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:11
02 Std. Gas	0.078	12:11
03 Room Air	0.000	12:11
04 Std. Gas	0.078	12:12
05 Room Air	0.000	12:12
06 Std. Gas	0.078	12:13
07 Room Air	0.000	12:13


Lot No = 17316080A2

Cyl No = 25

Exp Date = 08/05/2018

County = 08

Oper No. = 888888

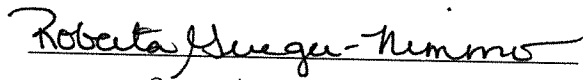

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks:

Calibration Check

0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005949  
Location = BSPD                    8164.14.00 09/16  
05/10/2017                            12:13

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:14
02 Std. Gas	0.078	12:15
03 Room Air	0.000	12:15
04 Std. Gas	0.077	12:15
05 Room Air	0.000	12:16
06 Std. Gas	0.078	12:16
07 Room Air	0.000	12:17

Lot No = 17316080A2

Cyl No = 25

Exp Date = 08/05/2018

County = 08

Oper No. = 888888

*Roberta Grieger-Nimmo*

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check  
0.080 AC

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005949  
Location = BSPD                    8164.14.00 09/16  
05/10/2017                            12:18

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:19
02 Std. Gas	0.079	12:19
03 Room Air	0.000	12:20
04 Std. Gas	0.078	12:20
05 Room Air	0.000	12:21
06 Std. Gas	0.078	12:21
07 Room Air	0.000	12:21

Lot No = 17316080A2

Cyl No = 25

Exp Date = 08/05/2018

County = 08

Oper No. = 888888

*Roberta Grieger-Nimmo*

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check  
0.080 AC

Form 106-I8000