

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005956

Location: Oliver CO SD <sup>OLIV</sup>

- Check When Done
- A. Is the breath tube heated? X
  - B. Is the diagnostic check complete? X
  - C. Is the time, date, and year correct (re-set if necessary)? X
  - D. Print test. (Attach test record.) X
  - E. Low AC. Use  $\leq 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN DR5113 Lot No. 201507F Rep AC 0.020AC X
  - F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN DR3378 Lot No. 16060 Rep AC 0.300AC X
  - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.  
(Attach test record.)  
Sim SN DR3803 Lot No. 1CS4 Solution 0.10 AC + 0.05% X  
Acetone
  - H. RFI check. Run CMS mode. Key radio during CMS test.  
(Attach test record.) X
  - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.) 0.080 AC

Lot No. 19615080A5 Cyl No. 33 Exp. Date 5 Sep 17

Test 1 <u>0.080</u>	Test 4 <u>0.079</u>	Test 7 <u>0.079</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.079</u>
Test 3 <u>0.079</u>	Test 6 <u>0.080</u>	Test 9 <u>0.079</u>

Average 0.079

J. Tank pressure, Level 3 Func D Sub G P: 151 psi Reg 150 psi

K. Remarks/Maintenance record: Annual Inspection.

10 May 17  
Date

10 May 17  
Date

Roberta Hungen-Timmo  
Field Inspector's Signature

Deb Sheaver  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-005956  
Location = OLIV                    8164.14.00 09/16  
05/10/2017                            14:03

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

Current Instrument Setup

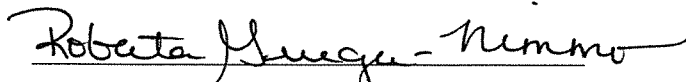
Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            19615080A5  
Standard Cyl #?            33  
Standard Expiration?        09/05/2017  
Oper No?                    888888

Flow Cal. Date:            06/21/2013  
Slope                        706  
Intercept                    -720901

IR Calibration Date:        04/02/2015  
                                    3um                    9um

-----  
0th Coef(\*100):            -23934                    -22166  
1st Coef(\*100):            279293                    133410  
2nd Coef(\*100):            3061                      1559  
H2O adj(mg/l\*10k):        483                        423

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks:    Print Test

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005956  
Location = OLIV      8164.14.00 09/16  
05/10/2017      14:35

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:36
02 Std. Sol.	0.021	14:36
03 Room Air	0.000	14:37
04 Std. Sol.	0.020	14:38
05 Room Air	0.000	14:38
06 Std. Sol.	0.020	14:39
07 Room Air	0.000	14:39

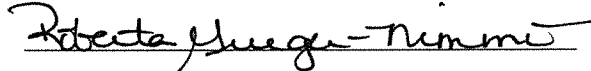
08 Sim Temp = 34.0°C

Simul Ser No = DR5113

Std Sol No = 201507F

County = 33

Oper No. = 888888



Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks:

Low AC  
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

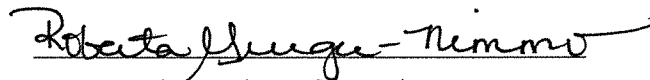
CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005956  
Location = OLIV      8164.14.00 09/16  
05/10/2017      14:42

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:42
02 Std. Sol.	0.306	14:43
03 Room Air	0.000	14:44
04 Std. Sol.	0.306	14:44
05 Room Air	0.000	14:45
06 Std. Sol.	0.305	14:46
07 Room Air	0.000	14:46

08 Sim Temp = 34.0°C

Simul Ser No = DR3378  
Std Sol No = 16060  
County = 33      Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: High AC  
0.300 AC

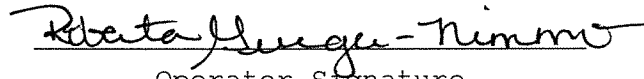
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-005956  
Location = OLIV                    8164.14.00 09/16  
05/10/2017                            14:47

Test	AC	Time
01 Room Air	0.000	14:48
02 *Subject Test	INT*	14:49
03 Room Air	0.000	14:49

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 19615080A5  
Cyl No = 33  
Expiration Date = 09/05/2017  
County = 33                            Oper No. = 888888



Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Interferent Check  
0.10AC + 0.05% Acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

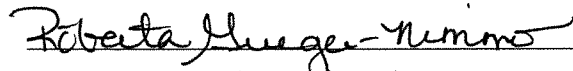
CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-005956  
Location = OLIV                      8164.14.00 09/16  
05/10/2017                              14:50

Test	AC	Time
01 Diagnostic	OK	14:51
02 Room Air	0.000	14:51
03 *Subject Test	RFI*	14:52
04 Room Air	0.000	14:52

\*Invalid Test  
  Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                      Weight = 150  
Test = DUI                              Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 19615080A5  
Cyl No = 33  
Expiration Date = 09/05/2017  
County = 33                              Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check

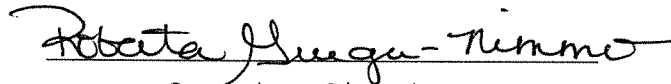
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005956  
Location = OLIV      8164.14.00 09/16  
05/10/2017      14:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:05
02 Std. Gas	0.080	14:06
03 Room Air	0.000	14:06
04 Std. Gas	0.080	14:06
05 Room Air	0.000	14:07
06 Std. Gas	0.079	14:07
07 Room Air	0.000	14:08

Lot No = 19615080A5  
Cyl No = 33  
Exp Date = 09/05/2017  
County = 33      Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check  
0.080 AC

Form 106-I8000

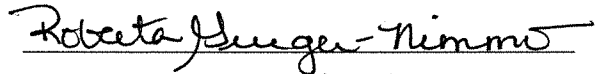
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-005956  
Location = OLIV                    8164.14.00 09/16  
05/10/2017                            14:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:27
02 Std. Gas	0.079	14:28
03 Room Air	0.000	14:28
04 Std. Gas	0.080	14:29
05 Room Air	0.000	14:29
06 Std. Gas	0.080	14:30
07 Room Air	0.000	14:30

Lot No = 19615080A5  
Cyl No = 33  
Exp Date = 09/05/2017  
County = 33                            Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check  
            0.080 AC



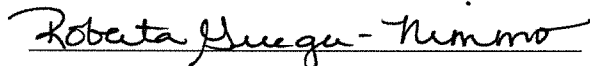
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005956  
Location = OLIV      8164.14.00 09/16  
05/10/2017      14:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:31
02 Std. Gas	0.079	14:31
03 Room Air	0.000	14:32
04 Std. Gas	0.079	14:32
05 Room Air	0.000	14:33
06 Std. Gas	0.079	14:33
07 Room Air	0.000	14:34

Lot No = 19615080A5  
Cyl No = 33  
Exp Date = 09/05/2017  
County = 33      Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check  
0.080 AC

Form 106-I8000