



ETHANOL GAS STANDARD CYLINDER REPORT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>Patrick Renz</i>		
Location <i>BSPD</i>	Intoxilyzer® Serial Number <i>ED-005949</i>	
Gas Lot Number <i>17316080A2</i>	Gas Cylinder Number <i>25</i>	Gas Expiration Date <i>8-05-2018</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. *0.077* AC
 B. *0.078* AC
 C. *0.077* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Pat Renz</i>	Date <i>5-10-17</i>
Reviewed By (Crime Laboratory Use Only) <i>Roberta Hueger-Munno</i>	Date <i>10 May 17</i>


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005949
Location = BSPD 8164.14.00 09/16
05/10/2017 12:02

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:03
02 Std. Gas	0.077	12:03
03 Room Air	0.000	12:04
04 Std. Gas	0.078	12:04
05 Room Air	0.000	12:04
06 Std. Gas	0.077	12:05
07 Room Air	0.000	12:05

Lot No = 17316080A2
Cyl No = 25
Exp Date = 08/05/2018
County = 08 Oper No. = 029710

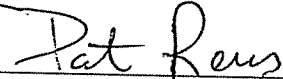

Operator Signature
PATRICK RENZ

Remarks: *New Gas Cylinder install*