

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004944

Location: TUXL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR 3846 Lot No. 16200 Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR 7111 Lot No. 16060 Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR 5135 Lot No. 1CS4 Solution 0.10 AC + 0.05% acetone ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 17316080A2 Cyl No. 27 Exp. Date 08/05/2018

Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.082</u>
Test 2 <u>0.081</u>	Test 5 <u>0.082</u>	Test 8 <u>0.082</u>
Test 3 <u>0.082</u>	Test 6 <u>0.081</u>	Test 9 <u>0.081</u>

Average 0.081

J. Tank pressure, Level 3 Func D Sub G P: 374 psi Reg 3385 psi

K. Remarks/Maintenance record: Annual inspection, replaced o-rings on breath hose, simulator return, cleaned intox, cleaned particulate screen.

11 May 17
Date

Deb Shannon
Field Inspector's Signature

11 May 17
Date

Roberta Siqueira-Nemmo
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004944
Location = ~~WFPD~~ *TOXL* 8164.13.00 06/09
05/11/2017 DS 09:21

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 27
Standard Expiration? 08/05/2018
Oper No? 777777

Flow Cal. Date: 07/01/2011
Slope 685
Intercept -634740

IR Calibration Date: 07/01/2011

	3um	9um
0th Coef(*100):	-22591	-17234
1st Coef(*100):	265915	132691
2nd Coef(*100):	2841	1562
H2O adj(mg/l*10k):	559	425

***** Printer Test End *****

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Print test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004944
Location = TOXL 8164.13.00 06/09
05/11/2017 10:17

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:18
02 Std. Sol.	0.022	10:19
03 Room Air	0.000	10:19
04 Std. Sol.	0.022	10:20
05 Room Air	0.000	10:20
06 Std. Sol.	0.022	10:21
07 Room Air	0.000	10:22

08 Sim Temp = 34.0°C

Simul Ser No = DR3846

Std Sol No = 16200

County = 08

Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Low AC 0.020AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004944
Location = TOXL 8164.13.00 06/09
05/11/2017 10:22

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:23
02 Std. Sol.	0.308	10:24
03 Room Air	0.000	10:24
04 Std. Sol.	0.309	10:25
05 Room Air	0.000	10:25
06 Std. Sol.	0.309	10:26
07 Room Air	0.000	10:27

08 Sim Temp = 34.0°C

Simul Ser No = DR7111

Std Sol No = 16060

County = 08

Oper No. = 777777

Deb Shanaver

Operator Signature

DEB SHANAVER

Remarks:

~~Calibration check~~ High AC 0.300 AC
DS

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004944
Location = TOXL 8164.13.00 06/09
05/11/2017 10:39

Test	AC	Time
01 Room Air	0.000	10:40
02 *Subject Test	INT*	10:40
03 Room Air	0.000	10:41

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

Interferent detected 0.10AC + 0.05% acetone

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004944
Location = TOXL 8164.13.00 06/09
05/11/2017 10:15

Test	AC	Time
01 Diagnostic	OK	10:16
02 Room Air	RFI*	10:16
03 Room Air	0.000	10:17

*Invalid Test
 Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

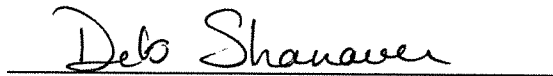

Operator Signature
DEB SHANAVER

Remarks:

RFI check

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004944
Location = ~~WFPD~~ ^{TOXL} 8164.13.00 06/09
05/11/2017 DS 09:18

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:18
02 Std. Gas	0.081	09:19
03 Room Air	0.000	09:19
04 Std. Gas	0.081	09:20
05 Room Air	0.000	09:20
06 Std. Gas	0.082	09:20
07 Room Air	0.000	09:21

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004944
Location = ~~WFPB~~ *TOXL* 8164.13.00 06/09
05/11/2017 *DS* 09:24

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:25
02 Std. Gas	0.081	09:25
03 Room Air	0.000	09:25
04 Std. Gas	0.082	09:26
05 Room Air	0.000	09:26
06 Std. Gas	0.081	09:27
07 Room Air	0.000	09:27

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:
Calibration check 0.080AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004944
Location = TOXL 8164.13.00 06/09
05/11/2017 09:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:52
02 Std. Gas	0.082	09:53
03 Room Air	0.000	09:53
04 Std. Gas	0.082	09:53
05 Room Air	0.000	09:54
06 Std. Gas	0.081	09:54
07 Room Air	0.000	09:55

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 777777


Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC