

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005943

Location: GLDN (GOLDEN VALLEY S.O.)

Check When Done

- A. Is the breath tube heated? ✓
- B. Is the diagnostic check complete? ✓
- C. Is the time, date, and year correct (re-set if necessary)? ✓
- D. Print test. (Attach test record.) ✓
- E. Low AC. Use  $\leq 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN DR5113 Lot No. 201507F Rep AC 0.020 ✓
- F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN DR3378 Lot No. 16060 Rep AC 0.300 ✓
- G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.  
(Attach test record.)  
Sim SN DR3803 Lot No. ICS4 Solution 0.10AC+0.05% ACETONE ✓
- H. RFI check. Run CMS mode. Key radio during CMS test.  
(Attach test record.) ✓
- I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.) 0.080 AC

Lot No. 09216080A3 Cyl No. 039 Exp. Date 5/5/2018  
Test 1 0.082 Test 4 0.081 Test 7 0.080  
Test 2 0.081 Test 5 0.080 Test 8 0.081  
Test 3 0.080 Test 6 0.081 Test 9 0.080  
Average 0.081

J. Tank pressure, Level 3 Func D Sub G P: 728 psi Reg 750 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION, REPLACED

BREATH HOSE O-RING

5/11/2017  
Date

Cecilia E. [Signature]  
Field Inspector's Signature

12 May 17  
Date

Roberto Mungo-Nimmo  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005943  
Location = GLDN                    8164.13.00 06/09  
05/11/2017                            09:43

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?         1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            09216080A3  
Standard Cyl #?            39  
Standard Expiration?       05/05/2018  
Oper No?                    666666

Flow Cal. Date:            06/20/2013  
Slope                        686  
Intercept                   -735246

IR Calibration Date:        08/26/2016  
                              3um                    9um

-----  
0th Coef(\*100):            -13589                    -19547  
1st Coef(\*100):            284566                    139614  
2nd Coef(\*100):            2717                      1336  
H2O adj(mg/l\*10k):        381                        404

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005943  
Location = GLDN      8164.13.00 06/09  
05/11/2017      10:07

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:08
02 Std. Sol.	0.019	10:09
03 Room Air	0.000	10:10
04 Std. Sol.	0.019	10:10
05 Room Air	0.000	10:11
06 Std. Sol.	0.019	10:11
07 Room Air	0.000	10:12

08 Sim Temp = 34.0°C

Simul Ser No = DR5113

Std Sol No = 201507F

County = 17

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

Low AC

0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005943  
Location = GLDN      8164.13.00 06/09  
05/11/2017      10:01

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:01
02 Std. Sol.	0.298	10:02
03 Room Air	0.000	10:03
04 Std. Sol.	0.301	10:03
05 Room Air	0.000	10:04
06 Std. Sol.	0.303	10:05
07 Room Air	0.000	10:05


08 Sim Temp = 34.0°C

Simul Ser No = DR3378

Std Sol No = 16060

County = 17

Oper No. = 666666

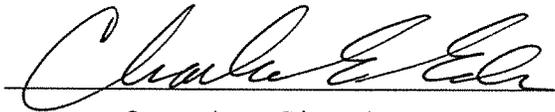

Operator Signature  
CHARLES EDER

Remarks:

HIGH AC  
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005943  
Location = GLDN      8164.13.00 06/09  
05/11/2017      09:54

Test	AC	Time
01 Room Air	0.000	09:55
02 *Subject Test	INT*	09:56
03 Room Air	0.000	09:57

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 09216080A3  
Cyl No = 39  
Expiration Date = 05/05/2018  
County = 17      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *Interferent*  
*0.10AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005943  
Location = GLDN                8164.13.00 06/09  
05/11/2017                        09:57

Test	AC	Time
01 Diagnostic	OK	09:58
02 Room Air	0.000	09:59
03 *Subject Test	RFI*	09:59
04 Room Air	0.000	09:59

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 09216080A3  
Cyl No = 39  
Expiration Date = 05/05/2018  
County = 17                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:    RFI CHECK

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005943  
Location = GLDN      8164.13.00 06/09  
05/11/2017      09:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:48
02 Std. Gas	0.082	09:48
03 Room Air	0.000	09:49
04 Std. Gas	0.081	09:49
05 Room Air	0.000	09:50
06 Std. Gas	0.080	09:50
07 Room Air	0.000	09:51

Lot No = 09216080A3

Cyl No = 39

Exp Date = 05/05/2018

County = 17

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

CALIBRATION CHECK

0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005943  
Location = GLDN      8164.13.00 06/09  
05/11/2017      10:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:15
02 Std. Gas	0.081	10:15
03 Room Air	0.000	10:16
04 Std. Gas	0.080	10:16
05 Room Air	0.000	10:17
06 Std. Gas	0.081	10:17
07 Room Air	0.000	10:18

Lot No = 09216080A3  
Cyl No = 39  
Exp Date = 05/05/2018  
County = 17      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*CALIBRATION CHECK*

*0.080 AC*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005943  
Location = GLDN      8164.13.00 06/09  
05/11/2017      10:18

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:19
02 Std. Gas	0.080	10:19
03 Room Air	0.000	10:20
04 Std. Gas	0.081	10:20
05 Room Air	0.000	10:21
06 Std. Gas	0.080	10:21
07 Room Air	0.000	10:21

Lot No = 09216080A3  
Cyl No = 39  
Exp Date = 05/05/2018  
County = 17      Oper No. = 666666

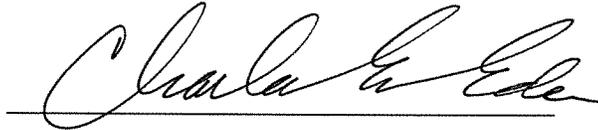

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC