

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-605941

Location: TOXL

- |  | Check When Done |
|--|-----------------|
| A. Is the breath tube heated?  | <u>X</u>        |
| B. Is the diagnostic check complete?   | <u>X</u>        |
| C. Is the time, date, and year correct (re-set if necessary)?  | <u>X</u>        |
| D. Print test. (Attach test record.)   | <u>X</u>        |
| E. Low AC. Use $\leq 0.03$ AC in ACA mode. (Attach test record.)<br>Sim SN <u>DR3846</u> Lot No. <u>16200</u> Rep AC <u>0.020AC</u>  | <u>X</u>        |
| F. High AC. Use $\geq 0.25$ AC in ACA mode. (Attach test record.)<br>Sim SN <u>DR7111</u> Lot No. <u>16060</u> Rep AC <u>0.300AC</u>   | <u>X</u>        |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.<br>(Attach test record.)<br>Sim SN <u>DR5135</u> Lot No. <u>1CS4</u> Solution <u>0.10AC + 0.05%</u><br><span style="margin-left: 150px;">Acetone</span> | <u>X</u>        |
| H. RFI check. Run CMS mode. Key radio during CMS test.<br>(Attach test record.)  | <u>X</u>        |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.<br>(Attach test record.) <u>0.080</u> AC  |                 |

Lot No. 17316080A2 Cyl No. 27 Exp. Date 5 Aug 18

Test 1 0.082 Test 4 0.083 Test 7 0.082

Test 2 0.082 Test 5 0.083 Test 8 0.082

Test 3 0.082 Test 6 0.082 Test 9 0.082

Average 0.082

J. Tank pressure, Level 3 Func D Sub G P: 411 psi Reg 400 psi

K. Remarks/Maintenance record: Annual inspection

12 May 17  
Date

12 May 17  
Date

Roberta Mungu-Nimmo  
Field Inspector's Signature

Deb Shanaver  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-005941  
Location = TOXL                    8164.14.00 09/16  
05/12/2017    10:00

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup

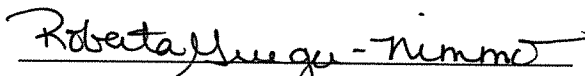
Data Entry Mode:                    Enabled  
Start Test Sequence:                DABACABA  
Display Prelim Rslt?                Yes  
Display Third Digit?                Yes  
Inhib Printer(Y/N)?                No  
Display Volume?                    No  
Disable On Memfull?                Yes  
# of Print Copies?                  1  
Select Std (D/W/I)?                Dry  
Standard Value?                    0.080  
Standard Lot #?                    09216080A3  
Standard Cyl #?                    41  
Standard Expiration?                05/05/2018  
Oper No?                            888888

Flow Cal. Date:                    06/18/2013  
Slope                                695  
Intercept                            -675557

IR Calibration Date:                06/18/2013  
    3um                                9um

-----  
0th Coef(\*100):                    -20306                                -25137  
1st Coef(\*100):                    291687                                138772  
2nd Coef(\*100):                    2450                                    1058  
H2O adj(mg/l\*10k):                562                                    452

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Print test

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

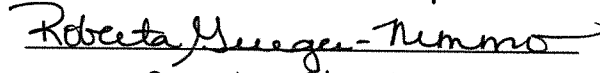
CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005941  
Location = TOXL      8164.14.00 09/16  
05/12/2017      10:01

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:02
02 Std. Sol.	0.019	10:02
03 Room Air	0.000	10:03
04 Std. Sol.	0.019	10:04
05 Room Air	0.000	10:04
06 Std. Sol.	0.020	10:05
07 Room Air	0.000	10:05

08 Sim Temp = 34.0°C

Simul Ser No = DR3846  
Std Sol No = 16200  
County = 08      Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: low AC  
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

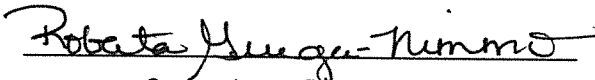
CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005941  
Location = TOXL      8164.14.00 09/16  
05/12/2017      10:07

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:07
02 Std. Sol.	0.300	10:08
03 Room Air	0.000	10:09
04 Std. Sol.	0.301	10:09
05 Room Air	0.000	10:10
06 Std. Sol.	0.301	10:11
07 Room Air	0.000	10:11

08 Sim Temp = 34.0°C

Simul Ser No = DR7111  
Std Sol No = 16060  
County = 08      Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: High AC  
0.300 AC

Form 106-I8000

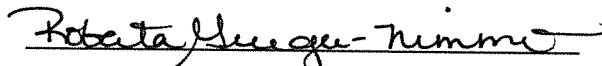
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005941  
Location = TOXL                    8164.14.00 09/16  
05/12/2017                            10:15

Test	AC	Time
01 Room Air	0.000	10:16
02 *Subject Test	INT*	10:17
03 Room Air	0.000	10:18

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 27  
Expiration Date = 08/05/2018  
County = 08                            Oper No. = 888888



Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Interferent Check  
0.100 AC + 0.05% Acetone

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

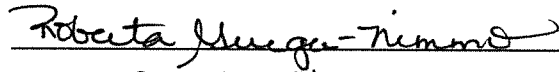
CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005941  
Location = TOXL                 8164.14.00 09/16  
05/12/2017                        10:19

Test	AC	Time
01 Diagnostic	OK	10:20
02 Room Air	0.000	10:21
03 *Subject Test	RFI*	10:21
04 Room Air	0.000	10:21

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 27  
Expiration Date = 08/05/2018  
County = 08                            Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check

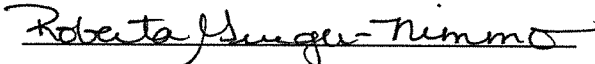
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005941  
Location = TOXL      8164.14.00 09/16  
05/12/2017      10:22

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:22
02 Std. Gas	0.082	10:23
03 Room Air	0.000	10:23
04 Std. Gas	0.082	10:23
05 Room Air	0.000	10:24
06 Std. Gas	0.082	10:24
07 Room Air	0.000	10:25

Lot No = 17316080A2  
Cyl No = 27  
Exp Date = 08/05/2018  
County = 08      Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check  
0.080 AC  
Form 106-I8000

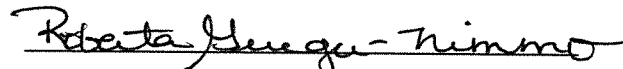
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005941  
Location = TOXL      8164.14.00 09/16  
05/12/2017      10:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:26
02 Std. Gas	0.083	10:26
03 Room Air	0.000	10:27
04 Std. Gas	0.083	10:27
05 Room Air	0.000	10:27
06 Std. Gas	0.082	10:28
07 Room Air	0.000	10:28

Lot No = 17316080A2  
Cyl No = 27  
Exp Date = 08/05/2018  
County = 08      Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check  
0.080 AC  
Form 106-I8000



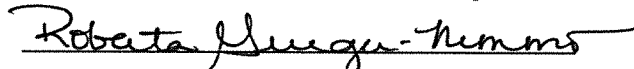
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005941  
Location = TOXL                    8164.14.00 09/16  
05/12/2017                            10:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:29
02 Std. Gas	0.082	10:30
03 Room Air	0.000	10:30
04 Std. Gas	0.082	10:30
05 Room Air	0.000	10:31
06 Std. Gas	0.082	10:31
07 Room Air	0.000	10:32

Lot No = 17316080A2  
Cyl No = 27  
Exp Date = 08/05/2018  
County = 08                            Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks:

Calibration Check  
0.080 AC

Form 106-I8000