

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006499

Location: JAME

- | | Check When Done |
|---|-----------------|
| A. Is the breath tube heated? | <u>X</u> |
| B. Is the diagnostic check complete? | <u>X</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>X</u> |
| D. Print test. (Attach test record.) | <u>X</u> |
| E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5113</u> Lot No. <u>201507F</u> Rep AC <u>0.020 AC</u> | <u>X</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3378</u> Lot No. <u>16060</u> Rep AC <u>0.300 AC</u> | <u>X</u> |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN <u>DR3803</u> Lot No. <u>1CS4</u> Solution <u>0.10 AC + 0.05%</u>
Acetone | <u>X</u> |
| H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) | <u>X</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) <u>0.080</u> AC | |

Lot No. <u>19615080AS</u>	Cyl No. <u>34</u>	Exp. Date <u>5 Sep 17</u>
Test 1 <u>0.081</u>	Test 4 <u>0.082</u>	Test 7 <u>0.081</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.082</u>
Average <u>0.081</u>		

J. Tank pressure, Level 3 Func D Sub G

P: 614 psi Reg 600 psi

K. Remarks/Maintenance record: Annual inspection

15 May 17
Date

15 May 17
Date

Roberta Guegan-Nimmis
Field Inspector's Signature

Dub Shanover
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

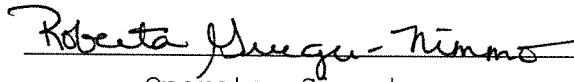
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006499
Location = JAME 8164.14.00 09/16
05/15/2017 12:04

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:04
02 Std. Sol.	0.020	12:05
03 Room Air	0.000	12:06
04 Std. Sol.	0.019	12:06
05 Room Air	0.000	12:07
06 Std. Sol.	0.019	12:08
07 Room Air	0.000	12:08

08 Sim Temp = 34.0°C

Simul Ser No = DR5113
Std Sol No = 201507F
County = 47 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Low AC*
0.020 AC
Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

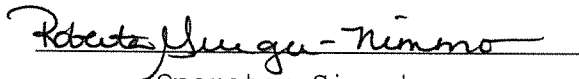
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006499
Location = JAME 8164.14.00 09/16
05/15/2017 12:10

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:11
02 Std. Sol.	0.303	12:11
03 Room Air	0.000	12:12
04 Std. Sol.	0.301	12:13
05 Room Air	0.000	12:13
06 Std. Sol.	0.302	12:14
07 Room Air	0.000	12:15

08 Sim Temp = 34.0°C

Simul Ser No = DR3378
Std Sol No = 16060
County = 47 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC
0.300 AC

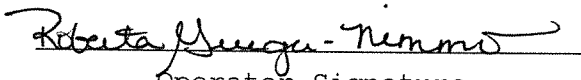
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006499
Location = JAME 8164.14.00 09/16
05/15/2017 12:17

Test	AC	Time
01 Room Air	0.000	12:18
02 *Subject Test	INT*	12:19
03 Room Air	0.000	12:19

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 34
Expiration Date = 09/05/2017
County = 47 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Interferent Check*
0.10 AC + 0.05% Acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

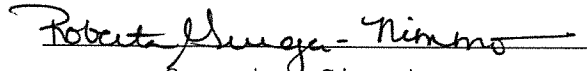
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006499
Location = JAME 8164.14.00 09/16
05/15/2017 12:20

Test	AC	Time
01 Diagnostic	OK	12:21
02 Room Air	0.000	12:21
03 *Subject Test	RFI*	12:21
04 Room Air	0.000	12:22

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 34
Expiration Date = 09/05/2017
County = 47 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check

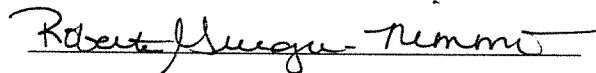
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006499
Location = JAME 8164.14.00 09/16
05/15/2017 12:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:26
02 Std. Gas	0.082	12:27
03 Room Air	0.000	12:27
04 Std. Gas	0.081	12:28
05 Room Air	0.000	12:28
06 Std. Gas	0.081	12:29
07 Room Air	0.000	12:29

Lot No = 19615080A5
Cyl No = 34
Exp Date = 09/05/2017
County = 47 Oper No. = 888888



Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006499
Location = JAME 8164.14.00 09/16
05/15/2017 12:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:30
02 Std. Gas	0.081	12:31
03 Room Air	0.000	12:31
04 Std. Gas	0.081	12:31
05 Room Air	0.000	12:32
06 Std. Gas	0.082	12:32
07 Room Air	0.000	12:33

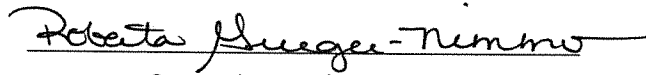
Lot No = 19615080A5

Cyl No = 34

Exp Date = 09/05/2017

County = 47

Oper No. = 888888



Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Form 106-I8000