

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006502

Location: STUT

- |  | Check When Done |
|--|-----------------|
| A. Is the breath tube heated?  | <u>✓</u>        |
| B. Is the diagnostic check complete?   | <u>✓</u>        |
| C. Is the time, date, and year correct (re-set if necessary)?  | <u>✓</u>        |
| D. Print test. (Attach test record.)   | <u>✓</u>        |
| E. Low AC. Use $\leq 0.03$ AC in ACA mode. (Attach test record.)<br>Sim SN <u>DR5113</u> Lot No. <u>201507 F</u> Rep AC <u>0.020</u>   | <u>✓</u>        |
| F. High AC. Use $\geq 0.25$ AC in ACA mode. (Attach test record.)<br>Sim SN <u>DR3378</u> Lot No. <u>16060</u> Rep AC <u>0.300</u>   | <u>✓</u>        |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.<br>(Attach test record.)<br>Sim SN <u>DR3803</u> Lot No. <u>1CS4</u> Solution <u>0.10Ac + 0.05% acetone</u> | <u>✓</u>        |
| H. RFI check. Run CMS mode. Key radio during CMS test.<br>(Attach test record.)  | <u>✓</u>        |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.<br>(Attach test record.) <u>0.080</u> AC  |                 |

Lot No. 19615080A5 Cyl No. 35 Exp. Date 09/05/2017

Test 1 0.081 Test 4 0.081 Test 7 0.081

Test 2 0.081 Test 5 0.081 Test 8 0.081

Test 3 0.081 Test 6 0.081 Test 9 0.081

Average 0.081

J. Tank pressure, Level 3 Func D Sub G P: 984 psi Reg 1000 psi

K. Remarks/Maintenance record: Annual inspection

15 May 17  
Date

Deb Shanover  
Field Inspector's Signature

15 May 17  
Date

Roberta Hunge-Thumm  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006502  
Location = STUT                8164.14.00 09/16  
05/15/2017                      11:16

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=-|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=-|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        No  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            19615080A5  
Standard Cyl #?            35  
Standard Expiration?       09/05/2017  
Oper No?                    777777

Flow Cal. Date:            08/19/2015  
Slope                        674  
Intercept                   -654491

IR Calibration Date:       08/19/2015  
                                    3um                      9um

0th Coef(*100):	-24446	-18103
1st Coef(*100):	267787	135417
2nd Coef(*100):	3387	1610
H2O adj(mg/l*10k):	740	513

\*\*\*\*\* Printer Test End \*\*\*\*\*

Deb Shanaver

Operator Signature  
DEB SHANAVER

Remarks:

Print test

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006502  
Location = STUT                    8164.14.00 09/16  
05/15/2017                            11:35

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:36
02 Std. Sol.	0.019	11:36
03 Room Air	0.000	11:37
04 Std. Sol.	0.019	11:38
05 Room Air	0.000	11:38
06 Std. Sol.	0.019	11:39
07 Room Air	0.000	11:40

08 Sim Temp = 34.0°C

Simul Ser No = DR5113  
Std Sol No = 201507F  
County = 47                            Oper No. = 77777

Deb Shanaver  
Operator Signature  
DEB SHANAVER

Remarks:

Low AC    0.020 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006502  
Location = STUT                    8164.14.00 09/16  
05/15/2017                            11:40

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:41
02 Std. Sol.	0.297	11:42
03 Room Air	0.000	11:42
04 Std. Sol.	0.298	11:43
05 Room Air	0.000	11:44
06 Std. Sol.	0.298	11:45
07 Room Air	0.000	11:45

08 Sim Temp = 34.0°C

Simul Ser No = DR3378

Std Sol No = 16060

County = 47

Oper No. = 77777

*Deb Shanaver*

Operator Signature

DEB SHANAVER

Remarks:

*JDS* High AC 0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006502  
Location = STUT                    8164.14.00 09/16  
05/15/2017                            11:47

Test	AC	Time
01 Room Air	0.000	11:47
02 *Subject Test	INT*	11:48
03 Room Air	0.000	11:49

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 19615080A5  
Cyl No = 35  
Expiration Date = 09/05/2017  
County = 47                            Oper No. = 777777

Deb Shanaver

Operator Signature  
DEB SHANAVER

Remarks:

*Interferent detected check 0.104C +  
6.05% acetone*

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-006502  
Location = STUT                              8164.14.00 09/16  
05/15/2017    11:20

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:21
02 Std. Gas	0.081	11:21
03 Room Air	0.000	11:22
04 Std. Gas	0.081	11:22
05 Room Air	0.000	11:23
06 Std. Gas	0.081	11:23
07 Room Air	0.000	11:24

Lot No = 19615080A5  
Cyl No = 35  
Exp Date = 09/05/2017  
County = 47                                      Oper No. = 777777

Deb Shanaver  
Operator Signature  
DEB SHANAVER

Remarks:

Calibration check 0.080AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006502  
Location = STUT      8164.14.00 09/16  
05/15/2017      11:24

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:25
02 Std. Gas	0.081	11:25
03 Room Air	0.000	11:26
04 Std. Gas	0.081	11:26
05 Room Air	0.000	11:27
06 Std. Gas	0.081	11:27
07 Room Air	0.000	11:28

Lot No = 19615080A5

Cyl No = 35

Exp Date = 09/05/2017

County = 47

Oper No. = 777777

*Deb Shanaver*

Operator Signature  
DEB SHANAVER

Remarks:

*Calibration check 0.080 AC*

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006502  
Location = STUT                    8164.14.00 09/16  
05/15/2017                            11:28

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:29
02 Std. Gas	0.081	11:29
03 Room Air	0.000	11:30
04 Std. Gas	0.081	11:30
05 Room Air	0.000	11:31
06 Std. Gas	0.081	11:31
07 Room Air	0.000	11:32

Lot No = 19615080A5

Cyl No = 35

Exp Date = 09/05/2017

County = 47

Oper No. = 77777

Deb Shanaver

Operator Signature  
DEB SHANAVER

Remarks:

Calibration check 0.080 AC

Form 106-I8000