

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005948

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR3846 Lot No. 16200 Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR7111 Lot No. 16060 Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. 1CS4 Solution 0.10AC + 0.05% acetone ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080AC

Lot No. 17316080A2 Cyl No. 27 Exp. Date 08/05/2018

Test 1 <u>0.081</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.079</u>	Test 5 <u>0.080</u>	Test 8 <u>0.081</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.079</u>

Average 0.080

J. Tank pressure, Level 3 Func D Sub G P: 269 psi Reg 275 psi

K. Remarks/Maintenance record: Annual inspection, cleaned intox, checked particulate screen

17 May 17
Date

Deb Shanaver
Field Inspector's Signature

17 May 17
Date

Roberto Yunque-Nimmo
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005948
Location = TOXL 8164.14.00 09/16
05/17/2017 09:27

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 27
Standard Expiration? 08/05/2018
Oper No? 777777

Flow Cal. Date: 06/20/2013
Slope 701
Intercept -716431

IR Calibration Date: 04/02/2015


	3um	9um
0th Coef(*100):	-25939	-20035
1st Coef(*100):	272687	134661
2nd Coef(*100):	3113	1576
H2O adj(mg/l*10k):	531	397

***** Printer Test End *****



Operator Signature
DEB SHANAVER

Remarks:

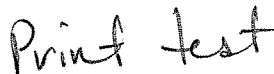

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005948
Location = TOXL 8164.14.00 09/16
05/17/2017 10:15

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:16
02 Std. Sol.	0.020	10:17
03 Room Air	0.000	10:17
04 Std. Sol.	0.020	10:18
05 Room Air	0.000	10:18
06 Std. Sol.	0.020	10:19
07 Room Air	0.000	10:20

08 Sim Temp = 34.0°C

Simul Ser No = DR3846
Std Sol No = 16200
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

Low AC 0.020AE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005948
Location = TOXL 8164.14.00 09/16
05/17/2017 10:27

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:27
02 Std. Sol.	0.302	10:28
03 Room Air	0.000	10:29
04 Std. Sol.	0.301	10:29
05 Room Air	0.000	10:30
06 Std. Sol.	0.301	10:31
07 Room Air	0.000	10:31

08 Sim Temp = 34.0°C

Simul Ser No = DR7111
Std Sol No = 16060
County = 08 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

High AC 0.300AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005948
Location = TOXL 8164.14.00 09/16
05/17/2017 10:39

Test	AC	Time
01 Room Air	0.000	10:40
02 *Subject Test	INT*	10:40
03 Room Air	0.000	10:41

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Interferent detected

0.10AC + 0.05% acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005948
Location = TOXL 8164.14.00 09/16
05/17/2017 10:41

Test	AC	Time
01 Diagnostic	OK	10:42
02 Room Air	RFI*	10:42
03 Room Air	0.000	10:42

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

RFI check


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005948
Location = TOXL 8164.14.00 09/16
05/17/2017 09:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:30
02 Std. Gas	0.081	09:30
03 Room Air	0.000	09:31
04 Std. Gas	0.079	09:31
05 Room Air	0.000	09:32
06 Std. Gas	0.080	09:32
07 Room Air	0.000	09:33

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005948
Location = TOXL 8164.14.00 09/16
05/17/2017 09:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:34
02 Std. Gas	0.080	09:34
03 Room Air	0.000	09:35
04 Std. Gas	0.080	09:35
05 Room Air	0.000	09:35
06 Std. Gas	0.080	09:36
07 Room Air	0.000	09:36

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005948
Location = TOXL 8164.14.00 09/16
05/17/2017 10:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:03
02 Std. Gas	0.080	10:04
03 Room Air	0.000	10:04
04 Std. Gas	0.081	10:05
05 Room Air	0.000	10:05
06 Std. Gas	0.079	10:06
07 Room Air	0.000	10:06

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC