

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-003072

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR3846 Lot No. 16200 Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR7111 Lot No. 16060 Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. 1CS4 Solution 0.10ACT + 0.05% acetone ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 17316080A2 Cyl No. 27 Exp. Date 08/05/2018

Test 1 <u>0.080</u>	Test 4 <u>0.081</u>	Test 7 <u>0.081</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.082</u>	Test 9 <u>0.081</u>

Average 0.081

J. Tank pressure, Level 3 Func D Sub G P: 292 psi Reg 300 psi

K. Remarks/Maintenance record: Annual inspection, cleaned intox,
replaced breath hose o-ring, checked particulate screen, CEE replaced
external battery.

17 May 17
Date

Deb Shanaver
Field Inspector's Signature

17 May 17
Date

Roberta Hueger-Himm
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
05/17/2017 14:00

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 27
Standard Expiration? 08/05/2018
Oper No? 777777

Flow Cal. Date: 03/09/2011
Slope 682
Intercept -620512

IR Calibration Date: 03/09/2011

	3um	9um
0th Coef(*100):	-24766	-17499
1st Coef(*100):	264559	138630
2nd Coef(*100):	1876	1114
H2O adj(mg/l*10k):	497	331

***** Printer Test End *****


Operator Signature
DEB SHANAVER

Remarks:

Print test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

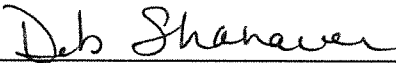
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = ~~GARR-TDXL~~ 8164.14.00 09/16
05/17/2017 DS 13:44

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:45
02 Std. Sol.	0.018	13:45
03 Room Air	0.000	13:46
04 Std. Sol.	0.018	13:47
05 Room Air	0.000	13:47
06 Std. Sol.	0.019	13:48
07 Room Air	0.000	13:48

08 Sim Temp = 34.0°C

Simul Ser No = DR3846
Std Sol No = 16200
County = 08 Oper No. = 777777


Operator Signature
DEB SHANAVER

Remarks:

Low AC 0.020AE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = ~~GARR~~ *TKL* 8164.14.00 09/16
05/17/2017 *DS* 13:49

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:49
02 Std. Sol.	0.295	13:50
03 Room Air	0.000	13:51
04 Std. Sol.	0.296	13:51
05 Room Air	0.000	13:52
06 Std. Sol.	0.296	13:53
07 Room Air	0.000	13:53

08 Sim Temp = 34.0°C

Simul Ser No = DR7111
Std Sol No = 16060
County = 08 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

High AC 0.300AE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
05/17/2017 13:58

Test	AC	Time
01 Room Air	0.000	13:59
02 *Subject Test	INT*	14:00
03 Room Air	0.000	14:00

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

*Interferent detected check 0.10 AC
+ 0.05% acetone*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
05/17/2017 14:01

Test	AC	Time
01 Diagnostic	OK	14:02
02 Room Air	RFI*	14:02
03 Room Air	0.000	14:02

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB SHANAVER

Remarks:

RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
05/17/2017 14:02

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:03
02 Std. Gas	0.080	14:03
03 Room Air	0.000	14:04
04 Std. Gas	0.081	14:04
05 Room Air	0.000	14:05
06 Std. Gas	0.081	14:05
07 Room Air	0.000	14:05

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
05/17/2017 14:06

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:07
02 Std. Gas	0.081	14:07
03 Room Air	0.000	14:08
04 Std. Gas	0.081	14:08
05 Room Air	0.000	14:08
06 Std. Gas	0.082	14:09
07 Room Air	0.000	14:09

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
05/17/2017 14:10

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:10
02 Std. Gas	0.081	14:10
03 Room Air	0.000	14:11
04 Std. Gas	0.081	14:11
05 Room Air	0.000	14:12
06 Std. Gas	0.081	14:12
07 Room Air	0.000	14:13

Lot No = 17316080A2

Cyl No = 27

Exp Date = 08/05/2018

County = 08

Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC

Form 106-I8000