

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004956

Location: TGXL

Check When Done

- A. Is the breath tube heated? ✓
- B. Is the diagnostic check complete? ✓
- C. Is the time, date, and year correct (re-set if necessary)? ✓
- D. Print test. (Attach test record.) ✓
- E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN MP3003 Lot No. 201610A Rep AC 0.020 ✓
- F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN MP3070 Lot No. 16060 Rep AC 0.300 ✓
- G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. ICS4 Solution 0.10AC + 0.05% ACETONE ✓
- H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
- I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 17316080A2 Cyl No. 28 Exp. Date 8/5/2018

Test 1 0.082 Test 4 0.081 Test 7 0.083

Test 2 0.081 Test 5 0.082 Test 8 0.082

Test 3 0.082 Test 6 0.082 Test 9 0.082

Average 0.082

J. Tank pressure, Level 3 Func D Sub G P: 511 psi Reg 500 psi

K. Remarks/Maintenance record: REPLACED EXTERNAL BATTERY, ANNUAL

INSPECTION

5/19/2017
Date

19 May 17
Date

Charles E. Ed
Field Inspector's Signature

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = TOXL 8164.14.00 09/16
05/19/2017 10:54

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 28
Standard Expiration? 08/05/2018
Oper No? 666666

Flow Cal. Date: 01/25/2010
Slope 672
Intercept -627011

IR Calibration Date: 04/09/2015
 3um 9um

0th Coef(*100): -23956 -28638
1st Coef(*100): 257675 133191
2nd Coef(*100): 2481 1339
H2O adj(mg/l*10k): 657 605

***** Printer Test End *****

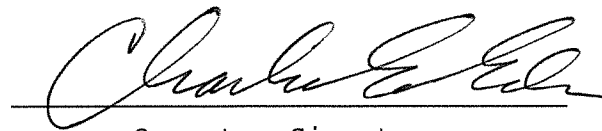

Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = TOXL 8164.14.00 09/16
05/19/2017 10:56

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:57
02 Std. Sol.	0.017	10:58
03 Room Air	0.000	10:58
04 Std. Sol.	0.017	10:59
05 Room Air	0.000	10:59
06 Std. Sol.	0.018	11:00
07 Room Air	0.000	11:01

08 Sim Temp = 34.0°C

Simul Ser No = MP3003
Std Sol No = 201610A
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

*Low AC
0.020 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = TOXL 8164.14.00 09/16
05/19/2017 11:07

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:08
02 Std. Sol.	0.293	11:09
03 Room Air	0.000	11:09
04 Std. Sol.	0.293	11:10
05 Room Air	0.000	11:10
06 Std. Sol.	0.294	11:11
07 Room Air	0.000	11:12

08 Sim Temp = 34.0°C

Simul Ser No = MP3070
Std Sol No = 16060
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = TOXL 8164.14.00 09/16
05/19/2017 11:31

Test	AC	Time
01 Room Air	0.000	11:32
02 *Subject Test	INT*	11:33
03 Room Air	0.000	11:33

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent
0.10AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = TOXL 8164.14.00 09/16
05/19/2017 11:34

Test	AC	Time
01 Diagnostic	OK	11:34
02 Room Air	0.000	11:35
03 *Subject Test	RFI*	11:35
04 Room Air	0.000	11:36

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = TOXL 8164.14.00 09/16
05/19/2017 11:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:19
02 Std. Gas	0.082	11:19
03 Room Air	0.000	11:20
04 Std. Gas	0.081	11:20
05 Room Air	0.000	11:21
06 Std. Gas	0.082	11:21
07 Room Air	0.000	11:22

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = TOXL 8164.14.00 09/16
05/19/2017 11:23

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:24
02 Std. Gas	0.081	11:24
03 Room Air	0.000	11:25
04 Std. Gas	0.082	11:25
05 Room Air	0.000	11:25
06 Std. Gas	0.082	11:26
07 Room Air	0.000	11:26

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 08 Oper No. = 666666

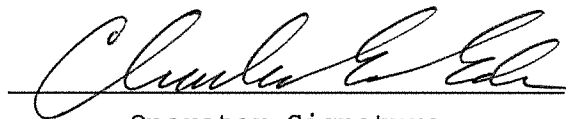

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = TOXL 8164.14.00 09/16
05/19/2017 11:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:36
02 Std. Gas	0.083	11:37
03 Room Air	0.000	11:37
04 Std. Gas	0.082	11:37
05 Room Air	0.000	11:38
06 Std. Gas	0.082	11:38
07 Room Air	0.000	11:39

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 08 Oper No. = 666666

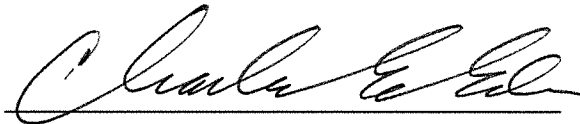

Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK

0.080 AC

Form 106-I8000