

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-003065

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
- B. Is the diagnostic check complete? ✓
- C. Is the time, date, and year correct (re-set if necessary)? ✓
- D. Print test. (Attach test record.) ✓
- E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR 3846 Lot No. 16200 Rep AC 0.020 ✓
- F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR 7111 Lot No. 16060 Rep AC 0.300 ✓
- G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.) ✓
Sim SN DR 5144 Lot No. 1CS4 Solution 0.10 AC + 0.05% acetone
- H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
- I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 17316080A2 Cyl No. 27 Exp. Date 08/05/2018

Test 1 0.080 Test 4 0.080 Test 7 0.079

Test 2 0.080 Test 5 0.080 Test 8 0.080

Test 3 0.080 Test 6 0.080 Test 9 0.079

Average 0.080

J. Tank pressure, Level 3 Func D Sub G P: 177 psi Reg ^{190 DS} 290 psi

K. Remarks/Maintenance record: Annual inspection, adjusted flow meter

19 May 17
Date

22 MAY 2017
Date

Deb Sharaver
Field Inspector's Signature

Charles E. Ed
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = TOXL 8164.14.00 09/16
05/19/2017 10:26

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=-|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=-|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer (Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 27
Standard Expiration? 08/05/2018
Oper No? 777777

Flow Cal. Date: 04/08/2008
Slope 694
Intercept -539075

IR Calibration Date: 08/23/2016
 3um 9um

0th Coef(*100): -14934 -12309
1st Coef(*100): 289519 141483
2nd Coef(*100): 1834 736
H2O adj(mg/l*10k): 268 136

***** Printer Test End *****



Operator Signature
DEB SHANAVER

Remarks:

Print test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = TOXL 8164.14.00 09/16
05/19/2017 10:18

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:19
02 Std. Sol.	0.021	10:20
03 Room Air	0.000	10:20
04 Std. Sol.	0.021	10:21
05 Room Air	0.000	10:21
06 Std. Sol.	0.021	10:22
07 Room Air	0.000	10:23

08 Sim Temp = 34.0°C

Simul Ser No = DR3846
Std Sol No = 16200
County = 08 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

Low AC 0.020AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = TOXL 8164.14.00 09/16
05/19/2017 09:30

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:31
02 Std. Sol.	0.295	09:31
03 Room Air	0.000	09:32
04 Std. Sol.	0.296	09:33
05 Room Air	0.000	09:33
06 Std. Sol.	0.297	09:34
07 Room Air	0.000	09:35

08 Sim Temp = 34.0°C

Simul Ser No = DR7111

Std Sol No = 16060

County = 08

Oper No. = 777777

Deb Shanaver

Operator Signature

DEB SHANAVER

Remarks:

High AC 0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = TOXL 8164.14.00 09/16
05/19/2017 10:37

Test	AC	Time
01 Room Air	0.000	10:38
02 *Subject Test	INT*	10:38
03 Room Air	0.000	10:39

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

*Interferent detected check 0.10AC +
0.05% acetone*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = TOXL 8164.14.00 09/16
05/19/2017 10:40

Test	AC	Time
01 Diagnostic	OK	10:41
02 Room Air	0.000	10:41
03 *Subject Test	RFI*	10:42
04 Room Air	0.000	10:42

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB SHANAVER

Remarks:



Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = TOXL 8164.14.00 09/16
05/19/2017 10:49

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:49
02 Std. Gas	0.080	10:49
03 Room Air	0.000	10:50
04 Std. Gas	0.080	10:50
05 Room Air	0.000	10:51
06 Std. Gas	0.080	10:51
07 Room Air	0.000	10:52

Lot No = 17316080A2

Cyl No = 27

Exp Date = 08/05/2018

County = 08

Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = TOXL 8164.14.00 09/16
05/19/2017 10:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:53
02 Std. Gas	0.080	10:53
03 Room Air	0.000	10:54
04 Std. Gas	0.080	10:54
05 Room Air	0.000	10:54
06 Std. Gas	0.080	10:55
07 Room Air	0.000	10:55

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003065
Location = TOXL 8164.14.00 09/16
05/19/2017 10:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:56
02 Std. Gas	0.079	10:57
03 Room Air	0.000	10:57
04 Std. Gas	0.080	10:57
05 Room Air	0.000	10:58
06 Std. Gas	0.079	10:58
07 Room Air	0.000	10:59

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC

Form 106-I8000