

INTOXILYZER® 8000 INSPECTION

(WILLIAMS County
CORR.
WILLIAMS
County L.E.C.)

Serial No.: 80-005946

Location: WILL

- | | Check When Done |
|--|-----------------|
| A. Is the breath tube heated? | <u>✓</u> |
| B. Is the diagnostic check complete? | <u>✓</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| D. Print test. (Attach test record.) | <u>✓</u> |
| E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5113</u> Lot No. <u>201507F</u> Rep AC <u>0.020</u> | <u>✓</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3378</u> Lot No. <u>16060</u> Rep AC <u>0.300</u> | <u>✓</u> |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN <u>DR3803</u> Lot No. <u>DCS4</u> Solution <u>0.10AC + 0.05% ACETONE</u> | <u>✓</u> |
| H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) | <u>✓</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) <u>0.080</u> AC | |

Lot No. <u>09216080AB</u>	Cyl No. <u>44</u>	Exp. Date <u>5/5/2018</u>
Test 1 <u>0.082</u>	Test 4 <u>0.081</u>	Test 7 <u>0.081</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.081</u>
Average <u>0.081</u>		

J. Tank pressure, Level 3 Func D Sub G P: 469 psi Reg 475 psi

K. Remarks/Maintenance record: REPLACED BREATH HOSE O-RING

ANNUAL INSPECTION

5/16/2017
Date

Charles E. Ed
Field Inspector's Signature

22 May 17
Date

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005946
Location = WILL 8164.14.00 09/16
05/16/2017 14:32

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 09216080A3
Standard Cyl #? 44
Standard Expiration? 05/05/2018
Oper No? 666666

Flow Cal. Date: 08/10/2015
Slope 673
Intercept -638348

IR Calibration Date: 08/10/2015

	3um	9um
0th Coef(*100):	-22507	-13665
1st Coef(*100):	273651	130615
2nd Coef(*100):	5249	2073
H2O adj(mg/l*10k):	617	504

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005946
Location = WILL 8164.14.00 09/16
05/16/2017 14:52

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:52
02 Std. Sol.	0.018	14:53
03 Room Air	0.000	14:54
04 Std. Sol.	0.019	14:54
05 Room Air	0.000	14:55
06 Std. Sol.	0.019	14:56
07 Room Air	0.000	14:56

08 Sim Temp = 34.0°C

Simul Ser No = DR5113
Std Sol No = 201507F
County = 53 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005946
Location = WILL 8164.14.00 09/16
05/16/2017 14:47

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:47
02 Std. Sol.	0.300	14:48
03 Room Air	0.000	14:49
04 Std. Sol.	0.300	14:49
05 Room Air	0.000	14:50
06 Std. Sol.	0.301	14:51
07 Room Air	0.000	14:51

08 Sim Temp = 34.0°C

Simul Ser No = DR3378
Std Sol No = 16060
County = 53 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *HIGH AC*
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005946
Location = WILL 8164.14.00 09/16
05/16/2017 14:39

Test	AC	Time
01 Room Air	0.000	14:40
02 *Subject Test	INT*	14:41
03 Room Air	0.000	14:42

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 44
Expiration Date = 05/05/2018
County = 53 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Interferent*
0.10 AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005946
Location = WILL 8164.14.00 09/16
05/16/2017 14:33

Test	AC	Time
01 Diagnostic	OK	14:34
02 Room Air	0.000	14:35
03 *Subject Test	RFI*	14:35
04 Room Air	0.000	14:35

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 44
Expiration Date = 05/05/2018
County = 53 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005946
Location = WILL 8164.14.00 09/16
05/16/2017 14:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:37
02 Std. Gas	0.082	14:37
03 Room Air	0.000	14:37
04 Std. Gas	0.080	14:38
05 Room Air	0.000	14:38
06 Std. Gas	0.081	14:39
07 Room Air	0.000	14:39

Lot No = 09216080A3
Cyl No = 44
Exp Date = 05/05/2018
County = 53 Oper No. = 666666

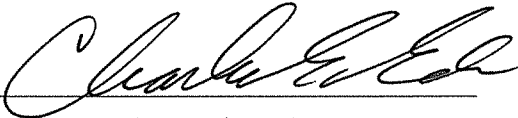
Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005946
Location = WILL 8164.14.00 09/16
05/16/2017 14:57

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:58
02 Std. Gas	0.081	14:58
03 Room Air	0.000	14:59
04 Std. Gas	0.080	14:59
05 Room Air	0.000	14:59
06 Std. Gas	0.081	15:00
07 Room Air	0.000	15:00

Lot No = 09216080A3
Cyl No = 44
Exp Date = 05/05/2018
County = 53 Oper No. = 666666

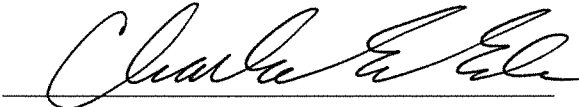

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005946
Location = WILL 8164.14.00 09/16
05/16/2017 15:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:01
02 Std. Gas	0.081	15:01
03 Room Air	0.000	15:02
04 Std. Gas	0.081	15:02
05 Room Air	0.000	15:03
06 Std. Gas	0.081	15:03
07 Room Air	0.000	15:04

Lot No = 09216080A3
Cyl No = 44
Exp Date = 05/05/2018
County = 53 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC