

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004947

Location: NAPO (LOGAN COUNTY SO.)

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5113 Lot No. 201507F Rep AC 0.026 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3378 Lot No. 16060 Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR3803 Lot No. ICS X5 Solution 0.10AC + 0.05% ✓
CEE ACETONE
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 09216080A3 Cyl No. 46 Exp. Date 5/5/2018

Test 1 <u>0.081</u>	Test 4 <u>0.082</u>	Test 7 <u>0.081</u>
Test 2 <u>0.082</u>	Test 5 <u>0.081</u>	Test 8 <u>0.080</u>
Test 3 <u>0.082</u>	Test 6 <u>0.082</u>	Test 9 <u>0.081</u>

Average 0.081

J. Tank pressure, Level 3 Func D Sub G

1005 CEE 1000 CEE
P. ~~1000~~ psi Reg ~~1005~~ psi

K. Remarks/Maintenance record: ANNUAL INSPECTION,

REPLACED INTERFERENT SOLUTION DURING INSPECTION

5/23/2017
Date

Charles E. Edr
Field Inspector's Signature

24 May 17
Date

Robata Hugu-Timmo
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004947
Location = NAPO 8164.14.00 09/16
05/23/2017 11:09

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 09216080A3
Standard Cyl #? 46
Standard Expiration? 05/05/2018
Oper No? 666666

Flow Cal. Date: 01/19/2010
Slope 679
Intercept -681178

IR Calibration Date: 05/12/2015
 3um 9um

0th Coef(*100): -14993 -20474
1st Coef(*100): 257346 132154
2nd Coef(*100): 2870 1330
H2O adj(mg/l*10k): 492 531

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004947
Location = NAPO 8164.14.00 09/16
05/23/2017 11:20

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:21
02 Std. Sol.	0.020	11:22
03 Room Air	0.000	11:22
04 Std. Sol.	0.020	11:23
05 Room Air	0.000	11:23
06 Std. Sol.	0.020	11:24
07 Room Air	0.000	11:25

08 Sim Temp = 34.0°C

Simul Ser No = DR5113

Std Sol No = 201507F

County = 24

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

Low AC

0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004947
Location = NAPO 8164.14.00 09/16
05/23/2017 11:26

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:27
02 Std. Sol.	0.307	11:28
03 Room Air	0.000	11:28
04 Std. Sol.	0.308	11:29
05 Room Air	0.000	11:30
06 Std. Sol.	0.308	11:30
07 Room Air	0.000	11:31


08 Sim Temp = 34.0°C

Simul Ser No = DR3378

Std Sol No = 16060

County = 24

Oper No. = 666666

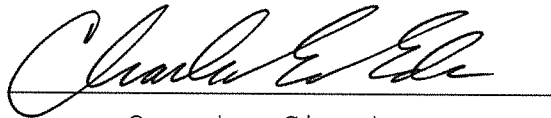

Operator Signature

CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004947
Location = NAPO 8164.14.00 09/16
05/23/2017 11:46

Test	AC	Time
01 Room Air	0.000	11:47
02 *Subject Test	INT*	11:48
03 Room Air	0.000	11:48

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 46
Expiration Date = 05/05/2018
County = 24 Oper No. = 666666

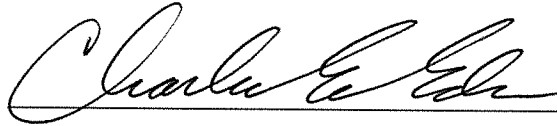

Operator Signature
CHARLES EDER

Remarks:

Interferent
0.10 AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004947
Location = NAPO 8164.14.00 09/16
05/23/2017 11:40

Test	AC	Time
01 Diagnostic	OK	11:41
02 Room Air	RFI*	11:41
03 Room Air	0.000	11:42

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 46
Expiration Date = 05/05/2018
County = 24 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004947
Location = NAPO 8164.14.00 09/16
05/23/2017 11:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:12
02 Std. Gas	0.081	11:13
03 Room Air	0.000	11:13
04 Std. Gas	0.082	11:13
05 Room Air	0.000	11:14
06 Std. Gas	0.082	11:14
07 Room Air	0.000	11:15

Lot No = 09216080A3

Cyl No = 46

Exp Date = 05/05/2018

County = 24

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004947
Location = NAPO 8164.14.00 09/16
05/23/2017 11:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:43
02 Std. Gas	0.082	11:44
03 Room Air	0.000	11:44
04 Std. Gas	0.081	11:45
05 Room Air	0.000	11:45
06 Std. Gas	0.082	11:45
07 Room Air	0.000	11:46

Lot No = 09216080A3

Cyl No = 46

Exp Date = 05/05/2018

County = 24

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004947
Location = NAPO 8164.14.00 09/16
05/23/2017 11:49

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:49
02 Std. Gas	0.081	11:50
03 Room Air	0.000	11:50
04 Std. Gas	0.080	11:50
05 Room Air	0.000	11:51
06 Std. Gas	0.081	11:51
07 Room Air	0.000	11:52

Lot No = 09216080A3
Cyl No = 46
Exp Date = 05/05/2018
County = 24 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK

0.080 AC

Form 106-I8000