

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004938

Location: Cavalier Co SD-LANG

- | | Check When Done |
|---|-----------------|
| A. Is the breath tube heated? | <u>✓</u> |
| B. Is the diagnostic check complete? | <u>✓</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| D. Print test. (Attach test record.) | <u>✓</u> |
| E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR 5113</u> Lot No. <u>201507F</u> Rep AC <u>0.020</u> | <u>✓</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR 3378</u> Lot No. <u>1160160</u> Rep AC <u>0.300</u> | <u>✓</u> |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN <u>DR 3803</u> Lot No. <u>ICS 5</u> Solution <u>0.10 AC + 0.05% acetone</u> | <u>✓</u> |
| H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) | <u>✓</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) <u>0.080</u> AC | |

Lot No. <u>19615080A5</u>	Cyl No. <u>15</u>	Exp. Date <u>09/05/2017</u>
Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.079</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080 AC</u>		

J. Tank pressure, Level 3 Func D Sub G P: 1006 psi Reg 1000 psi

K. Remarks/Maintenance record: Annual inspection, replaced breath hose & simulator return o-rings

25 May 17
Date

30 MAY 2017
Date

Deb Shaver
Field Inspector's Signature

[Signature]
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004938
Location = LANG 8164.14.00 09/16
05/25/2017 12:39

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:40
02 Std. Sol.	0.018	12:41
03 Room Air	0.000	12:41
04 Std. Sol.	0.018	12:42
05 Room Air	0.000	12:42
06 Std. Sol.	0.018	12:43
07 Room Air	0.000	12:44

08 Sim Temp = 34.0°C

Simul Ser No = DR5113

Std Sol No = 201507F

County = 10

Oper No. = 777777

Deb Shanaver

Operator Signature

DEB SHANAVER

Remarks:

low AC 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004938
Location = LANG 8164.14.00 09/16
05/25/2017 12:45

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:45
02 Std. Sol.	0.303	12:46
03 Room Air	0.000	12:47
04 Std. Sol.	0.306	12:47
05 Room Air	0.000	12:48
06 Std. Sol.	0.306	12:49
07 Room Air	0.000	12:49

08 Sim Temp = 34.0°C

Simul Ser No = DR3378

Std Sol No = 16060

County = 10

Oper No. = 777777

Deb Shanaver

Operator Signature

DEB SHANAVER

Remarks:

High AC 0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004938
Location = LANG 8164.14.00 09/16
05/25/2017 12:54

Test	AC	Time
01 Room Air	0.000	12:55
02 *Subject Test	INT*	12:56
03 Room Air	0.000	12:57

*Invalid Test
Interferent Detected

Sub Name = TEST, NA NA
Sub DOB = 09/23/1970
Sub Sex = Unknown Weight = NA
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 19615080A5
Cyl No = 15
Expiration Date = 09/05/2017
County = 10 Oper No. = 77777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Interferent detected check

Form 106-I8000

0.10AC + 0.05% acetone

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004938
Location = LANG 8164.14.00 09/16
05/25/2017 12:58

Test	AC	Time
01 Diagnostic	OK	12:59
02 Room Air	0.000	13:00
03 Reference	RFI*	13:00
04 Room Air	0.000	13:00

*Invalid Test
Inhibited - RFI

Sub Name = RFI, TEST NA
Sub DOB = 02/03/1994
Sub Sex = Unknown Weight = NA
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 19615080A5
Cyl No = 15
Expiration Date = 09/05/2017
County = 10 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004938
Location = LANG 8164.14.00 09/16
05/25/2017 12:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:29
02 Std. Gas	0.080	12:30
03 Room Air	0.000	12:30
04 Std. Gas	0.080	12:30
05 Room Air	0.000	12:31
06 Std. Gas	0.079	12:31
07 Room Air	0.000	12:32

Lot No = 19615080A5

Cyl No = 15

Exp Date = 09/05/2017

County = 10

Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004938
Location = LANG 8164.14.00 09/16
05/25/2017 12:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:33
02 Std. Gas	0.080	12:33
03 Room Air	0.000	12:34
04 Std. Gas	0.080	12:34
05 Room Air	0.000	12:35
06 Std. Gas	0.080	12:35
07 Room Air	0.000	12:35

Lot No = 19615080A5
Cyl No = 15
Exp Date = 09/05/2017
County = 10 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004938
Location = LANG 8164.14.00 09/16
05/25/2017 12:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:36
02 Std. Gas	0.080	12:36
03 Room Air	0.000	12:37
04 Std. Gas	0.080	12:37
05 Room Air	0.000	12:38
06 Std. Gas	0.080	12:38
07 Room Air	0.000	12:39

Lot No = 19615080A5
Cyl No = 15
Exp Date = 09/05/2017
County = 10 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC