

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006489

Location: GF PD

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5113 Lot No. 201507F Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3378 Lot No. 16060 Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR3803 Lot No. 1695 Solution 0.10AC + 0.05% acetone ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 01316080A2 Cyl No. 26 Exp. Date 02/05/2018

Test 1 <u>0.082</u>	Test 4 <u>0.082</u>	Test 7 <u>0.081</u>
Test 2 <u>0.080</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.079</u>	Test 9 <u>0.081</u>

Average 0.081

J. Tank pressure, Level 3 Func D Sub G P: 187 psi Reg 200 psi

K. Remarks/Maintenance record: Annual inspection, replaced breath
hose using

24 May 17
Date

Deb Shanaver
Field Inspector's Signature

30 MAY 2017
Date

Chuck E. Ed
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006489
Location = GFPD 8164.14.00 09/16
05/24/2017 15:38

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:38
02 Std. Sol.	0.018	15:39
03 Room Air	0.000	15:40
04 Std. Sol.	0.018	15:40
05 Room Air	0.000	15:41
06 Std. Sol.	0.018	15:42
07 Room Air	0.000	15:42

08 Sim Temp = 34.0°C

Simul Ser No = DR5113

Std Sol No = 201507F

County = 18

Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Low AC 0.020AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006489
Location = GFPD 8164.14.00 09/16
05/24/2017 15:43

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:44
02 Std. Sol.	0.295	15:44
03 Room Air	0.000	15:45
04 Std. Sol.	0.294	15:46
05 Room Air	0.000	15:47
06 Std. Sol.	0.294	15:47
07 Room Air	0.000	15:48

08 Sim Temp = 34.0°C

Simul Ser No = DR3378

Std Sol No = 16060

County = 18

Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

High AC ~~0.0~~^{DS} 0.300AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006489
Location = GFPD 8164.14.00 09/16
05/24/2017 15:49

Test	AC	Time
01 Room Air	0.000	15:50
02 *Subject Test	INT*	15:50
03 Room Air	0.000	15:51

*Invalid Test
Interferent Detected

Sub Name = TEST, NA NA
Sub DOB = 11/12/1995
Sub Sex = Unknown Weight = NA
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 01316080A2
Cyl No = 26
Expiration Date = 02/05/2018
County = 18 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Interferent Detected check

Form 106-I8000

0.10AC + 0.05% acetone

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006489
Location = GFPD 8164.14.00 09/16
05/24/2017 15:35

Test	AC	Time
01 Diagnostic	OK	15:36
02 Room Air	RFI*	15:36
03 Room Air	0.000	15:37

*Invalid Test
Inhibited - RFI

Sub Name = TEST, NA NA
Sub DOB = 10/10/1948
Sub Sex = Unknown Weight = NA
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 01316080A2
Cyl No = 26
Expiration Date = 02/05/2018
County = 18 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB SHANAVER

Remarks:

RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006489
Location = GFPD 8164.14.00 09/16
05/24/2017 15:24

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:24
02 Std. Gas	0.082	15:25
03 Room Air	0.000	15:25
04 Std. Gas	0.080	15:26
05 Room Air	0.000	15:26
06 Std. Gas	0.081	15:27
07 Room Air	0.000	15:27

Lot No = 01316080A2

Cyl No = 26

Exp Date = 02/05/2018

County = 18

Oper No. = 777777

Deb Shanaver

Operator Signature

DEB SHANAVER

Remarks:

Calibration check 0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006489
Location = GFPD 8164.14.00 09/16
05/24/2017 15:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:28
02 Std. Gas	0.082	15:28
03 Room Air	0.000	15:29
04 Std. Gas	0.081	15:29
05 Room Air	0.000	15:30
06 Std. Gas	0.079	15:30
07 Room Air	0.000	15:31

Lot No = 01316080A2

Cyl No = 26

Exp Date = 02/05/2018

County = 18

Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006489
Location = GFPD 8164.14.00 09/16
05/24/2017 15:31

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:32
02 Std. Gas	0.081	15:32
03 Room Air	0.000	15:33
04 Std. Gas	0.081	15:33
05 Room Air	0.000	15:34
06 Std. Gas	0.081	15:34
07 Room Air	0.000	15:35

Lot No = 01316080A2

Cyl No = 26

Exp Date = 02/05/2018

County = 18

Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC

Form 106-I8000