

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006493

Location: NCSO - hospital

- | | Check When Done |
|---|-----------------|
| A. Is the breath tube heated? | <u>✓</u> |
| B. Is the diagnostic check complete? | <u>✓</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| D. Print test. (Attach test record.) | <u>✓</u> |
| E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5113</u> Lot No. <u>201507F</u> Rep AC <u>0.020</u> | <u>✓</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3318</u> Lot No. <u>16060</u> Rep AC <u>0.300</u> | <u>✓</u> |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN <u>DR3803</u> Lot No. <u>1C55</u> Solution <u>0.10 AC + 0.05% acetone</u> | <u>✓</u> |
| H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) | <u>✓</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) <u>0.080</u> AC | |

Lot No. 09216080A3 Cyl No. 48 Exp. Date 05/05/2018

Test 1 0.081 Test 4 0.081 Test 7 0.081

Test 2 0.081 Test 5 0.081 Test 8 0.081

Test 3 0.081 Test 6 0.082 Test 9 0.082

Average 0.081

J. Tank pressure, Level 3 Func D Sub G P: 524 psi Reg 510 psi

K. Remarks/Maintenance record: Annual inspection

24 May 17
Date

30 MAY 2017
Date

Deb Shanaver
Field Inspector's Signature

Charles E. E.
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006493
Location = NCSO 8164.14.00 09/16
05/24/2017 11:08

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 09216080A3
Standard Cyl #? 48
Standard Expiration? 05/05/2018
Oper No? 777777

Flow Cal. Date: 02/12/2016
Slope 691
Intercept -663384

IR Calibration Date: 08/22/2016
 3um 9um

0th Coef(*100): -23064 -17586
1st Coef(*100): 268483 137311
2nd Coef(*100): 2939 1348
H2O adj(mg/l*10k): 636 433

***** Printer Test End *****

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Print test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006493
Location = NCSO 8164.14.00 09/16
05/24/2017 11:23

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:24
02 Std. Sol.	0.019	11:25
03 Room Air	0.000	11:25
04 Std. Sol.	0.020	11:26
05 Room Air	0.000	11:26
06 Std. Sol.	0.020	11:27
07 Room Air	0.000	11:28

08 Sim Temp = 34.0°C

Simul Ser No = DR5113

Std Sol No = 201507F

County = 32

Oper No. = 77777

Deb Shanaver

Operator Signature

DEB SHANAVER

Remarks:

Low AC 0.020AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006493
Location = NCSO 8164.14.00 09/16
05/24/2017 11:29

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:29
02 Std. Sol.	0.310	11:30
03 Room Air	0.000	11:31
04 Std. Sol.	0.310	11:32
05 Room Air	0.000	11:32
06 Std. Sol.	0.310	11:33
07 Room Air	0.000	11:33

08 Sim Temp = 34.0°C

Simul Ser No = DR3378

Std Sol No = 16060

County = 32

Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

High AC 0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006493
Location = NCSO 8164.14.00 09/16
05/24/2017 11:35

Test	AC	Time
01 Room Air	0.000	11:36
02 *Subject Test	INT*	11:37
03 Room Air	0.000	11:38

*Invalid Test
Interferent Detected

Sub Name = TEST, NA NA
Sub DOB = 08/23/1975
Sub Sex = Unknown Weight = NA
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 09216080A3
Cyl No = 48
Expiration Date = 05/05/2018
County = 32 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

*Interferent check 0.10AC +
0.05% acetone*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006493
Location = NCSO 8164.14.00 09/16
05/24/2017 11:20

Test	AC	Time
01 Diagnostic	OK	11:21
02 Room Air	RFI*	11:21
03 Room Air	0.000	11:22

*Invalid Test
Inhibited - RFI

Sub Name = TEST, NA NA
Sub DOB = 08/20/1960
Sub Sex = Unknown Weight = NA
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 09216080A3
Cyl No = 48
Expiration Date = 05/05/2018
County = 32 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006493
Location = NCSO 8164.14.00 09/16
05/24/2017 11:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:09
02 Std. Gas	0.081	11:10
03 Room Air	0.000	11:10
04 Std. Gas	0.081	11:10
05 Room Air	0.000	11:11
06 Std. Gas	0.081	11:11
07 Room Air	0.000	11:12

Lot No = 09216080A3
Cyl No = 48
Exp Date = 05/05/2018
County = 32 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006493
Location = NCSO 8164.14.00 09/16
05/24/2017 11:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:13
02 Std. Gas	0.081	11:13
03 Room Air	0.000	11:14
04 Std. Gas	0.081	11:14
05 Room Air	0.000	11:15
06 Std. Gas	0.082	11:15
07 Room Air	0.000	11:16

Lot No = 09216080A3
Cyl No = 48
Exp Date = 05/05/2018
County = 32 Oper No. = 77777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080Ae

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006493
Location = NCSO 8164.14.00 09/16
05/24/2017 11:16

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:17
02 Std. Gas	0.081	11:17
03 Room Air	0.000	11:18
04 Std. Gas	0.081	11:18
05 Room Air	0.000	11:18
06 Std. Gas	0.082	11:19
07 Room Air	0.000	11:19

Lot No = 09216080A3
Cyl No = 48
Exp Date = 05/05/2018
County = 32 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC