

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-060501 Location: GF jail

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5113 Lot No. 201507F Rep AC 0.020 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR 3318 Lot No. 16060 Rep AC 0.300 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR 3803 Lot No. 1CS5 Solution 0.1DAC + 0.05% acetone ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 09216080A3 Cyl No. 34 Exp. Date 05/05/2018

Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.081</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.082</u>	Test 9 <u>0.081</u>

Average 0.081

J. Tank pressure, Level 3 Func D Sub G P: 434 psi Reg 430 psi

K. Remarks/Maintenance record: Annual inspection, replaced breath hose o-ring

24 May 17
Date

30 MAY 2017
Date

Deb Shanaver
Field Inspector's Signature

[Signature]
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006501
Location = GFSO 8164.14.00 09/16
05/24/2017 14:26

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:27
02 Std. Sol.	0.019	14:27
03 Room Air	0.000	14:28
04 Std. Sol.	0.019	14:29
05 Room Air	0.000	14:29
06 Std. Sol.	0.018	14:30
07 Room Air	0.000	14:30

08 Sim Temp = 34.0°C

Simul Ser No = DR5113

Std Sol No = 201507F

County = 18

Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Low AC 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006501
Location = GFSO 8164.14.00 09/16
05/24/2017 14:31

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:32
02 Std. Sol.	0.298	14:33
03 Room Air	0.000	14:33
04 Std. Sol.	0.299	14:34
05 Room Air	0.000	14:35
06 Std. Sol.	0.298	14:35
07 Room Air	0.000	14:36

08 Sim Temp = 34.0°C

Simul Ser No = DR3378

Std Sol No = 16060

County = 18

Oper No. = 777777

Deb Shanaver

Operator Signature

DEB SHANAVER

Remarks:

High AC 0.300AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006501
Location = GFSO 8164.14.00 09/16
05/24/2017 14:37

Test	AC	Time
01 Room Air	0.000	14:38
02 *Subject Test	INT*	14:39
03 Room Air	0.000	14:39

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 34
Expiration Date = 05/05/2018
County = 18 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

Interferent detected check

Form 106-I8000

*0.10 AC
+ ^{DS} 0.05% acetone*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006501
Location = GFSO 8164.14.00 09/16
05/24/2017 14:46

Test	AC	Time
01 Diagnostic	OK	14:47
02 Room Air	RFI*	14:47
03 Room Air	0.000	14:48

*Invalid Test
Inhibited - RFI

Sub Name = RFI, TEST NA
Sub DOB = 09/26/1974
Sub Sex = Unknown Weight = NA
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 09216080A3
Cyl No = 34
Expiration Date = 05/05/2018
County = 18 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

RFI test

Form 106-I8000

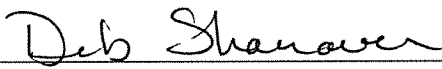
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006501
Location = GFSO 8164.14.00 09/16
05/24/2017 14:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:43
02 Std. Gas	0.081	14:44
03 Room Air	0.000	14:44
04 Std. Gas	0.081	14:45
05 Room Air	0.000	14:45
06 Std. Gas	0.081	14:46
07 Room Air	0.000	14:46

Lot No = 09216080A3
Cyl No = 34
Exp Date = 05/05/2018
County = 18 Oper No. = 777777


Operator Signature
DEB SHANAVER

Remarks:
Calibration check 0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006501
Location = GFSO 8164.14.00 09/16
05/24/2017 14:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:49
02 Std. Gas	0.081	14:49
03 Room Air	0.000	14:50
04 Std. Gas	0.081	14:50
05 Room Air	0.000	14:51
06 Std. Gas	0.082	14:51
07 Room Air	0.000	14:51

Lot No = 09216080A3

Cyl No = 34

Exp Date = 05/05/2018

County = 18

Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006501
Location = GFSO 8164.14.00 09/16
05/24/2017 14:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:52
02 Std. Gas	0.081	14:53
03 Room Air	0.000	14:53
04 Std. Gas	0.081	14:54
05 Room Air	0.000	14:54
06 Std. Gas	0.081	14:54
07 Room Air	0.000	14:55

Lot No = 09216080A3
Cyl No = 34
Exp Date = 05/05/2018
County = 18 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC