

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-005944

Location: TOXL

- | | Check When Done |
|--|-----------------|
| A. Is the breath tube heated? | <u>✓</u> |
| B. Is the diagnostic check complete? | <u>✓</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| D. Print test. (Attach test record.) | <u>✓</u> |
| E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>MP3003</u> Lot No. <u>16200</u> Rep AC <u>0.020</u> | <u>✓</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>MP3070</u> Lot No. <u>16060</u> Rep AC <u>0.300</u> | <u>✓</u> |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN <u>DRS135</u> Lot No. <u>ICS4</u> Solution <u>0.10AC + 0.05%
ACETONE</u> | <u>✓</u> |
| H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) | <u>✓</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) <u>0.080</u> AC | |

Lot No. <u>17316080A2</u>	Cyl No. <u>28</u>	Exp. Date <u>8-5-2018</u>
Test 1 <u>0.082</u>	Test 4 <u>0.082</u>	Test 7 <u>0.082</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.082</u>	Test 9 <u>0.081</u>
Average <u>0.081</u>		

J. Tank pressure, Level 3 Func D Sub G P: 218 psi Reg 225 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION, CALIBRATED

TANK SENSOR
12 JUNE 2017
Date
12 June 17
Date

Charles E. El...
Field Inspector's Signature
Deb Sheaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = TOXL 8164.14.00 09/16
06/12/2017 09:46

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPNOPQRSTUVWXYZ!@#%&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPNOPQRSTUVWXYZ!@#%&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 28
Standard Expiration? 08/05/2018
Oper No? 666666

Flow Cal. Date: 06/20/2013
Slope 677
Intercept -702620

IR Calibration Date: 06/09/2017

	3um	9um
0th Coef(*100):	-16158	-17815
1st Coef(*100):	266780	136962
2nd Coef(*100):	1117	1089
H2O adj(mg/l*10k):	317	364

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = TOXL 8164.14.00 09/16
06/12/2017 09:53

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:53
02 Std. Sol.	0.019	09:54
03 Room Air	0.000	09:55
04 Std. Sol.	0.020	09:55
05 Room Air	0.000	09:56
06 Std. Sol.	0.020	09:57
07 Room Air	0.000	09:57

08 Sim Temp = 34.0°C

Simul Ser No = MP3003
Std Sol No = 16200
County = 08 Oper No. = 666666

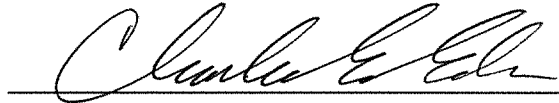

Operator Signature
CHARLES EDER

Remarks: *Low AC*
 0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = TOXL 8164.14.00 09/16
06/12/2017 10:04

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:05
02 Std. Sol.	0.306	10:05
03 Room Air	0.000	10:06
04 Std. Sol.	0.307	10:07
05 Room Air	0.000	10:07
06 Std. Sol.	0.307	10:08
07 Room Air	0.000	10:08

08 Sim Temp = 34.0°C

Simul Ser No = MP3070
Std Sol No = 16060
County = 08 Oper No. = 666666

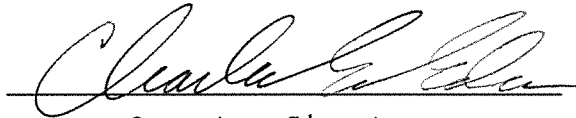

Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = TOXL 8164.14.00 09/16
06/12/2017 10:19

Test	AC	Time
01 Room Air	0.000	10:20
02 *Subject Test	INT*	10:20
03 Room Air	0.000	10:21

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Interferant Check*
0.10 AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = TOXL 8164.14.00 09/16
06/12/2017 10:21

Test	AC	Time
01 Diagnostic	OK	10:22
02 Room Air	0.000	10:22
03 *Subject Test	RFI*	10:23
04 Room Air	0.000	10:23

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = TOXL 8164.14.00 09/16
06/12/2017 10:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:27
02 Std. Gas	0.082	10:28
03 Room Air	0.000	10:28
04 Std. Gas	0.081	10:29
05 Room Air	0.000	10:29
06 Std. Gas	0.081	10:29
07 Room Air	0.000	10:30

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK

0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = TOXL 8164.14.00 09/16
06/12/2017 10:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:31
02 Std. Gas	0.082	10:31
03 Room Air	0.000	10:32
04 Std. Gas	0.081	10:32
05 Room Air	0.000	10:33
06 Std. Gas	0.082	10:33
07 Room Air	0.000	10:34

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 08 Oper No. = 666666


Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080AC
Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = TOXL 8164.14.00 09/16
06/12/2017 10:34

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:34
02 Std. Gas	0.082	10:35
03 Room Air	0.000	10:35
04 Std. Gas	0.081	10:36
05 Room Air	0.000	10:36
06 Std. Gas	0.081	10:37
07 Room Air	0.000	10:37

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000