



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (10-2016)

Serial Number 80-005952	Instrument Location Burleigh Morton Detention Center
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Rebecca Hopp</i>	Date 6/12/2017
--	-------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by <i>Deb Shanauer</i>	Certified Date 16 June 17
--	------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.14.00 09/16
06/12/2017 10:40

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 09216080A3
Standard Cyl #? 31
Standard Expiration? 05/05/2018
Oper No? 041005

Flow Cal. Date: 06/21/2013
Slope 734
Intercept -865305

IR Calibration Date: 06/21/2013
 3um 9um

0th Coef(*100): -21281 -23057
1st Coef(*100): 292132 137092
2nd Coef(*100): 2334 1231
H2O adj(mg/l*10k): 497 482

***** Printer Test End *****

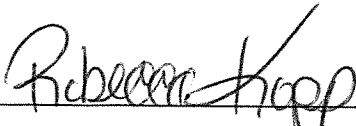

Operator Signature
REBECCA KOPP

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.14.00 09/16
06/12/2017 10:40

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:41
02 Std. Gas	0.082	10:41
03 Room Air	0.000	10:42
04 Std. Gas	0.082	10:42
05 Room Air	0.000	10:43
06 Std. Gas	0.082	10:43
07 Room Air	0.000	10:44


Lot No = 09216080A3

Cyl No = 31

Exp Date = 05/05/2018

County = 08

Oper No. = 041005

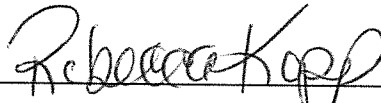

Operator Signature
REBECCA KOPP

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.14.00 09/16
06/12/2017 10:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:48
02 Std. Gas	0.082	10:48
03 Room Air	RFI*	10:48
04 Room Air	0.000	10:49

*Invalid Test
Inhibited - RFI

Lot No = 09216080A3
Cyl No = 31
Exp Date = 05/05/2018
County = 08 Oper No. = 041005

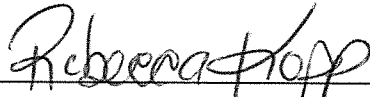

Operator Signature
REBECCA KOPP

Remarks:

Form 106-I8000