



ETHANOL GAS STANDARD CYLINDER REPORT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>Chad Thompson</i>		
Location <i>Cass Co. Jail</i>		Intoxilyzer® Serial Number <i>80-006506</i>
Gas Lot Number <i>20016080A2</i>	Gas Cylinder Number <i>37</i>	Gas Expiration Date <i>9-5-18</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. *0.079* AC
 B. *0.079* AC
 C. *0.079* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>7-18-17</i>
Reviewed By (Crime Laboratory Use Only) <i>Roberta Huggins-Nimms</i>	Date <i>19 July 17</i>


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006506
Location = CASS 8164.14.00 09/16
07/18/2017 21:37

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	21:38
02 Std. Gas	0.079	21:38
03 Room Air	0.000	21:39
04 Std. Gas	0.079	21:39
05 Room Air	0.000	21:40
06 Std. Gas	0.079	21:40
07 Room Air	0.000	21:41

Lot No = 20016080A2
Cyl No = 37
Exp Date = 09/05/2018
County = 09 Oper No. = 050910

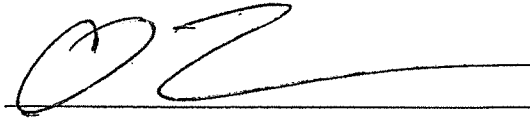

Operator Signature
CHAD THOMPSON

3561

Remarks: New Cylinder Install