



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (10-2016)

Serial Number <i>80-005-945</i>	Instrument Location <i>Room 1214 Grand TExPOX Justice Center New Town ND</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u><i>was asked to try State crime Lab</i></u>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000. *No*
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:  
*Box checked by RSN 23 Aug 17*  
 A. Print Test (Level 1, Function P).  
 B. ACA Test (Level 1, Function C).  
 C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *Plugged off*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Dustin Krueger</i> <i>5668 130472</i>	Date <i>8-13-15</i>
---	------------------------

Crime Laboratory Use Only

*test records were ran  
8-13-17 RSN*

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by <i>Roberta Senger-Nimmo</i>	Certified Date <i>23 Aug 17</i>
--	------------------------------------

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005945  
Location = NEWT      8164.14.00 09/16  
08/13/2017      11:59

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&\*()\_+?


Current Instrument Setup

Data Entry Mode:      Enabled  
Start Test Sequence:      DABACABA  
Display Prelim Rslt?      Yes  
Display Third Digit?      Yes  
Inhib Printer (Y/N)?      No  
Display Volume?      No  
Disable On Memfull?      Yes  
# of Print Copies?      1  
Select Std (D/W/I)?      Dry  
Standard Value?      0.080  
Standard Lot #?      17316080A2  
Standard Cyl #?      35  
Standard Expiration?      08/05/2018  
Oper No?      130472  
  
Flow Cal. Date:      06/20/2013  
Slope      637  
Intercept      -566407

IR Calibration Date:      05/22/2015  
    3um    9um

0th Coef(*100):	-21213	-23093
1st Coef(*100):	279171	139292
2nd Coef(*100):	2898	1291
H2O adj(mg/l*10k):	332	319

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
Dustin Krueger 5666

Operator Signature  
DUSTIN KRUEGER

Remarks: *Print Test*

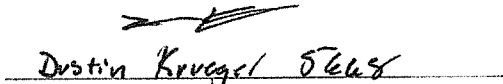
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005945  
Location = NEWT      8164.14.00 09/16  
08/13/2017      11:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:55
02 Std. Gas	0.078	11:56
03 Room Air	0.000	11:56
04 Std. Gas	0.079	11:57
05 Room Air	0.000	11:57
06 Std. Gas	0.079	11:57
07 Room Air	0.000	11:58

Lot No = 17316080A2  
Cyl No = 35  
Exp Date = 08/05/2018  
County = 60      Oper No. = 130472

  
Operator Signature  
DUSTIN KRUEGER

Remarks: *ACA Test*

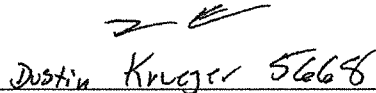
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005945  
Location = NEWT      8164.14.00 09/16  
08/13/2017      11:59

Test	AC	Time
01 Room Air	RFI*	12:00
02 Room Air	0.000	12:01

\*Invalid Test  
Inhibited - RFI

Sub Name = KRUEGER, DUSTIN CAL  
Sub DOB = 08/08/1979  
Sub Sex = Male      Weight = 200  
Test = DUI      Cit = 000  
Dr. Lic. = ND/KRU796350  
Lot No = 17316080A2  
Cyl No = 35  
Expiration Date = 08/05/2018  
County = 60      Oper No. = 130472

  
\_\_\_\_\_  
Operator Signature  
DUSTIN KRUEGER

Remarks: RFI Test