



ETHANOL GAS STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>Jonathan Lampi</i>		
Location <i>GFPD</i>	Intoxilyzer® Serial Number <i>80-006489</i>	
Gas Lot Number <i>20016080A2</i>	Gas Cylinder Number <i>051</i>	Gas Expiration Date <i>09-05-2018</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
A. *0.080* AC
B. *0.080* AC
C. *0.080* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Jon Lampi</i>	Date <i>08-17-17</i>
Reviewed By (Crime Laboratory Use Only) <i>Roberta Nueguero-Nunno</i>	Date <i>08-21-17</i>

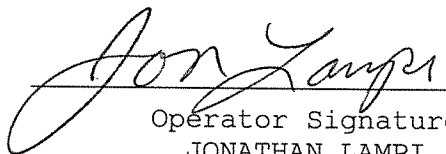
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006489
Location = GFPD 8164.14.00 09/16
08/17/2017 09:09

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:10
02 Std. Gas	0.080	09:10
03 Room Air	0.000	09:11
04 Std. Gas	0.080	09:11
05 Room Air	0.000	09:12
06 Std. Gas	0.080	09:12
07 Room Air	0.000	09:13

Lot No = 20016080A2
Cyl No = 051
Exp Date = 09/05/2018
County = 18 Oper No. = 039947


Operator Signature
JONATHAN LAMPI

Remarks:

Form 106-I8000

Gas cylinder change