



**ETHANOL GAS STANDARD CYLINDER REPORT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>BENNETT RITZ</i>		
Location <i>Stutsman LEC 1<sup>st</sup> floor</i>	Intoxilyzer® Serial Number <i>80-006502</i>	
Gas Lot Number <i>03417080A3</i>	Gas Cylinder Number <i>3</i>	Gas Expiration Date <i>3/05/2019</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).  
 A. 0. 081 AC  
 B. 0. 081 AC  
 C. 0. 081 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>9/21/17</i>
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i>	Date <i>9/21/17</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006502  
Location = STUT      8164.14.00 09/16  
09/21/2017      08:28

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:29
02 Std. Gas	0.081	08:29
03 Room Air	0.000	08:30
04 Std. Gas	0.081	08:30
05 Room Air	0.000	08:31
06 Std. Gas	0.081	08:31
07 Room Air	0.000	08:32

Lot No = 03417080A3  
Cyl No = 3  
Exp Date = 03/05/2019  
County = 47      Oper No. = 130997



Operator Signature  
BENNETT BITZ

Remarks:

Form 106-I8000