



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (10-2016)

Serial Number 80-005957	Instrument Location "CROS" Divide County Sheriff's Office
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): None needed
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Robert M. Melby</i> Robert M. Melby	Date 1/30/2018
---------------------------------------------------------------------	-------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by <i>Roberta Grieger-Nimmo</i> Roberta Grieger-Nimmo	Certified Date 1/31/2018
-----------------------------------------------------------------------------	-----------------------------


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005957
Location = CROS 8164.14.00 09/16
01/30/2018 16:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:15
02 Std. Gas	0.080	16:15
03 Room Air	0.000	16:15
04 Std. Gas	0.081	16:16
05 Room Air	0.000	16:16
06 Std. Gas	0.081	16:17
07 Room Air	0.000	16:17

Lot No = 17316080A2
Cyl No = 17
Exp Date = 08/05/2018
County = 12 Oper No. = 100011

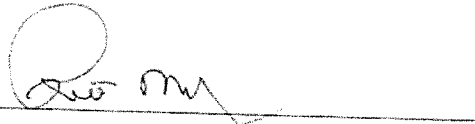


Operator Signature
ROBERT M. MELBY

Remarks:

Test Good

Form 106-I8000

The undersigned, having custody of the original record, certifies that the information hereon is a true and correct copy of the original document maintained as part of the activities of this office.

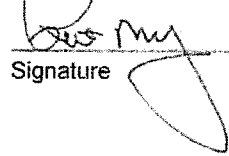

Signature

01/30/2018
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005957
Location = CROS 8164.14.00 09/16
01/30/2018 16:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:20
02 Std. Gas	RFI*	16:20
03 Room Air	0.000	16:21

*Invalid Test
Inhibited - RFI

Lot No = 17316080A2
Cyl No = 17
Exp Date = 08/05/2018
County = 12 Oper No. = 100011

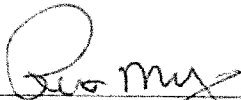

Operator Signature
ROBERT M. MELBY

Remarks: *Test Good*

Form 106-I8000

The undersigned, having custody of the original record, certifies that the information hereon is a true and correct copy of the original document maintained as part of the activities of this office.



Signature 01/30/20
Date