



ETHANOL GAS STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59282 (10-2013)

Chemical Test Operator Name (Print) Chad Thompson		
Location Cass Jail		Intoxilyzer® Serial Number 80-006506
Gas Lot Number 19817080A4	Gas Cylinder Number 16	Gas Expiration Date 09-05-2019

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0.079 AC
 - B. 0.080 AC
 - C. 0.080 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature 	Date 3/23/2018
Reviewed By (Crime Laboratory Use Only) Roberta Huggins-Nunn	Date 3/26/2018


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006506
Location = CASS 8164.14.00 09/16
03/23/2018 20:38

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	20:39
02 Std. Gas	0.079	20:39
03 Room Air	0.000	20:40
04 Std. Gas	0.080	20:40
05 Room Air	0.000	20:41
06 Std. Gas	0.080	20:41
07 Room Air	0.000	20:42

Lot No = 19817080A4
Cyl No = 16
Exp Date = 09/05/2019
County = 09 Oper No. = 050910



Operator Signature
CHAD THOMPSON

Remarks: *New Cylinder Installed*
3/23/18

Form 106-I8000