

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-002666

Location: TOXL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 321 psi Regulator: 325 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC  
Sim. Ser #: MP3002 Lot #: 17160 Exp. Date: 6-8-19
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.400 AC  
Sim. Ser #: MP3061 Lot #: 201605F Exp. Date: 5-19-18

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR7089 Lot #: ICS 5 Exp. Date: 5/15/19  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 03417080A3 Cyl #: 27 Exp. Date: 3/5/19

Test 1	<u>0.082</u>	Test 4	<u>0.083</u>	Test 7	<u>0.083</u>
Test 2	<u>0.083</u>	Test 5	<u>0.083</u>	Test 8	<u>0.083</u>
Test 3	<u>0.083</u>	Test 6	<u>0.082</u>	Test 9	<u>0.082</u>
Average	<u>0.083</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Edr  
Breath Analyst Signature

4.3.2018  
Date

Roberta Huggins-Timmer  
Reviewed by

3 Apr 18  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-002666  
Location = TOXL                    8164.14.00 09/16  
04/03/2018                            10:12

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            03417080A3  
Standard Cyl #?            27  
Standard Expiration?       03/05/2019  
Oper No?                    666666

Flow Cal. Date:            12/31/2012  
Slope                      657  
Intercept                  -418473

IR Calibration Date:       05/27/2016  
                                    3um                      9um

-----  
0th Coef(\*100):        -18254                    -11303  
1st Coef(\*100):        275128                    137601  
2nd Coef(\*100):        2576                      1178  
H2O adj(mg/l\*10k):    442                        273

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-002666  
Location = TOXL      8164.14.00 09/16  
04/03/2018      10:19

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:19
02 Std. Sol.	0.019	10:20
03 Room Air	0.000	10:21
04 Std. Sol.	0.019	10:21
05 Room Air	0.000	10:22
06 Std. Sol.	0.020	10:23
07 Room Air	0.000	10:23

08 Sim Temp = 34.0°C

Simul Ser No = MP3002

Std Sol No = 17160

County = 08

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Low AC*  
*0.020 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-002666  
Location = TOXL      8164.14.00 09/16  
04/03/2018      10:24

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:24
02 Std. Sol.	0.397	10:25
03 Room Air	0.000	10:26
04 Std. Sol.	0.399	10:26
05 Room Air	0.000	10:27
06 Std. Sol.	0.400	10:28
07 Room Air	0.000	10:28

08 Sim Temp = 34.0°C

Simul Ser No = MP3061  
Std Sol No = 201605F  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

HIGH AC  
0.400AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-002666  
Location = TOXL                8164.14.00 09/16  
04/03/2018                        10:30

Test	AC	Time
01 Room Air	0.000	10:31
02 *Subject Test	INT*	10:31
03 Room Air	0.000	10:32

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 03417080A3  
Cyl No = 27  
Expiration Date = 03/05/2019  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent Check*  
*0.10 AC + 0.05% ACETONE*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-002666  
Location = TOXL                8164.14.00 09/16  
04/03/2018                      10:33

Test	AC	Time
01 Diagnostic	OK	10:34
02 Room Air	0.000	10:34
03 *Subject Test	RFI*	10:34
04 Room Air	0.000	10:35

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                      Weight = 150  
Test = DUI                              Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 03417080A3  
Cyl No = 27  
Expiration Date = 03/05/2019  
County = 08                              Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

*RFI CHECK*


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-002666  
Location = TOXL      8164.14.00 09/16  
04/03/2018      10:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:36
02 Std. Gas	0.082	10:36
03 Room Air	0.000	10:37
04 Std. Gas	0.083	10:37
05 Room Air	0.000	10:38
06 Std. Gas	0.083	10:38
07 Room Air	0.000	10:39

Lot No = 03417080A3  
Cyl No = 27  
Exp Date = 03/05/2019  
County = 08      Oper No. = 666666

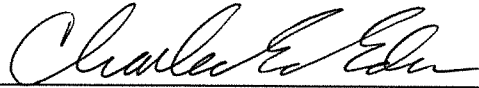

Operator Signature  
CHARLES EDER

Remarks:

*CALIBRATION CHECK*  
*0.080 AC*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-002666  
Location = TOXL      8164.14.00 09/16  
04/03/2018      10:39

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:39
02 Std. Gas	0.083	10:40
03 Room Air	0.000	10:40
04 Std. Gas	0.083	10:41
05 Room Air	0.000	10:41
06 Std. Gas	0.082	10:41
07 Room Air	0.000	10:42

Lot No = 03417080A3  
Cyl No = 27  
Exp Date = 03/05/2019  
County = 08      Oper No. = 666666

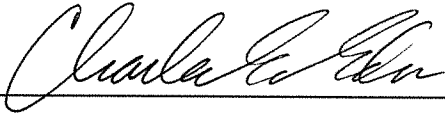

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080AC

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-002666  
Location = TOXL      8164.14.00 09/16  
04/03/2018      10:42

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:43
02 Std. Gas	0.083	10:43
03 Room Air	0.000	10:44
04 Std. Gas	0.083	10:44
05 Room Air	0.000	10:44
06 Std. Gas	0.082	10:45
07 Room Air	0.000	10:45

Lot No = 03417080A3  
Cyl No = 27  
Exp Date = 03/05/2019  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC