

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005958

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 745 psi Regulator: 750 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: 4P3007 Lot #: 201712B Exp. Date: 12 Dec 19
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: 4P3002 Lot #: 17350 Exp. Date: 11 Oct 19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.12 AC + 0.05% Acetone
Sim. Ser #: DR5144 Lot #: 1CSS Exp. Date: 15 May 19
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 20016080A2 Cyl #: 48 Exp. Date: 5 Sep 18

Test 1	<u>0.080</u>	Test 4	<u>0.081</u>	Test 7	<u>0.081</u>
Test 2	<u>0.081</u>	Test 5	<u>0.082</u>	Test 8	<u>0.080</u>
Test 3	<u>0.080</u>	Test 6	<u>0.082</u>	Test 9	<u>0.081</u>
Average	<u>0.081</u>				

C. Remarks/Maintenance: Annual inspection.

Instrument is acceptable to be used in the field.

Roberta Muga-Nimmo
Breath Analyst Signature

3 Apr 18
Date

Charles E. Eden
Reviewed by

4.3.2018
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005958
Location = TOXL 8164.13.00 06/09
04/03/2018 11:32

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 20016080A2
Standard Cyl #? 48
Standard Expiration? 09/05/2018
Oper No? 888888

Flow Cal. Date: 06/21/2013
Slope 716
Intercept -760436

IR Calibration Date: 04/07/2015
 3um 9um

0th Coef(*100): -17179 -17694
1st Coef(*100): 259328 130890
2nd Coef(*100): 3473 1654
H2O adj(mg/l*10k): 425 422

***** Printer Test End *****

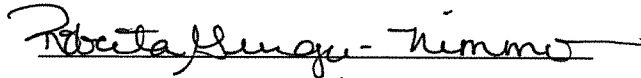

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Print Test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005958
Location = TOXL 8164.13.00 06/09
04/03/2018 11:37

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:38
02 Std. Sol.	0.018	11:39
03 Room Air	0.000	11:39
04 Std. Sol.	0.019	11:40
05 Room Air	0.000	11:40
06 Std. Sol.	0.019	11:41
07 Room Air	0.000	11:42


08 Sim Temp = 34.0°C

Simul Ser No = MP3067

Std Sol No = 201712B

County = 08

Oper No. = 888888

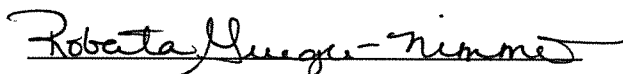

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005958
Location = TOXL 8164.13.00 06/09
04/03/2018 11:43

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:44
02 Std. Sol.	0.297	11:45
03 Room Air	0.000	11:45
04 Std. Sol.	0.299	11:46
05 Room Air	0.000	11:47
06 Std. Sol.	0.298	11:47
07 Room Air	0.000	11:48

08 Sim Temp = 34.0°C

Simul Ser No = MP3062
Std Sol No = 17350
County = 08 Oper No. = 888888

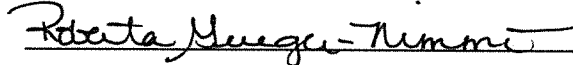

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005958
Location = TOXL 8164.13.00 06/09
04/03/2018 11:50

Test	AC	Time
01 Room Air	0.000	11:51
02 *Subject Test	INT*	11:52
03 Room Air	0.000	11:53

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT CHK
Dr. Lic. = ND/DIS821456
Lot No = 20016080A2
Cyl No = 48
Expiration Date = 09/05/2018
County = 08 Oper No. = 888888

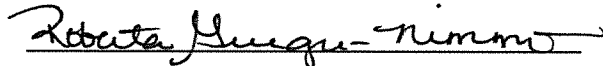

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Interferent Check*
0.10 AC + 0.05% Acetone

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005958
Location = TOXL 8164.13.00 06/09
04/03/2018 11:53

Test	AC	Time
01 Diagnostic	OK	11:54
02 Room Air	0.000	11:54
03 Subject Test 1	0.000	11:55
04 Room Air	0.000	11:57
05 Std. Gas	0.080	11:58
06 Room Air	0.000	11:59
07 Subject Test 2	0.000	11:59
08 Room Air	RFI*	12:00
09 Room Air	0.000	12:01

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 20016080A2
Cyl No = 48
Expiration Date = 09/05/2018
County = 08 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

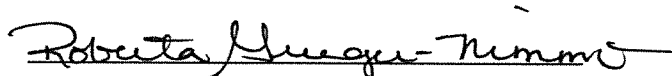

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: RFI check


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005958
Location = TOXL 8164.13.00 06/09
04/03/2018 12:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:02
02 Std. Gas	0.080	12:02
03 Room Air	0.000	12:03
04 Std. Gas	0.081	12:03
05 Room Air	0.000	12:04
06 Std. Gas	0.080	12:04
07 Room Air	0.000	12:05

Lot No = 20016080A2
Cyl No = 48
Exp Date = 09/05/2018
County = 08 Oper No. = 888888

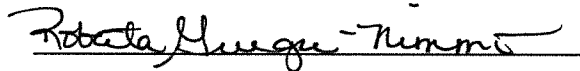
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005958
Location = TOXL 8164.13.00 06/09
04/03/2018 12:05

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:06
02 Std. Gas	0.081	12:06
03 Room Air	0.000	12:06
04 Std. Gas	0.082	12:07
05 Room Air	0.000	12:07
06 Std. Gas	0.082	12:08
07 Room Air	0.000	12:08

Lot No = 20016080A2
Cyl No = 48
Exp Date = 09/05/2018
County = 08 Oper No. = 888888

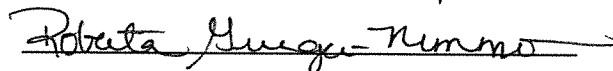

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005958
Location = TOXL 8164.13.00 06/09
04/03/2018 12:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:09
02 Std. Gas	0.081	12:09
03 Room Air	0.000	12:10
04 Std. Gas	0.080	12:10
05 Room Air	0.000	12:11
06 Std. Gas	0.081	12:11
07 Room Air	0.000	12:11

Lot No = 20016080A2
Cyl No = 48
Exp Date = 09/05/2018
County = 08 Oper No. = 888888

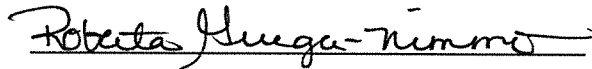

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check

0.080 AC

Form 106-I8000