

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-004205

Location: TOX L

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 97 psi Regulator: 100 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3002 Lot #: 17160 Exp. Date: 6.8.19
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.400 AC
Sim. Ser #: MP3061 Lot #: 201605F Exp. Date: 5.19.18

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR7089 Lot #: ICS 5 Exp. Date: 5.15.19
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 03417080A3 Cyl #: 27 Exp. Date: 3.5.19

Test 1 <u>0.079</u>	Test 4 <u>0.079</u>	Test 7 <u>0.079</u>
Test 2 <u>0.079</u>	Test 5 <u>0.079</u>	Test 8 <u>0.079</u>
Test 3 <u>0.079</u>	Test 6 <u>0.078</u>	Test 9 <u>0.079</u>
Average <u>0.079</u>		

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charla E. Eder
Breath Analyst Signature

4.4.2018
Date

Roberta D. Guzman
Reviewed by

4/5/2018
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.14.00 09/16
04/04/2018 16:01

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 03417080A3
Standard Cyl #? 27
Standard Expiration? 03/05/2019
Oper No? 666666

Flow Cal. Date: 04/20/2017
Slope 586
Intercept 208241

IR Calibration Date: 04/17/2015
 3um 9um

0th Coef(*100): -26411 -27902
1st Coef(*100): 266995 129434
2nd Coef(*100): 2919 1480
H2O adj(mg/l*10k): 363 481

***** Printer Test End *****

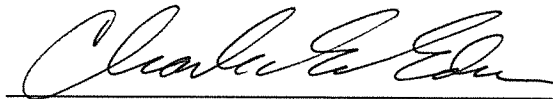

Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.14.00 09/16
04/04/2018 16:04

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:04
02 Std. Sol.	0.018	16:05
03 Room Air	0.000	16:05
04 Std. Sol.	0.018	16:06
05 Room Air	0.000	16:07
06 Std. Sol.	0.018	16:07
07 Room Air	0.000	16:08

08 Sim Temp = 34.0°C

Simul Ser No = MP3002

Std Sol No = 17160

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

Low AC

0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.14.00 09/16
04/04/2018 16:16

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:17
02 Std. Sol.	0.392	16:17
03 Room Air	0.000	16:18
04 Std. Sol.	0.395	16:19
05 Room Air	0.000	16:19
06 Std. Sol.	0.394	16:20
07 Room Air	0.000	16:20

08 Sim Temp = 34.0°C

Simul Ser No = MP3061
Std Sol No = 201605F
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.400 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.14.00 09/16
04/04/2018 16:21

Test	AC	Time
01 Room Air	0.000	16:22
02 *Subject Test	INT*	16:22
03 Room Air	0.000	16:23

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 03417080A3
Cyl No = 27
Expiration Date = 03/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.14.00 09/16
04/04/2018 16:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:26
02 Std. Gas	0.079	16:26
03 Room Air	0.000	16:27
04 Std. Gas	0.079	16:27
05 Room Air	0.000	16:28
06 Std. Gas	0.079	16:28
07 Room Air	0.000	16:29

Lot No = 03417080A3
Cyl No = 27
Exp Date = 03/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = TOXL 8164.14.00 09/16
04/04/2018 16:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:33
02 Std. Gas	0.079	16:33
03 Room Air	0.000	16:33
04 Std. Gas	0.079	16:34
05 Room Air	0.000	16:34
06 Std. Gas	0.079	16:35
07 Room Air	0.000	16:35

Lot No = 03417080A3
Cyl No = 27
Exp Date = 03/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *CALIBRATION CHECK*
0.080 AC