

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-004934

Location: TOXL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 130 psi Regulator: 125 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC  
Sim. Ser #: MP3002 Lot #: 17160 Exp. Date: 6-8-19
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.400 AC  
Sim. Ser #: MP3061 Lot #: 201605F Exp. Date: 5-19-18

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR7089 Lot #: ICS 5 Exp. Date: 5-15-19  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 03417080A3 Cyl #: 27 Exp. Date: 3-5-19

Test 1	<u>0.081</u>	Test 4	<u>0.081</u>	Test 7	<u>0.081</u>
Test 2	<u>0.082</u>	Test 5	<u>0.081</u>	Test 8	<u>0.080</u>
Test 3	<u>0.080</u>	Test 6	<u>0.081</u>	Test 9	<u>0.081</u>
Average	<u>0.081</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Ede  
Breath Analyst Signature

4.4.2018  
Date

Roberta Huego-Munoz  
Reviewed by

4-5-2018  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004934  
Location = TOXL                 8164.14.00 09/16  
04/04/2018                        13:13

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:                Enabled  
Start Test Sequence:            DABACABA  
Display Prelim Rslt?            Yes  
Display Third Digit?            Yes  
Inhib Printer(Y/N)?            No  
Display Volume?                No  
Disable On Memfull?            Yes  
# of Print Copies?              1  
Select Std (D/W/I)?            Dry  
Standard Value?                0.080  
Standard Lot #?                03417080A3  
Standard Cyl #?                27  
Standard Expiration?            03/05/2019  
Oper No?                        666666

Flow Cal. Date:                01/20/2010  
Slope                            698  
Intercept                       -694859

IR Calibration Date:            05/22/2017

	3um	9um
0th Coef(*100):	-19346	-20433
1st Coef(*100):	262347	132345
2nd Coef(*100):	1369	1153
H2O adj(mg/l*10k):	384	416

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004934  
Location = TOXL      8164.14.00 09/16  
04/04/2018      13:16

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:16
02 Std. Sol.	0.018	13:17
03 Room Air	0.000	13:18
04 Std. Sol.	0.019	13:18
05 Room Air	0.000	13:19
06 Std. Sol.	0.020	13:20
07 Room Air	0.000	13:20


08 Sim Temp = 34.0°C

Simul Ser No = MP3002

Std Sol No = 17160

County = 08

Oper No. = 666666

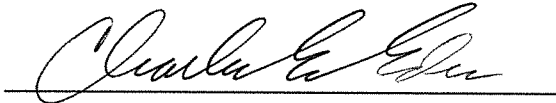

Operator Signature  
CHARLES EDER

Remarks:

Low AC  
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004934  
Location = TOXL      8164.14.00 09/16  
04/04/2018      13:24

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:25
02 Std. Sol.	0.397	13:25
03 Room Air	0.000	13:26
04 Std. Sol.	0.399	13:27
05 Room Air	0.000	13:27
06 Std. Sol.	0.399	13:28
07 Room Air	0.000	13:29

08 Sim Temp = 34.0°C

Simul Ser No = MP3061  
Std Sol No = 201605F  
County = 08      Oper No. = 666666

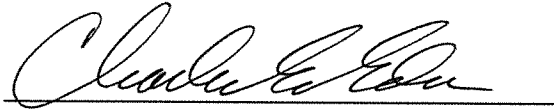

Operator Signature  
CHARLES EDER

Remarks: ~~Interferent Check cee~~  
0.10 cee HIGH AC  
Form 106-I8000      0.400 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004934  
Location = TOXL      8164.14.00 09/16  
04/04/2018      13:29

Test	AC	Time
01 Room Air	0.000	13:30
02 *Subject Test	INT*	13:31
03 Room Air	0.000	13:31

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 03417080A3  
Cyl No = 27  
Expiration Date = 03/05/2019  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent Check*  
*0.10AC + 0.05% ACETONE*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004934  
Location = TOXL                    8164.14.00 09/16  
04/04/2018                            13:36

Test	AC	Time
01 Diagnostic	OK	13:36
02 Room Air	0.000	13:37
03 *Subject Test	RFI*	13:37
04 Room Air	0.000	13:38

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 03417080A3  
Cyl No = 27  
Expiration Date = 03/05/2019  
County = 08                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

*RFI CHECK*


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004934  
Location = TOXL      8164.14.00 09/16  
04/04/2018      13:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:33
02 Std. Gas	0.081	13:33
03 Room Air	0.000	13:34
04 Std. Gas	0.082	13:34
05 Room Air	0.000	13:35
06 Std. Gas	0.080	13:35
07 Room Air	0.000	13:35

Lot No = 03417080A3  
Cyl No = 27  
Exp Date = 03/05/2019  
County = 08      Oper No. = 666666

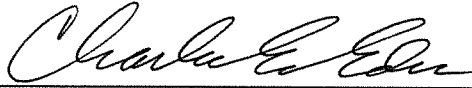

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004934  
Location = TOXL      8164.14.00 09/16  
04/04/2018      13:38

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:38
02 Std. Gas	0.081	13:39
03 Room Air	0.000	13:39
04 Std. Gas	0.081	13:40
05 Room Air	0.000	13:40
06 Std. Gas	0.081	13:40
07 Room Air	0.000	13:41

Lot No = 03417080A3  
Cyl No = 27  
Exp Date = 03/05/2019  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004934  
Location = TOXL                8164.14.00 09/16  
04/04/2018                        13:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:42
02 Std. Gas	0.081	13:42
03 Room Air	0.000	13:43
04 Std. Gas	0.080	13:43
05 Room Air	0.000	13:43
06 Std. Gas	0.081	13:44
07 Room Air	0.000	13:44

Lot No = 03417080A3  
Cyl No = 27  
Exp Date = 03/05/2019  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *CALIBRATION CHECK*  
*0.080AC*