



ETHANOL GAS STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59282 (10-2013)

Chemical Test Operator Name (Print) DELBERT HOFFMANN		
Location SHER	Intoxilyzer® Serial Number 80-005950	
Gas Lot Number 34917080A3	Gas Cylinder Number 4	Gas Expiration Date 02-05-2020

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0.082 AC
 - B. 0.082 AC
 - C. 0.081 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature Delbert Hoffmann	Date 04-06-2018
Reviewed By (Crime Laboratory Use Only) Roberta Suega-Morris	Date 04-06-2018

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005950
Location = SHER 8164.13.00 06/09
04/06/2018 09:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:27
02 Std. Gas	0.082	09:27
03 Room Air	0.000	09:28
04 Std. Gas	0.082	09:28
05 Room Air	0.000	09:29
06 Std. Gas	0.081	09:29
07 Room Air	0.000	09:30


Lot No = 34917080A3

Cyl No = 4

Exp Date = 02/05/2020

County = 42

Oper No. = 048909

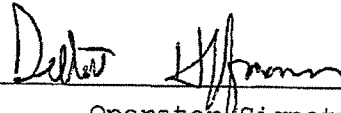

Operator Signature
DELBERT HOFFMANN

Remarks:

ACA TEST

Form 106-I8000