

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-004945

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 1014 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3002 Lot #: 17160 Exp. Date: 6.8.19
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.400 AC
Sim. Ser #: MP3061 Lot #: 201605F Exp. Date: 5.19.18

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR7089 Lot #: IPS 5 Exp. Date: 5.15.19
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 19817080A4 Cyl #: 21 Exp. Date: 9/5/19

Test 1	<u>0.081</u>	Test 4	<u>0.081</u>	Test 7	<u>0.081</u>
Test 2	<u>0.081</u>	Test 5	<u>0.081</u>	Test 8	<u>0.082</u>
Test 3	<u>0.081</u>	Test 6	<u>0.081</u>	Test 9	<u>0.081</u>
Average	<u>0.081</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION, REPLACED BREATH HOSE AND SIMULATOR RETURN O-RINGS

Instrument is acceptable to be used in the field.

Charles E. Ede
Breath Analyst Signature

09 APR 2018
Date

Roberta Yungui-Nunne
Reviewed by

09 APR 2018
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/06/2018 16:52

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19817080A4
Standard Cyl #? 21
Standard Expiration? 09/05/2019
Oper No? 666666

Flow Cal. Date: 09/26/2012
Slope 622
Intercept -485836

IR Calibration Date: 04/22/2016
 3um 9um

0th Coef(*100): -23427 -26406
1st Coef(*100): 266989 144341
2nd Coef(*100): 2085 1078
H2O adj(mg/l*10k): 539 400

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/06/2018 16:55

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:56
02 Std. Sol.	0.019	16:57
03 Room Air	0.000	16:57
04 Std. Sol.	0.020	16:58
05 Room Air	0.000	16:58
06 Std. Sol.	0.020	16:59
07 Room Air	0.000	16:59

08 Sim Temp = 34.0°C

Simul Ser No = MP3002
Std Sol No = 17160
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
 0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/09/2018 11:46

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:46
02 Std. Sol.	0.394	11:47
03 Room Air	0.000	11:48
04 Std. Sol.	0.396	11:48
05 Room Air	0.000	11:49
06 Std. Sol.	0.398	11:50
07 Room Air	0.000	11:50

08 Sim Temp = 34.0°C

Simul Ser No = MP3061
Std Sol No = 201605F
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *HIGH AC*
0.400 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/09/2018 11:52

Test	AC	Time
01 Room Air	0.000	11:53
02 *Subject Test	INT*	11:54
03 Room Air	0.000	11:54

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 21
Expiration Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10 AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/09/2018 11:55

Test	AC	Time
01 Diagnostic	OK	11:55
02 Room Air	0.000	11:56
03 *Subject Test	RFI*	11:56
04 Room Air	0.000	11:57

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 21
Expiration Date = 09/05/2019
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

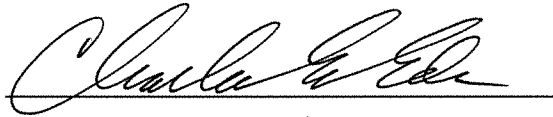

Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/09/2018 11:57

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:58
02 Std. Gas	0.081	11:58
03 Room Air	0.000	11:59
04 Std. Gas	0.081	11:59
05 Room Air	0.000	12:00
06 Std. Gas	0.081	12:00
07 Room Air	0.000	12:01

Lot No = 19817080A4
Cyl No = 21
Exp Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/09/2018 12:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:02
02 Std. Gas	0.081	12:02
03 Room Air	0.000	12:02
04 Std. Gas	0.081	12:03
05 Room Air	0.000	12:03
06 Std. Gas	0.081	12:04
07 Room Air	0.000	12:04

Lot No = 19817080A4
Cyl No = 21
Exp Date = 09/05/2019
County = 08 Oper No. = 666666

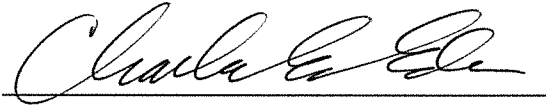

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/09/2018 12:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:05
02 Std. Gas	0.081	12:05
03 Room Air	0.000	12:06
04 Std. Gas	0.082	12:06
05 Room Air	0.000	12:07
06 Std. Gas	0.081	12:07
07 Room Air	0.000	12:07

Lot No = 19817080A4
Cyl No = 21
Exp Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *CALIBRATION CHECK*
0.080 AC