

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005358

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 986 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3002 Lot #: 17160 Exp. Date: 6.8.19
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.400 AC
Sim. Ser #: MP3061 Lot #: 201605F Exp. Date: 5.19.18

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR7089 Lot #: ICS 5 Exp. Date: 5.15.19
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 19817080A4 Cyl #: 21 Exp. Date: 9/5/2019

Test 1 <u>0.079</u>	Test 4 <u>0.081</u>	Test 7 <u>0.081</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.079</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

C. Remarks/Maintenance: ANNUAL INSPECTION, REPLACED BREATH HOSE O-RING.

Instrument is acceptable to be used in the field.

Christina E. Eden
Breath Analyst Signature

4.6.2018
Date

Roberta Mungu-Nimmo
Reviewed by

9 Apr 18
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005358
Location = TOXL 8164.14.00 09/16
04/06/2018 16:11

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19817080A4
Standard Cyl #? 21
Standard Expiration? 09/05/2019
Oper No? 666666

Flow Cal. Date: 04/17/2014
Slope 622
Intercept -324577

IR Calibration Date: 04/17/2015
3um 9um

0th Coef(*100):	-16867	-25778
1st Coef(*100):	266890	139601
2nd Coef(*100):	3270	1479
H2O adj(mg/l*10k):	332	443

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005358
Location = TOXL 8164.14.00 09/16
04/06/2018 16:11

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:12
02 Std. Sol.	0.018	16:13
03 Room Air	0.000	16:13
04 Std. Sol.	0.018	16:14
05 Room Air	0.000	16:15
06 Std. Sol.	0.019	16:15
07 Room Air	0.000	16:16

08 Sim Temp = 34.0°C

Simul Ser No = MP3002
Std Sol No = 17160
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005358
Location = TOXL 8164.14.00 09/16
04/06/2018 16:20

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:21
02 Std. Sol.	0.400	16:21
03 Room Air	0.000	16:22
04 Std. Sol.	0.403	16:23
05 Room Air	0.000	16:23
06 Std. Sol.	0.402	16:24
07 Room Air	0.000	16:25

08 Sim Temp = 34.0°C

Simul Ser No = MP3061
Std Sol No = 201605F
County = 08 Oper No. = 666666

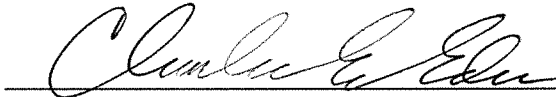

Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.400 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005358
Location = TOXL 8164.14.00 09/16
04/06/2018 16:26

Test	AC	Time
01 Room Air	0.000	16:26
02 *Subject Test	INT*	16:27
03 Room Air	0.000	16:28

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 21
Expiration Date = 09/05/2019
County = 08 Oper No. = 666666

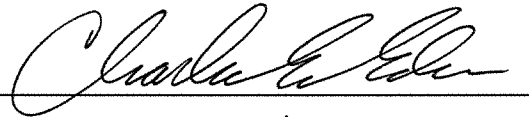

Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10 AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005358
Location = TOXL 8164.14.00 09/16
04/06/2018 16:28

Test	AC	Time
01 Diagnostic	OK	16:29
02 Room Air	0.000	16:29
03 *Subject Test	RFI*	16:30
04 Room Air	0.000	16:30

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 21
Expiration Date = 09/05/2019
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

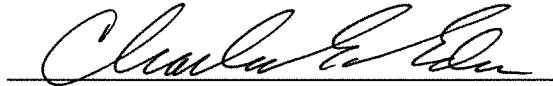

Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005358
Location = TOXL 8164.14.00 09/16
04/06/2018 16:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:31
02 Std. Gas	0.079	16:31
03 Room Air	0.000	16:32
04 Std. Gas	0.080	16:32
05 Room Air	0.000	16:33
06 Std. Gas	0.080	16:33
07 Room Air	0.000	16:34

Lot No = 19817080A4
Cyl No = 21
Exp Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005358
Location = TOXL 8164.14.00 09/16
04/06/2018 16:34

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:34
02 Std. Gas	0.081	16:35
03 Room Air	0.000	16:35
04 Std. Gas	0.080	16:36
05 Room Air	0.000	16:36
06 Std. Gas	0.080	16:37
07 Room Air	0.000	16:37

Lot No = 19817080A4
Cyl No = 21
Exp Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005358
Location = TOXL 8164.14.00 09/16
04/06/2018 16:37

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:38
02 Std. Gas	0.081	16:38
03 Room Air	0.000	16:39
04 Std. Gas	0.079	16:39
05 Room Air	0.000	16:40
06 Std. Gas	0.080	16:40
07 Room Air	0.000	16:41

Lot No = 19817080A4
Cyl No = 21
Exp Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *CALIBRATION CHECK*
0.080 AC