

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 8D-006504

Location: TOLL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 403 psi Regulator: 400 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC  
Sim. Ser #: MP3060 Lot #: 16200 Exp. Date: 17 Aug 18
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.300 AC  
Sim. Ser #: MP3062 Lot #: 17350 Exp. Date: 11 Oct 19

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR5144 Lot #: 1CS5 Exp. Date: 15 May 19  
 Display reads "Interferent Detect"
3.  LRFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 2001608DA2 Cyl #: 48 Exp. Date: 5 Sep 18

Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.082</u>
Test 2 <u>0.082</u>	Test 5 <u>0.082</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.082</u>
Average <u>0.081</u>		

C. Remarks/Maintenance: Annual inspection. Completed calibration adjustment / tare of tank pressure sensor. Replaced O-ring on breath hose.

Instrument is acceptable to be used in the field.

Roberta Huggins  
Breath Analyst Signature

9 Apr 18  
Date

Charles E. Egan  
Reviewed by

09 APR 2018  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006504  
Location = TOXL                    8164.14.00 09/16  
04/09/2018                            11:18

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup


Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            20016080A2  
Standard Cyl #?            48  
Standard Expiration?       09/05/2018  
Oper No?                    888888

Flow Cal. Date:            08/20/2015  
Slope                      681  
Intercept                  -673151

IR Calibration Date:       05/31/2017

	3um	9um
0th Coef(*100):	-21488	-16160
1st Coef(*100):	265581	137324
2nd Coef(*100):	2507	1149
H2O adj(mg/l*10k):	519	348

\*\*\*\*\* Printer Test End \*\*\*\*\*

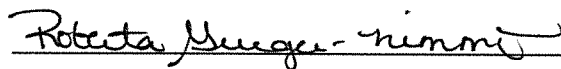

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Print Test

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006504  
Location = TOXL      8164.14.00 09/16  
04/09/2018      11:30

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:31
02 Std. Sol.	0.020	11:32
03 Room Air	0.000	11:32
04 Std. Sol.	0.020	11:33
05 Room Air	0.000	11:33
06 Std. Sol.	0.020	11:34
07 Room Air	0.000	11:35

08 Sim Temp = 34.0°C

Simul Ser No = MP3060  
Std Sol No = 16200  
County = 08      Oper No. = 888888

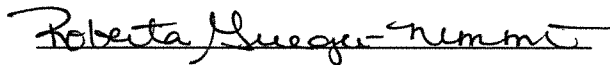

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Low AC  
0.020 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006504  
Location = TOXL      8164.14.00 09/16  
04/09/2018      11:37

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:38
02 Std. Sol.	0.297	11:39
03 Room Air	0.000	11:40
04 Std. Sol.	0.297	11:40
05 Room Air	0.000	11:41
06 Std. Sol.	0.297	11:42
07 Room Air	0.000	11:42

08 Sim Temp = 34.0°C

Simul Ser No = MP3062  
Std Sol No = 17350  
County = 08      Oper No. = 888888

Roberta Grieger-Nimmo  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: High AC  
0.300 AC


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006504  
Location = TOXL      8164.14.00 09/16  
04/09/2018      11:57

Test	AC	Time
01 Room Air	0.000	11:58
02 *Subject Test	INT*	11:59
03 Room Air	0.000	12:00

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 20016080A2  
Cyl No = 48  
Expiration Date = 09/05/2018  
County = 08      Oper No. = 888888

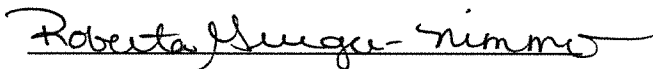

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: *Interferent Check*

*0.10 AC + 0.05% Acetone*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006504  
Location = TOXL                    8164.14.00 09/16  
04/09/2018                            12:00

Test	AC	Time
01 Diagnostic	OK	12:01
02 Room Air	RFI*	12:01
03 Room Air	0.000	12:02

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI  
Dr. Lic. = ND/DIS821456  
Lot No = 20016080A2  
Cyl No = 48  
Expiration Date = 09/05/2018  
County = 08                            Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

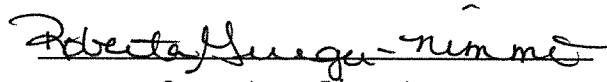

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006504  
Location = TOXL      8164.14.00 09/16  
04/09/2018      11:44

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:44
02 Std. Gas	0.081	11:45
03 Room Air	0.000	11:45
04 Std. Gas	0.082	11:45
05 Room Air	0.000	11:46
06 Std. Gas	0.081	11:46
07 Room Air	0.000	11:47

Lot No = 20016080A2  
Cyl No = 48  
Exp Date = 09/05/2018  
County = 08      Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check  
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006504  
Location = TOXL      8164.14.00 09/16  
04/09/2018      11:49

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:49
02 Std. Gas	0.081	11:50
03 Room Air	0.000	11:50
04 Std. Gas	0.082	11:51
05 Room Air	0.000	11:51
06 Std. Gas	0.081	11:52
07 Room Air	0.000	11:52

Lot No = 20016080A2  
Cyl No = 48  
Exp Date = 09/05/2018  
County = 08      Oper No. = 888888

*Roberta Grieger-Nimmo*

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: *Calibration Check*

*0.080 AC*

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006504  
Location = TOXL      8164.14.00 09/16  
04/09/2018      11:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:53
02 Std. Gas	0.082	11:53
03 Room Air	0.000	11:54
04 Std. Gas	0.081	11:54
05 Room Air	0.000	11:55
06 Std. Gas	0.082	11:55
07 Room Air	0.000	11:56


Lot No = 20016080A2

Cyl No = 48

Exp Date = 09/05/2018

County = 08

Oper No. = 888888

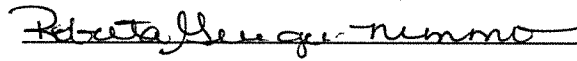

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check

0.080 AC

Form 106-I8000