

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-004188

Location: TKL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 342 psi Regulator: 350 psi

B. Tests (Sign and attach test records):

- Configure simulator for the following tests (Level 1,S).
1. Wet Calibration Check
    - a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC  
Sim. Ser #: mp3060 Lot #: 16200 Exp. Date: 17 Aug 18
    - b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.300 AC  
Sim. Ser #: mp3062 Lot #: 17350 Exp. Date: 11 Oct 19

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR5144 Lot #: 1055 Exp. Date: 15 May 19  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 20016080A2 Cyl #: 48 Exp. Date: 5 Sep 18

Test 1	<u>0.083</u>	Test 4	<u>0.083</u>	Test 7	<u>0.083</u>
Test 2	<u>0.083</u>	Test 5	<u>0.083</u>	Test 8	<u>0.083</u>
Test 3	<u>0.083</u>	Test 6	<u>0.083</u>	Test 9	<u>0.083</u>
Average <u>0.083</u>					

C. Remarks/Maintenance: Annual inspection. Replaced O-ring on breath hose and simulator return.

Instrument is acceptable to be used in the field.

Roberta Mueggel-Nimmer  
Breath Analyst Signature

10 Apr 18  
Date

Charles E. Edin  
Reviewed by

10 APR. 2018  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004188  
Location = TOXL                    8164.14.00 09/16  
04/10/2018                            09:32

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=  
|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=  
|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Wet  
Standard Value?            0.020  
Standard Solution #?        16200  
Sim Serial #?              MP3060  
Oper No?                    888888

Flow Cal. Date:            07/05/2011  
Slope                      668  
Intercept                  -597204

IR Calibration Date:        04/05/2012  
                                  3um                    9um

-----  
0th Coef(\*100):            -15090                    -16406  
1st Coef(\*100):            263383                    126966  
2nd Coef(\*100):            6533                      2371  
H2O adj(mg/l\*10k):        801                        809

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Print Test

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

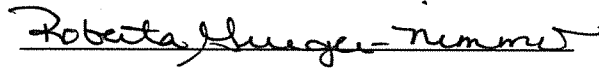
CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004188  
Location = TOXL      8164.14.00 09/16  
04/10/2018      09:27

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:28
02 Std. Sol.	0.021	09:29
03 Room Air	0.000	09:29
04 Std. Sol.	0.020	09:30
05 Room Air	0.000	09:30
06 Std. Sol.	0.021	09:31
07 Room Air	0.000	09:32

08 Sim Temp = 34.0°C

Simul Ser No = MP3060  
Std Sol No = 16200  
County = 08      Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: low AC  
0.020AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

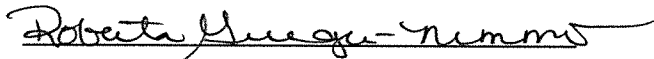
CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004188  
Location = TOXL      8164.14.00 09/16  
04/10/2018      09:33

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:34
02 Std. Sol.	0.298	09:35
03 Room Air	0.000	09:35
04 Std. Sol.	0.298	09:36
05 Room Air	0.000	09:37
06 Std. Sol.	0.300	09:37
07 Room Air	0.000	09:38

08 Sim Temp = 34.0°C

Simul Ser No = MP3062  
Std Sol No = 17350  
County = 08      Oper No. = 888888



Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: High AC  
0.300 AC

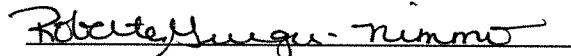
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004188  
Location = TOXL                8164.14.00 09/16  
04/10/2018                        09:21

Test	AC	Time
01 Room Air	0.000	09:22
02 *Subject Test	INT*	09:22
03 Room Air	0.000	09:23

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                Weight = 150  
Test = DUI                        Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 20016080A2  
Cyl No = 48  
Expiration Date = 09/05/2018  
County = 08                        Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Interferent Check  
0.10AC + 0.05% Acetone

Form 106-I8000



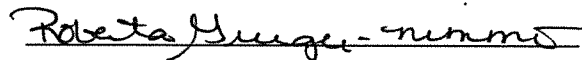
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004188  
Location = TOXL      8164.14.00 09/16  
04/10/2018      08:51

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:52
02 Std. Gas	0.083	08:52
03 Room Air	0.000	08:53
04 Std. Gas	0.083	08:53
05 Room Air	0.000	08:53
06 Std. Gas	0.083	08:54
07 Room Air	0.000	08:54

Lot No = 20016080A2  
Cyl No = 48  
Exp Date = 09/05/2018  
County = 08      Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check  
0.080 AC

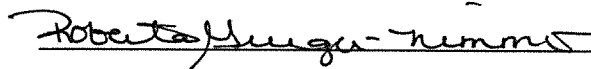
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004188  
Location = TOXL      8164.14.00 09/16  
04/10/2018      08:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:55
02 Std. Gas	0.083	08:56
03 Room Air	0.000	08:56
04 Std. Gas	0.083	08:56
05 Room Air	0.000	08:57
06 Std. Gas	0.083	08:57
07 Room Air	0.000	08:58

Lot No = 20016080A2  
Cyl No = 48  
Exp Date = 09/05/2018  
County = 08      Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check  
0.080 AC



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004188  
Location = TOXL      8164.14.00 09/16  
04/10/2018      08:58

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:59
02 Std. Gas	0.083	08:59
03 Room Air	0.000	09:00
04 Std. Gas	0.083	09:00
05 Room Air	0.000	09:00
06 Std. Gas	0.083	09:01
07 Room Air	0.000	09:01

Lot No = 20016080A2  
Cyl No = 48  
Exp Date = 09/05/2018  
County = 08      Oper No. = 888888

*Roberta Grieger-Nimmo*

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: *Calibration Check*  
*0.080 AC*

Form 106-I8000