

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005948

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 705 psi Regulator: 900 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3002 Lot #: 17160 Exp. Date: 6.8.19
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.400 AC
Sim. Ser #: MP3061 Lot #: 201605F Exp. Date: 5.19.18

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR7089 Lot #: ICS #5 Exp. Date: 5.15.19
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 19817080A4 Cyl #: 21 Exp. Date: 9.5.19

Test 1	<u>0.081</u>	Test 4	<u>0.080</u>	Test 7	<u>0.080</u>
Test 2	<u>0.080</u>	Test 5	<u>0.080</u>	Test 8	<u>0.080</u>
Test 3	<u>0.081</u>	Test 6	<u>0.081</u>	Test 9	<u>0.080</u>
Average	<u>0.080</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION, EXTERNAL BATTERY
REPLACED AND TRAINED, REPLACED PRESSURE GAUGE O-RINGS

Instrument is acceptable to be used in the field.

Charles E. Egan
Breath Analyst Signature

4.9.2018
Date

Roberta Muegge-Humms
Reviewed by

10 Apr 18
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005948
Location = TOXL 8164.14.00 09/16
04/09/2018 15:24

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:24
02 Std. Sol.	0.018	15:25
03 Room Air	0.000	15:26
04 Std. Sol.	0.018	15:26
05 Room Air	0.000	15:27
06 Std. Sol.	0.018	15:28
07 Room Air	0.000	15:28

08 Sim Temp = 34.0°C

Simul Ser No = MP3002
Std Sol No = 17160
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
 0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005948
Location = TOXL 8164.14.00 09/16
04/09/2018 15:29

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:29
02 Std. Sol.	0.402	15:30
03 Room Air	0.000	15:31
04 Std. Sol.	0.402	15:31
05 Room Air	0.000	15:32
06 Std. Sol.	0.402	15:33
07 Room Air	0.000	15:33

08 Sim Temp = 34.0°C

Simul Ser No = MP3061
Std Sol No = 201605F
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.400 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005948
Location = TOXL 8164.14.00 09/16
04/09/2018 15:34

Test	AC	Time
01 Room Air	0.000	15:35
02 *Subject Test	INT*	15:36
03 Room Air	0.000	15:36

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 21
Expiration Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10AE + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005948
Location = TOXL 8164.14.00 09/16
04/09/2018 15:37

Test	AC	Time
01 Diagnostic	OK	15:37
02 Room Air	0.000	15:38
03 *Subject Test	RFI*	15:38
04 Room Air	0.000	15:38

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 21
Expiration Date = 09/05/2019
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005948
Location = TOXL 8164.14.00 09/16
04/09/2018 15:39

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:39
02 Std. Gas	0.081	15:40
03 Room Air	0.000	15:40
04 Std. Gas	0.080	15:40
05 Room Air	0.000	15:41
06 Std. Gas	0.081	15:41
07 Room Air	0.000	15:42

Lot No = 19817080A4
Cyl No = 21
Exp Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005948
Location = TOXL 8164.14.00 09/16
04/09/2018 15:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:44
02 Std. Gas	0.080	15:44
03 Room Air	0.000	15:45
04 Std. Gas	0.080	15:45
05 Room Air	0.000	15:46
06 Std. Gas	0.081	15:46
07 Room Air	0.000	15:46

Lot No = 19817080A4
Cyl No = 21
Exp Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005948
Location = TOXL 8164.14.00 09/16
04/09/2018 15:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:47
02 Std. Gas	0.080	15:47
03 Room Air	0.000	15:48
04 Std. Gas	0.080	15:48
05 Room Air	0.000	15:49
06 Std. Gas	0.080	15:49
07 Room Air	0.000	15:50

Lot No = 19817080A4
Cyl No = 21
Exp Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000