

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-003059 Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 21 psi Regulator: 225 psi

B. Tests (Sign and attach test records):

- Configure simulator for the following tests (Level 1,S).
1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3060 Lot #: 16200 Exp. Date: 17 Aug 18
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: MP3062 Lot #: 17350 Exp. Date: 11 Oct 19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR5144 Lot #: 1CS5 Exp. Date: 15 May 19
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 20016080A2 Cyl #: 48 Exp. Date: 5 Sep 18

Test 1	<u>0.079</u>	Test 4	<u>0.081</u>	Test 7	<u>0.080</u>
Test 2	<u>0.081</u>	Test 5	<u>0.080</u>	Test 8	<u>0.080</u>
Test 3	<u>0.081</u>	Test 6	<u>0.081</u>	Test 9	<u>0.081</u>
Average <u>0.080</u>					

C. Remarks/Maintenance: Annual inspection. Replaced o-rings on simulator return and breath hose.

Instrument is acceptable to be used in the field.

Roberta Huegu-Nimmo
Breath Analyst Signature

10 APR 18
Date

Charles E. Egan
Reviewed by

10 APR. 2018
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003059
Location = TOXL 8164.14.00 09/16
04/10/2018 13:48

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup

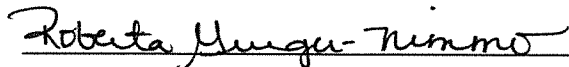
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 20016080A2
Standard Cyl #? 48
Standard Expiration? 09/05/2018
Oper No? 888888

Flow Cal. Date: 04/07/2008
Slope 713
Intercept -692349

IR Calibration Date: 04/10/2015
 3um 9um

0th Coef(*100): -22715 -12336
1st Coef(*100): 274413 134965
2nd Coef(*100): 2480 1348
H2O adj(mg/l*10k): 437 307

***** Printer Test End *****



Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Print Test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

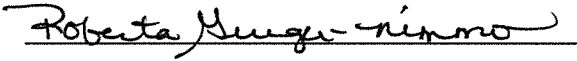
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003059
Location = TOXL 8164.14.00 09/16
04/10/2018 14:09

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:09
02 Std. Sol.	0.018	14:10
03 Room Air	0.000	14:11
04 Std. Sol.	0.019	14:11
05 Room Air	0.000	14:12
06 Std. Sol.	0.019	14:13
07 Room Air	0.000	14:13

08 Sim Temp = 34.0°C

Simul Ser No = MP3060
Std Sol No = 16200
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Low AC*
0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

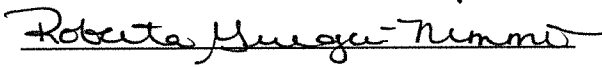
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003059
Location = TOXL 8164.14.00 09/16
04/10/2018 14:15

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:15
02 Std. Sol.	0.292	14:16
03 Room Air	0.000	14:17
04 Std. Sol.	0.293	14:17
05 Room Air	0.000	14:18
06 Std. Sol.	0.294	14:19
07 Room Air	0.000	14:19

08 Sim Temp = 34.0°C

Simul Ser No = MP3062
Std Sol No = 17350
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC
 0.300AC

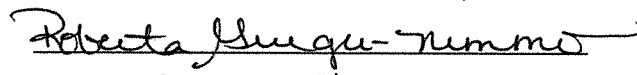
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003059
Location = TOXL 8164.14.00 09/16
04/10/2018 14:03

Test	AC	Time
01 Room Air	0.000	14:04
02 *Subject Test	INT*	14:04
03 Room Air	0.000	14:05

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 20016080A2
Cyl No = 48
Expiration Date = 09/05/2018
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Interferent Check
0.10 AC + 0.059g

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

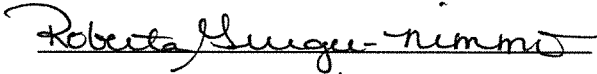
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003059
Location = TOXL 8164.14.00 09/16
04/10/2018 14:05

Test	AC	Time
01 Diagnostic	OK	14:06
02 Room Air	0.000	14:07
03 *Subject Test	RFI*	14:07
04 Room Air	0.000	14:07

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 20016080A2
Cyl No = 48
Expiration Date = 09/05/2018
County = 08 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check

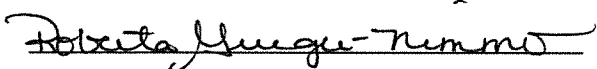
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003059
Location = TOXL 8164.14.00 09/16
04/10/2018 13:50

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:51
02 Std. Gas	0.079	13:51
03 Room Air	0.000	13:52
04 Std. Gas	0.081	13:52
05 Room Air	0.000	13:52
06 Std. Gas	0.081	13:53
07 Room Air	0.000	13:53

Lot No = 20016080A2
Cyl No = 48
Exp Date = 09/05/2018
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

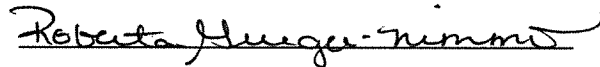
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003059
Location = TOXL 8164.14.00 09/16
04/10/2018 13:54

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:54
02 Std. Gas	0.081	13:55
03 Room Air	0.000	13:55
04 Std. Gas	0.080	13:56
05 Room Air	0.000	13:56
06 Std. Gas	0.081	13:56
07 Room Air	0.000	13:57

Lot No = 20016080A2
Cyl No = 48
Exp Date = 09/05/2018
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080AC

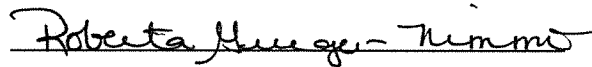
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003059
Location = TOXL 8164.14.00 09/16
04/10/2018 13:58

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:58
02 Std. Gas	0.080	13:59
03 Room Air	0.000	13:59
04 Std. Gas	0.080	13:59
05 Room Air	0.000	14:00
06 Std. Gas	0.081	14:00
07 Room Air	0.000	14:01

Lot No = 20016080A2
Cyl No = 48
Exp Date = 09/05/2018
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Form 106-I8000