

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-003060

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 187 psi Regulator: 200 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3060 Lot #: 16200 Exp. Date: 17 Aug 18
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: MP3062 Lot #: 172350 Exp. Date: 11 Oct 19
BLW

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR5144 Lot #: 1255 Exp. Date: 15 May 19

Display reads "Interferent Detect"

3. RFI Check (CMS Mode) Display reads "RFI Detect"

4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 20016080A2Eyl #: 48 Exp. Date: 5 Sep 18

Test 1	<u>0.080</u>	Test 4	<u>0.080</u>	Test 7	<u>0.081</u>
Test 2	<u>0.081</u>	Test 5	<u>0.081</u>	Test 8	<u>0.080</u>
Test 3	<u>0.080</u>	Test 6	<u>0.081</u>	Test 9	<u>0.081</u>
Average <u>0.081</u>					

C. Remarks/Maintenance: Annual inspection. Replaced o-ring on breath hose and simulator return.

Instrument is acceptable to be used in the field.

Roberta Henriquez Nimmis
Breath Analyst Signature

10 Apr 18
Date

Charles E. Egan
Reviewed by

10 APR. 18
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.14.00 09/16
04/10/2018 14:24

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 20016080A2
Standard Cyl #? 48
Standard Expiration? 09/05/2018
Oper No? 888888

Flow Cal. Date: 04/08/2008
Slope 702
Intercept -647610

IR Calibration Date: 06/04/2008

	3um	9um
0th Coef(*100):	-5190	-7737
1st Coef(*100):	270580	138779
2nd Coef(*100):	2499	956
H2O adj(mg/l*10k):	377	188

***** Printer Test End *****

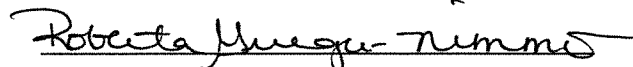

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Print Test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.14.00 09/16
04/10/2018 14:44

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:45
02 Std. Sol.	0.020	14:46
03 Room Air	0.000	14:46
04 Std. Sol.	0.020	14:47
05 Room Air	0.000	14:48
06 Std. Sol.	0.021	14:48
07 Room Air	0.000	14:49

08 Sim Temp = 34.0°C

Simul Ser No = MP3060
Std Sol No = 16200
County = 08 Oper No. = 888888

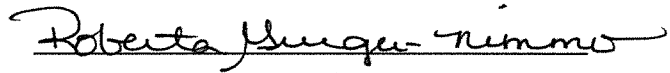
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: low AC
0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.14.00 09/16
04/10/2018 15:01

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:02
02 Std. Sol.	0.301	15:03
03 Room Air	0.000	15:03
04 Std. Sol.	0.302	15:04
05 Room Air	0.000	15:04
06 Std. Sol.	0.301	15:05
07 Room Air	0.000	15:06


08 Sim Temp = 34.0°C

Simul Ser No = MP3065

Std Sol No = 17350

County = 08

Oper No. = 888888

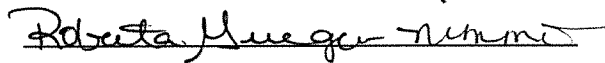

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.14.00 09/16
04/10/2018 14:38

Test	AC	Time
01 Room Air	0.000	14:39
02 *Subject Test	INT*	14:40
03 Room Air	0.000	14:40

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 20016080A2
Cyl No = 48
Expiration Date = 09/05/2018
County = 08 Oper No. = 888888

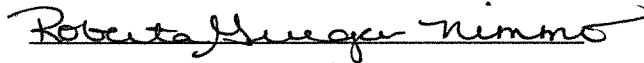

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Interferent Check*
0.10 AC + 0.05% Acetone

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.14.00 09/16
04/10/2018 14:41

Test	AC	Time
01 Diagnostic	OK	14:42
02 Room Air	0.000	14:42
03 *Subject Test	RFI*	14:42
04 Room Air	0.000	14:43

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 20016080A2
Cyl No = 48
Expiration Date = 09/05/2018
County = 08 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

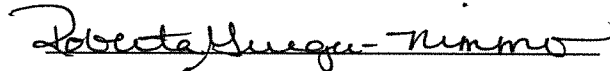

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.14.00 09/16
04/10/2018 14:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:27
02 Std. Gas	0.080	14:27
03 Room Air	0.000	14:28
04 Std. Gas	0.081	14:28
05 Room Air	0.000	14:29
06 Std. Gas	0.080	14:29
07 Room Air	0.000	14:30

Lot No = 20016080A2
Cyl No = 48
Exp Date = 09/05/2018
County = 08 Oper No. = 888888

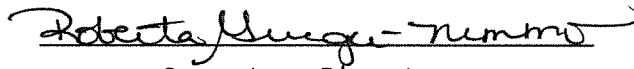

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.14.00 09/16
04/10/2018 14:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:31
02 Std. Gas	0.080	14:31
03 Room Air	0.000	14:32
04 Std. Gas	0.081	14:32
05 Room Air	0.000	14:33
06 Std. Gas	0.081	14:33
07 Room Air	0.000	14:33


Lot No = 20016080A2

Cyl No = 48

Exp Date = 09/05/2018

County = 08

Oper No. = 888888

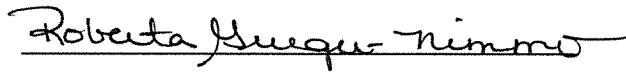

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = TOXL 8164.14.00 09/16
04/10/2018 14:34

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:35
02 Std. Gas	0.081	14:35
03 Room Air	0.000	14:36
04 Std. Gas	0.080	14:36
05 Room Air	0.000	14:36
06 Std. Gas	0.081	14:37
07 Room Air	0.000	14:37

Lot No = 20016080A2
Cyl No = 48
Exp Date = 09/05/2018
County = 08 Oper No. = 888888

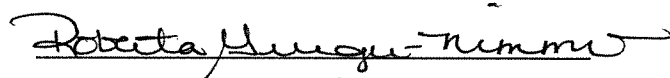
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC