

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-004939 Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 747 psi Regulator: 750 psi

B. Tests (Sign and attach test records):

- Configure simulator for the following tests (Level 1,S).
1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3002 Lot #: 17160 Exp. Date: 6.8.19
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.400 AC
Sim. Ser #: MP3061 Lot #: 201605F Exp. Date: 5.19.18

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR7089 Lot #: ICS #5 Exp. Date: 5.15.19
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 19817080A4 Cyl #: 21 Exp. Date: 9.5.19

Test 1	<u>0.081</u>	Test 4	<u>0.081</u>	Test 7	<u>0.081</u>
Test 2	<u>0.080</u>	Test 5	<u>0.081</u>	Test 8	<u>0.081</u>
Test 3	<u>0.080</u>	Test 6	<u>0.081</u>	Test 9	<u>0.080</u>
Average <u>0.081</u>					

C. Remarks/Maintenance: ANNUAL INSPECTION, REPLACED SIMULATOR
RETURN AND BREATH HOSE O-RINGS

Instrument is acceptable to be used in the field.

Charles E. Edin
Breath Analyst Signature

4.10.2018
Date

Roberto Yunguer Nunez
Reviewed by

10 Apr 18
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = TOXL 8164.14.00 09/16
04/10/2018 14:23

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=-|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=-|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19817080A4
Standard Cyl #? 21
Standard Expiration? 09/05/2019
Oper No? 666666

Flow Cal. Date: 01/18/2010
Slope 724
Intercept -779308

IR Calibration Date: 04/05/2017

	3um	9um
0th Coef(*100):	-24406	-20805
1st Coef(*100):	261875	135290
2nd Coef(*100):	2828	1427
H2O adj(mg/l*10k):	563	393

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = TOXL 8164.14.00 09/16
04/10/2018 14:25

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:26
02 Std. Sol.	0.019	14:26
03 Room Air	0.000	14:27
04 Std. Sol.	0.020	14:28
05 Room Air	0.000	14:28
06 Std. Sol.	0.020	14:29
07 Room Air	0.000	14:29

08 Sim Temp = 34.0°C

Simul Ser No = MP3002
Std Sol No = 17160
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Low AC
0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = TOXL 8164.14.00 09/16
04/10/2018 14:30

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:30
02 Std. Sol.	0.407	14:31
03 Room Air	0.000	14:32
04 Std. Sol.	0.408	14:32
05 Room Air	0.000	14:33
06 Std. Sol.	0.407	14:34
07 Room Air	0.000	14:34

08 Sim Temp = 34.0°C

Simul Ser No = MP3061

Std Sol No = 201605F

County = 08

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

HIGH AC

0.408 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = TOXL 8164.14.00 09/16
04/10/2018 14:36

Test	AC	Time
01 Room Air	0.000	14:37
02 *Subject Test	INT*	14:37
03 Room Air	0.000	14:38

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 21
Expiration Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = TOXL 8164.14.00 09/16
04/10/2018 14:39

Test	AC	Time
01 Diagnostic	OK	14:40
02 Room Air	0.000	14:40
03 *Subject Test	RFI*	14:40
04 Room Air	0.000	14:41

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 21
Expiration Date = 09/05/2019
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = TOXL 8164.14.00 09/16
04/10/2018 14:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:42
02 Std. Gas	0.081	14:42
03 Room Air	0.000	14:43
04 Std. Gas	0.080	14:43
05 Room Air	0.000	14:43
06 Std. Gas	0.080	14:44
07 Room Air	0.000	14:44

Lot No = 19817080A4

Cyl No = 21

Exp Date = 09/05/2019

County = 08

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = TOXL 8164.14.00 09/16
04/10/2018 14:46

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:46
02 Std. Gas	0.081	14:47
03 Room Air	0.000	14:47
04 Std. Gas	0.081	14:47
05 Room Air	0.000	14:48
06 Std. Gas	0.081	14:48
07 Room Air	0.000	14:49

Lot No = 19817080A4
Cyl No = 21
Exp Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004939
Location = TOXL 8164.14.00 09/16
04/10/2018 14:49

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:50
02 Std. Gas	0.081	14:50
03 Room Air	0.000	14:50
04 Std. Gas	0.081	14:51
05 Room Air	0.000	14:51
06 Std. Gas	0.080	14:52
07 Room Air	0.000	14:52

Lot No = 19817080A4

Cyl No = 21

Exp Date = 09/05/2019

County = 08

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000