

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-004950

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 867 psi Regulator: 875 psi

B. Tests (Sign and attach test records):

- Configure simulator for the following tests (Level 1,S).
1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3002 Lot #: 17160 Exp. Date: 6.8.19
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.400 AC
Sim. Ser #: MP3061 Lot #: 201605F Exp. Date: 5.19.18

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR 7089 Lot #: ICS #5 Exp. Date: 5.15.19
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC,
Gas Cyl Lot #: 19817080A4 Cyl #: 21 Exp. Date: 9/5/19

Test 1	<u>0.080</u>	Test 4	<u>0.080</u>	Test 7	<u>0.080</u>
Test 2	<u>0.080</u>	Test 5	<u>0.081</u>	Test 8	<u>0.080</u>
Test 3	<u>0.081</u>	Test 6	<u>0.081</u>	Test 9	<u>0.080</u>
Average	<u>0.080</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION, REPLACED PRESSURE GAUGE O-RING, REPLACED BREATH HOSE O-RING

Instrument is acceptable to be used in the field.

Charles E. Ed
Breath Analyst Signature

4.10.2018
Date

Roberta Munoz-Nemmo
Reviewed by

10 Apr 18
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.14.00 09/16
04/10/2018 11:39

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19817080A4
Standard Cyl #? 21
Standard Expiration? 09/05/2019
Oper No? 666666

Flow Cal. Date: 01/18/2010
Slope 774
Intercept -908904

IR Calibration Date: 05/05/2016

	3um	9um
0th Coef(*100):	-13717	-19495
1st Coef(*100):	271916	140513
2nd Coef(*100):	3504	1503
H2O adj(mg/l*10k):	370	426

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.14.00 09/16
04/10/2018 11:39

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:40
02 Std. Sol.	0.018	11:41
03 Room Air	0.000	11:41
04 Std. Sol.	0.018	11:42
05 Room Air	0.000	11:42
06 Std. Sol.	0.019	11:43
07 Room Air	0.000	11:44

08 Sim Temp = 34.0°C

Simul Ser No = MP3002

Std Sol No = 17160

County = 08

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Low AC

0.020AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.14.00 09/16
04/10/2018 11:44

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:45
02 Std. Sol.	0.394	11:45
03 Room Air	0.000	11:46
04 Std. Sol.	0.397	11:47
05 Room Air	0.000	11:47
06 Std. Sol.	0.397	11:48
07 Room Air	0.000	11:49

08 Sim Temp = 34.0°C

Simul Ser No = MP3061

Std Sol No = 201605F

County = 08

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.400 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.14.00 09/16
04/10/2018 11:49

Test	AC	Time
01 Room Air	0.000	11:50
02 *Subject Test	INT*	11:50
03 Room Air	0.000	11:51

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 21
Expiration Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10 AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.14.00 09/16
04/10/2018 11:51

Test	AC	Time
01 Diagnostic	OK	11:52
02 Room Air	0.000	11:53
03 *Subject Test	RFI*	11:53
04 Room Air	0.000	11:53

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 21
Expiration Date = 09/05/2019
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI Check

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.14.00 09/16
04/10/2018 11:53

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:54
02 Std. Gas	0.080	11:54
03 Room Air	0.000	11:55
04 Std. Gas	0.080	11:55
05 Room Air	0.000	11:56
06 Std. Gas	0.081	11:56
07 Room Air	0.000	11:57

Lot No = 19817080A4
Cyl No = 21
Exp Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.14.00 09/16
04/10/2018 11:57

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:57
02 Std. Gas	0.080	11:58
03 Room Air	0.000	11:58
04 Std. Gas	0.081	11:58
05 Room Air	0.000	11:59
06 Std. Gas	0.081	11:59
07 Room Air	0.000	12:00

Lot No = 19817080A4
Cyl No = 21
Exp Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = TOXL 8164.14.00 09/16
04/10/2018 12:00

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:01
02 Std. Gas	0.080	12:01
03 Room Air	0.000	12:01
04 Std. Gas	0.080	12:02
05 Room Air	0.000	12:02
06 Std. Gas	0.080	12:03
07 Room Air	0.000	12:03

Lot No = 19817080A4
Cyl No = 21
Exp Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC