

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-003067

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 967 psi Regulator: 975 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3063 Lot #: 16200 Exp. Date: 17 Aug 18
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: MP3065 Lot #: 17350 Exp. Date: 11 Oct 19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR5144 Lot #: 1CS5 Exp. Date: 15 May 19
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 34917080A3 Cyl #: 28 Exp. Date: 5 Feb 20

Test 1 <u>0.080</u>	Test 4 <u>0.081</u>	Test 7 <u>0.081</u>
Test 2 <u>0.080</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

C. Remarks/Maintenance: Annual inspection. Replaced o-ring on
breath hose and simulator return. Completed tank sensor
calibration.

Instrument is acceptable to be used in the field.

Roberta Sueger-Nimmo
Breath Analyst Signature

18 Apr 18
Date

Charles E. Eden
Reviewed by

18 APR 18
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
04/18/2018 11:29

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 34917080A3
Standard Cyl #? 28
Standard Expiration? 02/05/2020
Oper No? 888888

Flow Cal. Date: 01/23/2013
Slope 664
Intercept -509770

IR Calibration Date: 04/30/2015

	3um	9um
0th Coef(*100):	-19633	-12115
1st Coef(*100):	278224	137001
2nd Coef(*100):	2596	1415
H2O adj(mg/l*10k):	323	247

***** Printer Test End *****

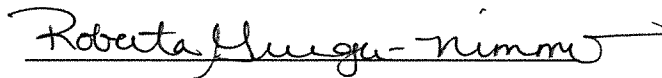

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Print test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
04/18/2018 12:05

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:06
02 Std. Sol.	0.021	12:07
03 Room Air	0.000	12:07
04 Std. Sol.	0.021	12:08
05 Room Air	0.000	12:09
06 Std. Sol.	0.020	12:09
07 Room Air	0.000	12:10

08 Sim Temp = 34.0°C

Simul Ser No = MP3063
Std Sol No = 16200
County = 08 Oper No. = 888888



Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: LOW AC
0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
04/18/2018 11:59

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:00
02 Std. Sol.	0.298	12:01
03 Room Air	0.000	12:02
04 Std. Sol.	0.298	12:03
05 Room Air	0.000	12:03
06 Std. Sol.	0.297	12:04
07 Room Air	0.000	12:04

08 Sim Temp = 34.0°C

Simul Ser No = MP3065
Std Sol No = 17350
County = 08 Oper No. = 888888

Roberta Grieger-Nimmo
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC
0.300 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
04/18/2018 11:44

Test	AC	Time
01 Room Air	0.000	11:45
02 *Subject Test	INT*	11:45
03 Room Air	0.000	11:46

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 34917080A3
Cyl No = 28
Expiration Date = 02/05/2020
County = 08 Oper No. = 888888

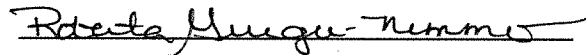

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Interferent Check
0.10 AC + 0.05% Acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
04/18/2018 11:46

Test	AC	Time
01 Diagnostic	OK	11:47
02 Room Air	0.000	11:48
03 *Subject Test	RFI*	11:48
04 Room Air	0.000	11:48

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 34917080A3
Cyl No = 28
Expiration Date = 02/05/2020
County = 08 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Roberta Grieger-Nimmo
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
04/18/2018 11:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:34
02 Std. Gas	0.080	11:34
03 Room Air	0.000	11:34
04 Std. Gas	0.080	11:35
05 Room Air	0.000	11:35
06 Std. Gas	0.080	11:36
07 Room Air	0.000	11:36

Lot No = 34917080A3
Cyl No = 28
Exp Date = 02/05/2020
County = 08 Oper No. = 888888

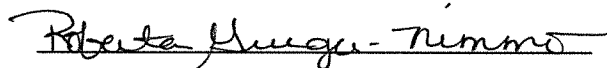
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
04/18/2018 11:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:37
02 Std. Gas	0.081	11:37
03 Room Air	0.000	11:38
04 Std. Gas	0.081	11:38
05 Room Air	0.000	11:39
06 Std. Gas	0.080	11:39
07 Room Air	0.000	11:40

Lot No = 34917080A3
Cyl No = 28
Exp Date = 02/05/2020
County = 08 Oper No. = 888888

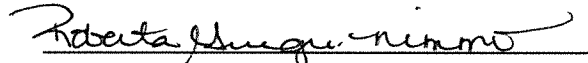
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check

0.080 AC
Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
04/18/2018 11:40

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:41
02 Std. Gas	0.081	11:41
03 Room Air	0.000	11:42
04 Std. Gas	0.081	11:42
05 Room Air	0.000	11:42
06 Std. Gas	0.080	11:43
07 Room Air	0.000	11:43

Lot No = 34917080A3
Cyl No = 28
Exp Date = 02/05/2020
County = 08 Oper No. = 888888

Roberta Grieger-Nimmo
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC