

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-003061

Location: TXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 999 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3063 Lot #: 16200 Exp. Date: 11 Oct 19
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: MP3065 Lot #: 17350 Exp. Date: 17 Aug 18

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR5144 Lot #: 1C55 Exp. Date: 15 May 19
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 34917080A3 Cyl #: 28 Exp. Date: 5 Feb 20

Test 1 <u>0.079</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.079</u>	Test 5 <u>0.080</u>	Test 8 <u>0.079</u>
Test 3 <u>0.079</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

C. Remarks/Maintenance: Annual inspection. Replaced o-ring on Simulator return and breath hose.

Instrument is acceptable to be used in the field.

Roberta Hanger-Morris
Breath Analyst Signature

18 Apr 18
Date

Charles E. Edr
Reviewed by

19 APR 18
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/18/2018 14:35

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer (Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 34917080A3
Standard Cyl #? 28
Standard Expiration? 02/05/2020
Oper No? 888888

Flow Cal. Date: 03/04/2011
Slope 700
Intercept -623726

IR Calibration Date: 04/10/2015
 3um 9um

0th Coef(*100): -25722 -17879
1st Coef(*100): 266163 136703
2nd Coef(*100): 2302 1151
H2O adj(mg/l*10k): 412 294

***** Printer Test End *****

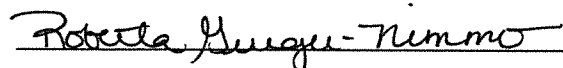

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Print Test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/18/2018 14:53

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:54
02 Std. Sol.	0.020	14:55
03 Room Air	0.000	14:55
04 Std. Sol.	0.019	14:56
05 Room Air	0.000	14:56
06 Std. Sol.	0.019	14:57
07 Room Air	0.000	14:58


08 Sim Temp = 34.0°C

Simul Ser No = MP3063

Std Sol No = 16200

County = 08

Oper No. = 888888

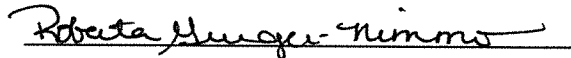

Operator Signature

ROBERTA GRIEGER-NIMMO

Remarks: Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/18/2018 15:09

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:09
02 Std. Sol.	0.293	15:10
03 Room Air	0.000	15:11
04 Std. Sol.	0.294	15:11
05 Room Air	0.000	15:12
06 Std. Sol.	0.294	15:13
07 Room Air	0.000	15:13

08 Sim Temp = 34.0°C

Simul Ser No = MP3065
Std Sol No = 17350
County = 08 Oper No. = 888888

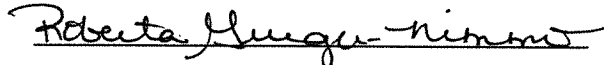

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC
0.300AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/18/2018 14:37

Test	AC	Time
01 Room Air	0.000	14:38
02 *Subject Test	INT*	14:38
03 Room Air	0.000	14:39

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 34917080A3
Cyl No = 28
Expiration Date = 02/05/2020
County = 08 Oper No. = 888888

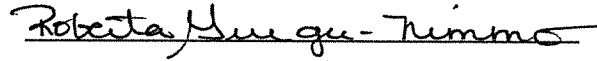
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Interferent Check*
0.080 AC + 0.05% Acetone
0.10
Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/18/2018 14:39

Test	AC	Time
01 Diagnostic	OK	14:40
02 Room Air	RFI*	14:40
03 Room Air	0.000	14:41

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 34917080A3
Cyl No = 28
Expiration Date = 02/05/2020
County = 08 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

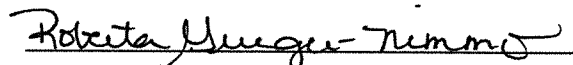

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *RFI Check*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/18/2018 14:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:41
02 Std. Gas	0.079	14:42
03 Room Air	0.000	14:42
04 Std. Gas	0.079	14:43
05 Room Air	0.000	14:43
06 Std. Gas	0.079	14:44
07 Room Air	0.000	14:44


Lot No = 34917080A3

Cyl No = 28

Exp Date = 02/05/2020

County = 08

Oper No. = 888888

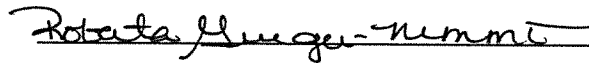

Operator Signature

ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check

0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/18/2018 14:44

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:45
02 Std. Gas	0.080	14:45
03 Room Air	0.000	14:46
04 Std. Gas	0.080	14:46
05 Room Air	0.000	14:47
06 Std. Gas	0.080	14:47
07 Room Air	0.000	14:47

Lot No = 34917080A3
Cyl No = 28
Exp Date = 02/05/2020
County = 08 Oper No. = 888888

Roberta Grieger-Nimmo
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080AC
Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/18/2018 14:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:48
02 Std. Gas	0.080	14:49
03 Room Air	0.000	14:49
04 Std. Gas	0.079	14:49
05 Room Air	0.000	14:50
06 Std. Gas	0.080	14:50
07 Room Air	0.000	14:51

Lot No = 34917080A3
Cyl No = 28
Exp Date = 02/05/2020
County = 08 Oper No. = 888888

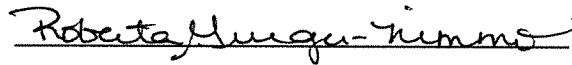

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080AC

Form 106-I8000