

INTOXILYZER® 8000 INSPECTION *Burleigh Morton Detention Center*

Intoxilyzer® 8000 Serial Number: 80-005952 Location: BCSO

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 397 psi Regulator: 400 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR3377 Lot #: 201610A Exp. Date: 4 Oct 18
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR5191 Lot #: 17350 Exp. Date: 11 Oct 19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: 1CS5 Exp. Date: 15 May 19
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 03417080A Cyl #: 36 Exp. Date: 5 Mar 19

Test 1	<u>0.083</u>	Test 4	<u>0.083</u>	Test 7	<u>0.083</u>
Test 2	<u>0.083</u>	Test 5	<u>0.082</u>	Test 8	<u>0.083</u>
Test 3	<u>0.082</u>	Test 6	<u>0.082</u>	Test 9	<u>0.082</u>
Average <u>0.083</u>					

C. Remarks/Maintenance: Annual inspection. Replaced o-ring on breath hose.

Instrument is acceptable to be used in the field.

Roberto Murgu - Nimmo
Breath Analyst Signature

26 Apr 18
Date

Charles E. Eden
Reviewed by

26 APR. 2018
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.14.00 09/16
04/26/2018 11:19

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup

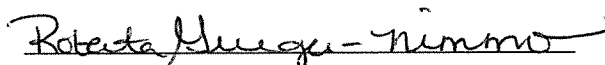
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 03417080A3
Standard Cyl #? 36
Standard Expiration? 03/05/2019
Oper No? 888888

Flow Cal. Date: 06/21/2013
Slope 734
Intercept -865305

IR Calibration Date: 06/21/2013
 3um 9um

0th Coef(*100): -21281 -23057
1st Coef(*100): 292132 137092
2nd Coef(*100): 2334 1231
H2O adj(mg/l*10k): 497 482

***** Printer Test End *****


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Print Test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

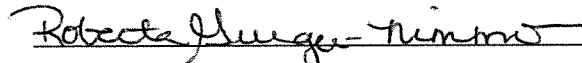
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.14.00 09/16
04/26/2018 11:38

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:38
02 Std. Sol.	0.019	11:39
03 Room Air	0.000	11:40
04 Std. Sol.	0.020	11:40
05 Room Air	0.000	11:41
06 Std. Sol.	0.019	11:42
07 Room Air	0.000	11:42

08 Sim Temp = 34.0°C

Simul Ser No = DR3377
Std Sol No = 201610A
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: low AC
 0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

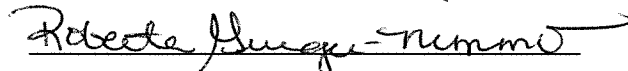
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.14.00 09/16
04/26/2018 11:44

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:44
02 Std. Sol.	0.298	11:45
03 Room Air	0.000	11:46
04 Std. Sol.	0.299	11:46
05 Room Air	0.000	11:47
06 Std. Sol.	0.299	11:48
07 Room Air	0.000	11:48

08 Sim Temp = 34.0°C

Simul Ser No = DR5191
Std Sol No = 17350
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC
 0.300 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.14.00 09/16
04/26/2018 11:32

Test	AC	Time
01 Room Air	0.000	11:33
02 *Subject Test	INT*	11:33
03 Room Air	0.000	11:34

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 03417080A3
Cyl No = 36
Expiration Date = 03/05/2019
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Interferent Check
0.10AC + 0.05% Acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

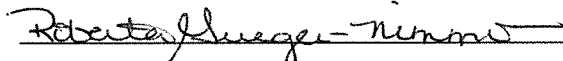
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.14.00 09/16
04/26/2018 11:30

Test	AC	Time
01 Diagnostic	OK	11:31
02 Room Air	0.000	11:31
03 *Subject Test	RFI*	11:31
04 Room Air	0.000	11:32

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 03417080A3
Cyl No = 36
Expiration Date = 03/05/2019
County = 08 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check

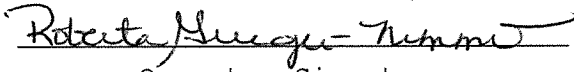
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.14.00 09/16
04/26/2018 11:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:20
02 Std. Gas	0.083	11:20
03 Room Air	0.000	11:21
04 Std. Gas	0.083	11:21
05 Room Air	0.000	11:21
06 Std. Gas	0.082	11:22
07 Room Air	0.000	11:22

Lot No = 03417080A3
Cyl No = 36
Exp Date = 03/05/2019
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.14.00 09/16
04/26/2018 11:23

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:23
02 Std. Gas	0.083	11:24
03 Room Air	0.000	11:24
04 Std. Gas	0.082	11:25
05 Room Air	0.000	11:25
06 Std. Gas	0.082	11:25
07 Room Air	0.000	11:26

Lot No = 03417080A3

Cyl No = 36

Exp Date = 03/05/2019

County = 08

Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check

0.080 AC

Form 106-I8000

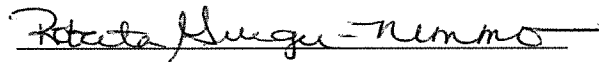
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.14.00 09/16
04/26/2018 11:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:27
02 Std. Gas	0.083	11:27
03 Room Air	0.000	11:28
04 Std. Gas	0.083	11:28
05 Room Air	0.000	11:29
06 Std. Gas	0.082	11:29
07 Room Air	0.000	11:29

Lot No = 03417080A3
Cyl No = 36
Exp Date = 03/05/2019
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
 0.080 AC