



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (10-2016)

Serial Number 80-005946	Instrument Location Williams County LEC
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): None (moved machine)
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature Darryn Abele	Date 05-03-18
--	-------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by Robert Huggins	Certified Date 05-11-18
--	-----------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005946
Location = WILL 8164.14.00 09/16
05/03/2018 09:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:26
02 Std. Gas	0.081	09:26
03 Room Air	0.000	09:26
04 Std. Gas	0.080	09:27
05 Room Air	0.000	09:27
06 Std. Gas	0.079	09:28
07 Room Air	0.000	09:28

Lot No = 03417080A3
Cyl No = 34
Exp Date = 03/05/2019
County = 53 Oper No. = 010101



Operator Signature
DARCY ABERLE

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005946
Location = WILL 8164.14.00 09/16
05/03/2018 09:29

Test	AC	Time
01 Diagnostic	OK	09:31
02 Room Air	RFI*	09:31
03 Room Air	0.000	09:31

*Invalid Test
Inhibited - RFI

Sub Name = TEST, TEST TEST
Sub DOB = 09/09/1909
Sub Sex = Male Weight = 150
Test = DUI Cit = 000000
Dr. Lic. = ND/NA
Lot No = 03417080A3
Cyl No = 34
Expiration Date = 03/05/2019
County = 53 Oper No. = 010101

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DARCY ABERLE

Remarks: