



# ETHANOL GAS STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION  
Toxicology Section/Breath Alcohol Program  
SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>CHARLES E. EDER, STATE TOX., ANN. INSP.</i>		
Location <i>OLIV OLIVER COUNTY S.O.</i>	Intoxilyzer® Serial Number <i>80-005956</i>	
Gas Lot Number <i>34917080A3</i>	Gas Cylinder Number <i>34</i>	Gas Expiration Date <i>2/5/2020</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
  - A. 0.077 AC
  - B. 0.078 AC
  - C. 0.078 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>Charles E. Eder</i>	Date <i>5-4-2018</i>
Reviewed By (Crime Laboratory Use Only) <i>Roberta Guigou-Thummo</i>	Date <i>5-8-2018</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005956  
Location = OLIV      8164.14.00 09/16  
05/04/2018      11:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:25
02 Std. Gas	0.077	11:26
03 Room Air	0.000	11:26
04 Std. Gas	0.078	11:27
05 Room Air	0.000	11:27
06 Std. Gas	0.078	11:27
07 Room Air	0.000	11:28

Lot No = 34917080A3  
Cyl No = 34  
Exp Date = 02/05/2020  
County = 33      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

ACA CHECK