



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program

SFN 59281 (10-2016)

| | |
|---|--|
| Serial Number 80-005940 | Instrument Location New Town |
| Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____ | |

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000. *No line as of yet*
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *install all good*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

| | |
|--|-------------------------|
| Field Inspector Signature <i>Dustin Krueger</i> | Date <i>5-8-2018</i> |
|--|-------------------------|

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

| | |
|--|------------------------------------|
| Reviewed/Certified by <i>Roberta Mueger Nimmo</i> | Certified Date <i>5-15-2018</i> |
|--|------------------------------------|

10


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005940
Location = NEWT 8164.14.00 09/16
05/08/2018 17:02

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 17:03 |
| 02 Std. Gas | 0.079 | 17:03 |
| 03 Room Air | 0.000 | 17:04 |
| 04 Std. Gas | 0.079 | 17:04 |
| 05 Room Air | 0.000 | 17:05 |
| 06 Std. Gas | 0.079 | 17:05 |
| 07 Room Air | 0.000 | 17:06 |

Lot No = 17316080A2
Cyl No = 35
Exp Date = 08/05/2018
County = 60 Oper No. = 130472

Dustin Krueger 
Operator Signature
DUSTIN KRUEGER

Remarks: *ACA Test*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005940
Location = NEWT 8164.14.00 09/16
05/08/2018 17:06

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | RFI* | 17:07 |
| 02 Room Air | 0.000 | 17:08 |

*Invalid Test
Inhibited - RFI

Sub Name = KRUEGER, DUSTIN CAL
Sub DOB = 08/08/1979
Sub Sex = Male Weight = 195
Test = OTH Cit = 582018
Dr. Lic. = ND/KRU796350
Lot No = 17316080A2
Cyl No = 35
Expiration Date = 08/05/2018
County = 60 Oper No. = 130472

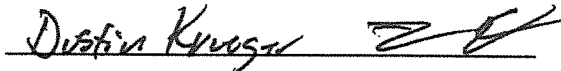

Operator Signature
DUSTIN KRUEGER

Remarks: *RFI Test*