

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005945

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 822 psi Regulator: 825 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3063 Lot #: 16200 Exp. Date: 17 Aug 18
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: MP3065 Lot #: 17350 Exp. Date: 11 Oct 19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR5144 Lot #: 1CS5 Exp. Date: 15 May 19

Display reads "Interferent Detect"

3. RFI Check (CMS Mode) Display reads "RFI Detect"

4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC

Gas Cyl Lot #: 34917080A3 Cyl #: 28 Exp. Date: 5 Feb 20

Test 1 0.078 Test 4 0.078 Test 7 0.079

Test 2 0.079 Test 5 0.079 Test 8 0.079

Test 3 0.078 Test 6 0.078 Test 9 0.078

Average 0.078

C. Remarks/Maintenance: Replaced O-ring on breath hose. Replaced and trained external battery. Completed tank sensor calibration.

Instrument is acceptable to be used in the field.

Roberta Pungu-Nunno
Breath Analyst Signature

10 May 18
Date

Charles E. Edin
Reviewed by

11 MAY 18
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = TOXL 8164.14.00 09/16
05/09/2018 15:20

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 34917080A3
Standard Cyl #? 28
Standard Expiration? 02/05/2020
Oper No? 888888

Flow Cal. Date: 06/20/2013
Slope 637
Intercept -566407

IR Calibration Date: 05/22/2015

	3um	9um
0th Coef(*100):	-21213	-23093
1st Coef(*100):	279171	139292
2nd Coef(*100):	2898	1291
H2O adj(mg/l*10k):	332	319

***** Printer Test End *****

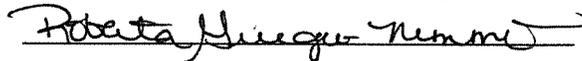

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Print Test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = TOXL 8164.14.00 09/16
05/10/2018 14:30

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:31
02 Std. Sol.	0.019	14:31
03 Room Air	0.000	14:32
04 Std. Sol.	0.020	14:33
05 Room Air	0.000	14:33
06 Std. Sol.	0.019	14:34
07 Room Air	0.000	14:34


08 Sim Temp = 34.0°C

Simul Ser No = MP3063

Std Sol No = 16200

County = 08

Oper No. = 888888

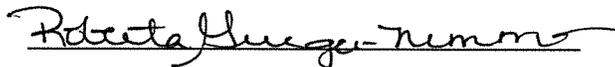

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = TOXL 8164.14.00 09/16
05/10/2018 14:36

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:36
02 Std. Sol.	0.301	14:37
03 Room Air	0.000	14:38
04 Std. Sol.	0.303	14:39
05 Room Air	0.000	14:39
06 Std. Sol.	0.301	14:40
07 Room Air	0.000	14:40


08 Sim Temp = 34.0°C

Simul Ser No = MP3065

Std Sol No = 17350

County = 08

Oper No. = 888888

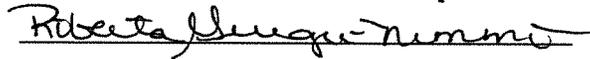

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = TOXL 8164.14.00 09/16
05/09/2018 15:48

Test	AC	Time
01 Room Air	0.000	15:48
02 *Subject Test	INT*	15:49
03 Room Air	0.000	15:50

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 34917080A3
Cyl No = 28
Expiration Date = 02/05/2020
County = 08 Oper No. = 888888

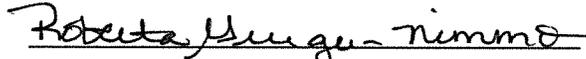

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Interferent Check
0.10 AC + 0.059% Acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = TOXL 8164.14.00 09/16
05/09/2018 15:50

Test	AC	Time
01 Diagnostic	OK	15:51
02 Room Air	0.000	15:52
03 Subject Test 1	0.000	15:52
04 Room Air	0.000	15:54
05 Std. Gas	0.077	15:55
06 Room Air	RFI*	15:56
07 Room Air	0.000	15:56

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 34917080A3
Cyl No = 28
Expiration Date = 02/05/2020
County = 08 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

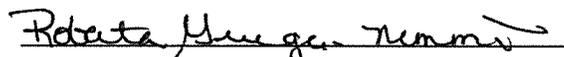
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = TOXL 8164.14.00 09/16
05/09/2018 15:21

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:21
02 Std. Gas	0.078	15:21
03 Room Air	0.000	15:22
04 Std. Gas	0.079	15:22
05 Room Air	0.000	15:23
06 Std. Gas	0.078	15:23
07 Room Air	0.000	15:24

Lot No = 34917080A3
Cyl No = 28
Exp Date = 02/05/2020
County = 08 Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Calibration Check*

0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = TOXL 8164.14.00 09/16
05/09/2018 15:24

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:25
02 Std. Gas	0.078	15:25
03 Room Air	0.000	15:26
04 Std. Gas	0.079	15:26
05 Room Air	0.000	15:27
06 Std. Gas	0.078	15:27
07 Room Air	0.000	15:27

Lot No = 34917080A3
Cyl No = 28
Exp Date = 02/05/2020
County = 08 Oper No. = 888888

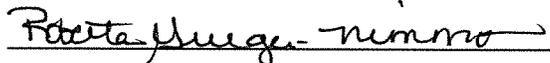
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = TOXL 8164.14.00 09/16
05/09/2018 15:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:32
02 Std. Gas	0.079	15:33
03 Room Air	0.000	15:33
04 Std. Gas	0.079	15:34
05 Room Air	0.000	15:34
06 Std. Gas	0.078	15:34
07 Room Air	0.000	15:35

Lot No = 34917080A3
Cyl No = 28
Exp Date = 02/05/2020
County = 08 Oper No. = 888888

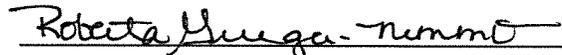
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC