

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005955

Location: DUNN
DUNN COUNTY
S.O. CEE

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 971 psi Regulator: 975 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 ~~0.000~~ AC
Sim. Ser #: DR3377 Lot #: 201610A Exp. Date: 10.4.18
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR5191 Lot #: 17350 Exp. Date: 10.11.19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: ICS #5 Exp. Date: 5.15.19
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 03417080A3 Cyl #: 31 Exp. Date: 3.5.19

Test 1	<u>0.082</u>	Test 4	<u>0.081</u>	Test 7	<u>0.082</u>
Test 2	<u>0.082</u>	Test 5	<u>0.081</u>	Test 8	<u>0.081</u>
Test 3	<u>0.081</u>	Test 6	<u>0.081</u>	Test 9	<u>0.081</u>
Average	<u>0.081</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Edr
Breath Analyst Signature

5.7.2018
Date

Roberta Yungu-Timm
Reviewed by

14 May 18
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.14.00 09/16
05/07/2018 08:53

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 03417080A3
Standard Cyl #? 31
Standard Expiration? 03/05/2019
Oper No? 666666

Flow Cal. Date: 06/21/2013
Slope 757
Intercept -837385

IR Calibration Date: 06/21/2013

	3um	9um
0th Coef(*100):	-19897	-22878
1st Coef(*100):	268381	131979
2nd Coef(*100):	5157	1473
H2O adj(mg/l*10k):	717	600

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.14.00 09/16
05/07/2018 09:14

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:14
02 Std. Sol.	0.017	09:15
03 Room Air	0.000	09:16
04 Std. Sol.	0.017	09:16
05 Room Air	0.000	09:17
06 Std. Sol.	0.017	09:18
07 Room Air	0.000	09:18

08 Sim Temp = 34.0°C

Simul Ser No = DR3377

Std Sol No = 201610A

County = 13

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Low AC

0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.14.00 09/16
05/07/2018 09:19

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:20
02 Std. Sol.	0.310	09:21
03 Room Air	0.000	09:21
04 Std. Sol.	0.313	09:22
05 Room Air	0.000	09:22
06 Std. Sol.	0.310	09:23
07 Room Air	0.000	09:24

08 Sim Temp = 34.0°C

Simul Ser No = DR5191

Std Sol No = 17350

County = 13

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.14.00 09/16
05/07/2018 09:10

Test	AC	Time
01 Room Air	0.000	09:11
02 *Subject Test	INT*	09:12
03 Room Air	0.000	09:12

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 03417080A3
Cyl No = 31
Expiration Date = 03/05/2019
County = 13 Oper No. = 666666

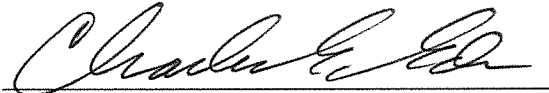

Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10 AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.14.00 09/16
05/07/2018 08:53

Test	AC	Time
01 Diagnostic	OK	08:55
02 Room Air	0.000	08:55
03 *Subject Test	RFI*	08:55
04 Room Air	0.000	08:56

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 03417080A3
Cyl No = 31
Expiration Date = 03/05/2019
County = 13 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.14.00 09/16
05/07/2018 08:57

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:57
02 Std. Gas	0.082	08:58
03 Room Air	0.000	08:58
04 Std. Gas	0.082	08:59
05 Room Air	0.000	08:59
06 Std. Gas	0.081	08:59
07 Room Air	0.000	09:00

Lot No = 03417080A3
Cyl No = 31
Exp Date = 03/05/2019
County = 13 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.14.00 09/16
05/07/2018 09:06

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:06
02 Std. Gas	0.081	09:07
03 Room Air	0.000	09:07
04 Std. Gas	0.081	09:07
05 Room Air	0.000	09:08
06 Std. Gas	0.081	09:08
07 Room Air	0.000	09:09

Lot No = 03417080A3
Cyl No = 31
Exp Date = 03/05/2019
County = 13 Oper No. = 666666

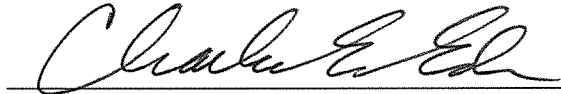

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = DUNN 8164.14.00 09/16
05/07/2018 09:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:27
02 Std. Gas	0.082	09:27
03 Room Air	0.000	09:28
04 Std. Gas	0.081	09:28
05 Room Air	0.000	09:29
06 Std. Gas	0.081	09:29
07 Room Air	0.000	09:30

Lot No = 03417080A3
Cyl No = 31
Exp Date = 03/05/2019
County = 13 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK

0.080 AC

Form 106-I8000