

**INTOXILYZER® 8000 INSPECTION**

Intoxilyzer® 8000 Serial Number: 80-006489

Location: GFPD  
GRAND FORKS PD

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 375 psi Regulator: 400 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC  
Sim. Ser #: DR3377 Lot #: 201610A Exp. Date: 10.4.18
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.300 AC  
Sim. Ser #: DR5191 Lot #: 17350 Exp. Date: 10.11.19

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR3451 Lot #: ICS#5 Exp. Date: 5.15.19  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 20016080A2 Cyl #: 51 Exp. Date: 9/5/18

Test 1	<u>0.080</u>	Test 4	<u>0.082</u>	Test 7	<u>0.082</u>
Test 2	<u>0.082</u>	Test 5	<u>0.082</u>	Test 8	<u>0.082</u>
Test 3	<u>0.082</u>	Test 6	<u>0.082</u>	Test 9	<u>0.082</u> 3
Average	<u>0.082</u>				<u>CEE</u>

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Eda  
Breath Analyst Signature

5-8-2018  
Date

Roberto Mueque-Timono  
Reviewed by

14 May 18  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006489  
Location = GFPD                    8164.14.00 09/16  
05/10/2018                            16:30

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%&\*()\_+?

Current Instrument Setup


Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            20016080A2  
Standard Cyl #?            051  
Standard Expiration?        09/05/2018  
Oper No?                    666666

Flow Cal. Date:            08/18/2015  
Slope                      663  
Intercept                  -631787

IR Calibration Date:        08/18/2015  
                                  3um                    9um

-----  
0th Coef(\*100):            -24626                -16551  
1st Coef(\*100):            269218                130789  
2nd Coef(\*100):            3390                   1656  
H2O adj(mg/l\*10k):        742                    460

\*\*\*\*\* Printer Test End \*\*\*\*\*

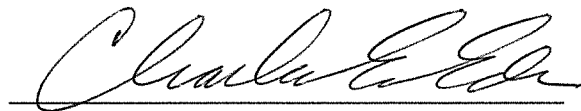

Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006489  
Location = GFPD      8164.14.00 09/16  
05/10/2018      16:44

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:44
02 Std. Sol.	0.019	16:45
03 Room Air	0.000	16:46
04 Std. Sol.	0.019	16:46
05 Room Air	0.000	16:47
06 Std. Sol.	0.019	16:48
07 Room Air	0.000	16:48

08 Sim Temp = 34.0°C

Simul Ser No = DR3377

Std Sol No = 201610A

County = 18

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

Low AC

0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006489  
Location = GFPD      8164.14.00 09/16  
05/10/2018      16:50

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:50
02 Std. Sol.	0.295	16:51
03 Room Air	0.000	16:52
04 Std. Sol.	0.295	16:53
05 Room Air	0.000	16:53
06 Std. Sol.	0.297	16:54
07 Room Air	0.000	16:55

08 Sim Temp = 34.0°C

Simul Ser No = DR5191  
Std Sol No = 17350  
County = 18      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: HIGH AC  
0.300 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006489  
Location = GFPD                    8164.14.00 09/16  
05/10/2018                            16:33

Test	AC	Time
01 Room Air	0.000	16:34
02 *Subject Test	INT*	16:34
03 Room Air	0.000	16:35

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 03/01/1982  
Sub Sex = Male                            Weight = 120  
Test = DUI                                Cit = INTERFERENT  
Dr. Lic. = ND/DIS821449  
Lot No = 20016080A2  
Cyl No = 051  
Expiration Date = 09/05/2018  
County = 18                                Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent Check*  
*0.10AC + 0.05% ACETONE*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006489  
Location = GFPD                    8164.14.00 09/16  
05/10/2018                            16:31

Test	AC	Time
01 Diagnostic	OK	16:32
02 Room Air	0.000	16:32
03 *Subject Test	RFI*	16:32
04 Room Air	0.000	16:33

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 03/01/1982  
Sub Sex = Male                    Weight = 120  
Test = DUI                        Cit = RFI CHECK  
Dr. Lic. = ND/DIS821449  
Lot No = 20016080A2  
Cyl No = 051  
Expiration Date = 09/05/2018  
County = 18                        Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006489  
Location = GFPD      8164.14.00 09/16  
05/10/2018      16:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:36
02 Std. Gas	0.080	16:36
03 Room Air	0.000	16:37
04 Std. Gas	0.082	16:37
05 Room Air	0.000	16:38
06 Std. Gas	0.082	16:38
07 Room Air	0.000	16:39

Lot No = 20016080A2  
Cyl No = 051  
Exp Date = 09/05/2018  
County = 18      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-18000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006489  
Location = GFPD      8164.14.00 09/16  
05/10/2018      16:39

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:40
02 Std. Gas	0.082	16:40
03 Room Air	0.000	16:41
04 Std. Gas	0.082	16:41
05 Room Air	0.000	16:42
06 Std. Gas	0.082	16:42
07 Room Air	0.000	16:43

Lot No = 20016080A2  
Cyl No = 051  
Exp Date = 09/05/2018  
County = 18      Oper No. = 666666

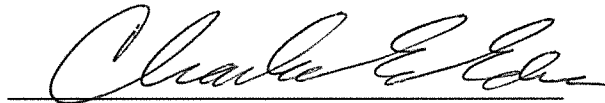

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006489  
Location = GFPD      8164.14.00 09/16  
05/10/2018      16:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:56
02 Std. Gas	0.082	16:57
03 Room Air	0.000	16:57
04 Std. Gas	0.082	16:58
05 Room Air	0.000	16:58
06 Std. Gas	0.083	16:59
07 Room Air	0.000	16:59

Lot No = 20016080A2  
Cyl No = 51  
Exp Date = 09/05/2018  
County = 18      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000